

Spirit[®]







**PATIENT
OPERATING
INSTRUCTIONS**


SPIRIT[®] 300/600/1200

Spirit® Portable

Warning Information

Read this page carefully before using your unit.

-  • The unit contains liquid oxygen which is extremely cold; almost 300 degrees below zero Fahrenheit. Exposure to such a low temperature can cause severe frostbite.
-  • Liquid and gaseous oxygen, though nonflammable, cause other materials to burn faster than normal. This hazard, along with the low temperature of liquid oxygen, warrants certain safety precautions.
- Do not use or store your unit in an area where combustible materials such as oils, greases, aerosol sprays, lotions or solvents are present.
-  • **Do not smoke while operating unit.**
- Do not use or store your unit within five feet of electrical appliances, especially stoves, heaters, toasters, and hair dryers.
-  • Keep your unit in a well-ventilated area.

- In the event of an accidental tip-over, immediately but cautiously return unit into an upright position. If any liquid oxygen is escaping, leave the area immediately and call your health care provider.
-  • **Do not touch the frosted parts of any unit.**
- Do not store or operate the Spirit® coupled to the stationary unit.
- Applicable EMC information can be found in the Spirit® Service Manual at www.cairemedical.com.

Explanation of Other Symbols



Name and address of manufacturer



Authorized representative in the European Community



Caution, consult accompanying documents



Non-flammable gas



Oxidizing substances



Type B Applied Part (degree of protection against electric shock).

Patient's Records

Patient's Name _____

Prescribed Oxygen Flow Setting _____

Doctor's Telephone Number _____

Distributor's Name _____

Person to Contact _____

Distributor's Emergency Telephone Number _____

Special Instructions _____

Spirit[®] Portable

Table Of Contents

Introduction	4
Physician Notes	5
Controls	6-7
Filling Instructions	8-11
Operating Instructions	12-17
Additional Information	18

Specifications:

- Mode of Operation: Continuous
- Type of Protection Against Electric Shock: Internally Powered Equipment
- Degree of Protection Against Electric Shock: Type B Applied Part
- Classification according to the degree of protection against ingress of water: IPX0 - Ordinary Equipment
- Equipment is not suitable for use in the presence of flammable mixtures

Important: *Read this manual thoroughly before operating the Spirit[®] unit.*

Rx only.

Warning: *This device is not intended for life sustaining use.*

Caution: *Use the Spirit[®] only as directed by your doctor.*

Warning: *If you feel the equipment is not operating properly, call your health care provider. Do not attempt to repair or adjust the units yourself.*

Introduction

The liquid oxygen system includes a stationary unit and the Spirit[®], which provides supplementary oxygen as prescribed by your physician. This POI contains the instructions for using the Spirit[®]. Refer to the POI supplied with the stationary unit for its operation.

The Liberator[®] is the larger container and is intended for stationary use. You may take oxygen directly from the Liberator[®]. The Liberator[®] is filled by your health care provider.

The Spirit[®] Portable provides an ambulatory source of oxygen for an extended period of time. You fill the Spirit[®] from a liquid oxygen stationary unit. The compact, lightweight Spirit[®] was developed to increase mobility and improve comfort of the typical supplemental oxygen patient.



Spirit[®] 300 Shown

Spirit[®] Portable

Physician Notes

1. Do not use with Patients who breathe below 12 breaths per minute (BPM) or above 40 BPM.
2. Do not use with Patients who consistently fail to trigger equipment (i.e. very light mouth breathers).
3. **Verify Patient is getting adequate PaO₂ or SaO₂ levels, while using the Spirit[®].**
4. Use only a standard, single tube, 7' (or shorter) nasal cannula. Do not use a pediatric (low flow) nasal cannula or mask.
5. Do not use other equipment (i.e. humidifier, nebulizer, etc.) in conjunction with the Spirit[®].

Spirit[®] Portable



Controls

1. Contents Indicator
2. Flow Rate Selection Knob
3. Oxygen Outlet Connector
4. Fill Connections
(See next page for QDV Identification)
5. Vent Valve
6. Batteries (2) C-Cell (inside case)

Spirit[®] Portable

Quick Disconnect Valve (QDV) Identification

Portable
Connector:



Stationary
Connector:



QDV Style:

CAIRE
Side Fill

LINCARE
(UC)

CRYO2

LIFE-OX

PENOX

TAEMA

PB

SF Locking

TF Locking

TF Push-on

To Fill the Top Fill Spirit[®]:

1. Clean the fill connectors on both the stationary and the Spirit[®] with a clean, dry, lint-free cloth.

Caution: *The fill connectors must be dry, because moisture can cause your Spirit[®] and stationary unit to freeze together.*

2. Make sure the flow control knobs on both the Spirit[®] and reservoir are in the off (0) position.
3. Position the fill connector in the upright position over the stationary unit's connector.



Step 3


4. a. If your connector is a push-on : Lower the Spirit[®] unit on to the stationary until you feel the connector engage.

- b. If your connector is Cryo2, then rotate Spirit[®] until you feel the connectors engage. Then, carefully and firmly, rotate the Spirit[®] counterclockwise (↺) while pressing downward.
 - c. If the connector is a Penox, Life-Ox, or Lincare: Rotate Spirit[®] until you feel the connector engage. Then, carefully and firmly, rotate the Spirit[®] clockwise (↻) while pressing downward.
5. Now the units are locked together

Note: *Continued on step 6 (page 10).*



Step 4a

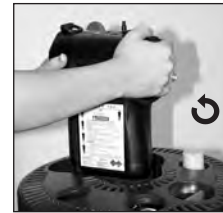

Step 4b



Step 4c

Spirit® Portable

To Fill the Side Fill Spirit®:

1. Clean the fill connectors on both the stationary and the Spirit® with a clean, dry, lint-free cloth.

Caution: *The fill connectors must be dry, because moisture can cause your Spirit® and stationary unit to freeze together.*

2. Make sure the flow control knobs on both the Spirit® and reservoir are in the off (0) position
3. Mate the Spirit® fill connector in the upright position to the stationary connector.
4. Rotate the Spirit® counterclockwise (↺) until you feel the pin and slot engage.
5. Carefully and firmly rotate the Spirit® back to the upright position. Now the units are locked together.

Note: *Continued on step 6*



Step 3



Step 4



Step 5

Note: For push-on style connector, hold the unit in place throughout the fill.

6. Open the vent valve to begin filling your Spirit[®]. You should hear some oxygen escaping; do not worry. The hissing sound you hear is perfectly normal.
7. You may also see vapor around the connection. This is normal and is due to the great difference in temperature between the liquid oxygen and the warm air surrounding the units.
8. When the unit is full, close the vent valve. You can tell that the unit is full when the hissing sound changes in tone, and some liquid comes out of the vent line.

Note: Closing the valve after a nearly continuous stream of liquid is discharging fills the Spirit[®] most completely, but wastes some liquid oxygen.



Step 6

Spirit® Portable

Note: Closing the valve at the first sign of liquid discharge does not fill the Spirit® as completely, but saves liquid oxygen.

9. For the push-on style connectors, push the release button down until the units separate. For all other connectors, rotate the unit until it separates from the stationary unit.

Caution: Check liquid level gauge only after the valve is closed.

Caution: If the Spirit® does not separate easily, do not use force. The units may be frozen together. Wait until the units warm up—then they'll separate easily. Do not touch any frosted parts. Should there be any liquid leakage from the stationary unit or Spirit® after separation, reattach the Spirit® immediately. Again separate the units. If the problem persists, leave units coupled and call your health care provider.

Warning: Should the leakage be excessive to the point that a stream of liquid is present, leave the area and call your health care provider immediately.

Caution: If the Spirit® is being refilled immediately after a period of use, fill the Spirit® and then wait 10 minutes after filling with the flow rate selection knob set at zero for the pressure to stabilize. Then operate as normal.

Warning: Liquid or cold gaseous oxygen can frostbite the skin. Oxygen will result in increased fire hazard.

Portable Release Button (push-on style only)



Operating Instructions

1. Push the cannula breathing tube firmly on to the oxygen tube connector.
2. Adjust the cannula breathing tube to the proper position so that you will be able to breath the oxygen comfortably.
3. Turn the flow rate selection knob on top of the Spirit[®] clockwise until the prescribed flow rate (numeral) is visible in the knob “window” and a positive detent is felt.
4. With each breath, you should now be receiving a set dosage of oxygen gas. The LED will illuminate green with each breath, signaling that the Spirit[®] delivery system is being triggered. When the LED illuminates red with each breath, it is time to replace the batteries.

Note: Make sure that the flow selector knob fully engages at the selected setting. If the knob is between settings, the unit will turn off and then back on and deliver a prolonged burst of O₂ as it powers up.

Note: The Spirit[®] delivery system is designed for a maximum breathing rate of 40 BPM. With breathing rates greater than 40 BPM oxygen dosage may not be delivered with each breath.

Note: When turning the flow control knob from either (0) or (CF) to a flow setting, there will be a prolonged burst of O₂ delivered through the cannula*. Please do not be alarmed as this is normal operation.



*Cannula is not included.

Spirit® Portable

Note: The LED will illuminate green with each detected breath, regardless of the liquid oxygen contents, even if the Spirit® has no liquid oxygen left.

- To stop the oxygen delivery system, turn the flow rate selection knob counterclockwise back to the off position.

Caution: always turn the flow control knob off (0) when not in use.

- The Spirit® delivery system is powered by (2) C-Cell alkaline batteries. When the LED illuminates red, it is time to replace the batteries. As long as the red LED illuminates, the Spirit® will deliver oxygen unless it is empty.
- To replace the batteries, unzip the side zipper, as shown in the picture.



Step 3



Step 7



Step 7

Note: If replacement batteries are not readily available, turn the flow rate selection knob to the furthest clockwise position labeled (CF). The Spirit[®] will now deliver a continuous flow rate of 2 liters per minute (LPM).

Warning: Using the Spirit[®] in the continuous flow (CF) mode will consume the oxygen at a faster rate. Monitor the contents indicator, so you do not run out of oxygen.

Note: The continuous flow (CF) mode is not powered by the batteries, so it can be used whether the low battery LED is on or not.

Note: When using the Spirit[®] in continuous flow, the LED will NOT illuminate green or red with each breath.



Spirit[®] Portable

8. Use the following chart as a guideline to determine the length of time the Spirit[®] will operate:

		Flow Rate Setting							
Model		Off	1.0	1.5	2.0	3.0	4.0	5.0	CF
Spirit 300	Nominal	16.0	14.5	10.0	7.0	4.7	3.5	3.0	2.0
Spirit 600	Nominal	32.0	29.0	20.0	14.0	9.4	7.0	6.0	4.0
Spirit 1200	Nominal	64.0	58.0	40.0	28.0	18.8	14.0	12.0	8.0

Note: Times are in hours.

Note: Times indicated for 20 breaths per minute (BPM) patient. Faster breathing rates will result in shorter use times.

Note: The “Nominal” times are for ideal conditions, (i.e. maximum fill, good loss rate, little movement, etc.). These times are the maximum expected.

Note: Your individual results will vary.

9. To verify the level of liquid oxygen in unit with the contents indicator:

- Support the unit directly above the black contents indicator spring scale chamber.
- Allow the unit to stabilize (i.e. not bounce up and down), then read the exposed colored scale for approximate liquid oxygen contents.

Note: *If the unit is empty, only the red colored band of the scale will be exposed. If this is the case, the unit must be filled prior to use.*



Spirit® Portable



Condensate Pad Maintenance:

1. Under certain environmental conditions, especially with continuous flow and little movement, the Spirit® may develop excess frost around the vaporizer coil and on the case. You may reduce this frost by tapping the unit and/or wiping any accumulated frost off the case.
2. After the Spirit® is empty and has warmed up to room temperature, remove any moisture from the condensation pad.
 - Unzip the zipper at the base of the unit to access the condensation pad.
 - Remove the pad and wring out any absorbed moisture. Allow the pad to dry completely before replacing it back in the unit.
 - Place dry pad back in the unit. Make sure to completely rezip the zipper.

Cleaning Standard

- **Warning:** *Clean only after unit is empty and vented.*
- Clean by applying household glass cleaner to a lint-free cloth and wipe external surfaces—do not get glass cleaner on any internal components or valves.
- Allow unit to dry thoroughly before using.

Note: *Note to health care provider: For reprocessing procedure, see applicable service manual.*

WEEE and RoHS



This symbol is to remind the equipment owners to return it to a recycling facility at the end of its life, per Waste Electrical and Electronic Equipment (WEEE) Directive.

Our products will comply with the Restriction of Hazardous Substances (RoHS) directive. They will not contain more than trace amounts of lead or other hazardous material content.

Accessories

The light weight carrying cart weighs only 3-1/4 lbs. The cart accommodates all Spirit[®] units.



The padded belt pack was designed for patient comfort. The adjustable belt fits waist sizes 28"–45" (71–144 cm). The dual function waist extension allows the belt pack to extend to fit 45"–69" (114–175cm), which allows the belt pack to easily convert into a back pack (see photos).





2200 Airport Industrial Drive
Ste 500
Ball Ground, GA 30107
U.S.A.
www.cairemedical.com



M. D. D. Representative: Medical Product Services
Borngasse 20



35619 Braunfels, Germany
Ref 13138017 Rev F 5/16

