

SERVICE BULLETIN

#20533121

DATE: 10/11/2011

MODELS: All Companion Reservoirs

ISSUE: New Contents Indicator Material

NOTES: Effective immediately, all Companion Reservoir contents indicators (C21:

B-775845-00, C31: B-775846-00, C41: B-775847-00) will be constructed with a new plastic material for the body, piston and retainer. This change will increase the component's resistance to cleaning and leak detector

solutions.







New Material

(Figure 1 - - Contents Indicator Material Change)

Ref 20533121 Rev B Page 1





Old Material

New Material

(Figure 2 - - Contents Indicator blue housing base)

The new plastic material of the body has a light blue tint. This new material will be more resistant to approved cleaners and leak detectors. The approved cleaners and leak detectors are given below:

Function	Acceptable Product
Cleaning	Sporicidin Disinfectant Solution
	Mild dish washing
	detergent/warm water solution
Disinfecting	Sporicidin Disinfectant Solution
	Houshold bleach (1:10 dilution
	with water, freshly made within
	24 hours)
Leak Detection	Snoop Liquid Leak Detector

Sporicidin is a registered trademark of: Sporicidin International

121 Congressional Lane Rockville, MD 20802 Tel: 301-231-7700 www.sporicidin.com

SNOOP is a registered trademark of: Swag

Swagelok Co. 29500 Solon Road Solon, OH 44139 Tel: 440-248-4600 www.swagelok.com

Ref 20533121 Rev B Page 2

To avoid unnecessary damage to the new contents indicator material when cleaning, ensure the following:

- Do not spray directly onto the Contents Indicator. Spray onto a cloth and apply using a cloth.
- After cleaning, the surface must be rinsed with water

The components made with the new plastic are interchangeable with parts made with the older plastic. This change will not alter the functionality of the contents indicator in any way. The unit will still operate on the concepts of differential pressure and all internal components will be the same. The level sensing integrity will not be decreased by this material change.

CONTACT: For any technical questions or concerns, contact Technical Service:

USA: 800-482-2473

techservice.usa@chart-ind.com

Europe: +44 1344 403100

technicalservice.europe@chart-ind.com

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

customerservice.usa@chart-ind.com

Europe: +44 1344 403100

<u>customerservice.europe@chart-ind.com</u>

Ref 20533121 Rev B Page 3