


# COMPANION 5™

**Figure 1: Outer Components**

Item Number Companion 5	Item Number Companion 5 MDD	Description	Qty	Seq No
20916443	20916443	Front Case	1	1
15073173	15073173	Foam Acoustic Front*	1	NA
15073165	15073165	Foam Gasket LCD Screen*	1	NA
15063311	15063311	Back Case*	1	2
15073181	15073181	Foam Acoustic Back*	1	NA
20970192	20970192	Companion ECO Label	1	3
20970191	20970191	CAIRE Label	1	4
15062861	15062861	Caster	4	5
CD038-12	CD038-12	Power Cord, 115V, USA	1	6
CD038-13	CD038-13	Power Cord, 230V, Euro	1	6
CD038-20	CD038-20	Power Cord, Brazil	1	6
CD006-1	CD006-1	Strain Relief	1	7
15063266	21494452	Flowmeter	1	8
15063282	15063282	On/Off Switch	1	9
NA	21485458	Switch Cover	1	10
15067540	15067540	Display Label O2 Monitoring	1	11
15067700	15067700	Humidifier Bottle Strap	1	12
15063696	15063696	Warning LED Light Pipe	3	NA

\* When replacing Front or Back cases, Foam Kits must be ordered separately.

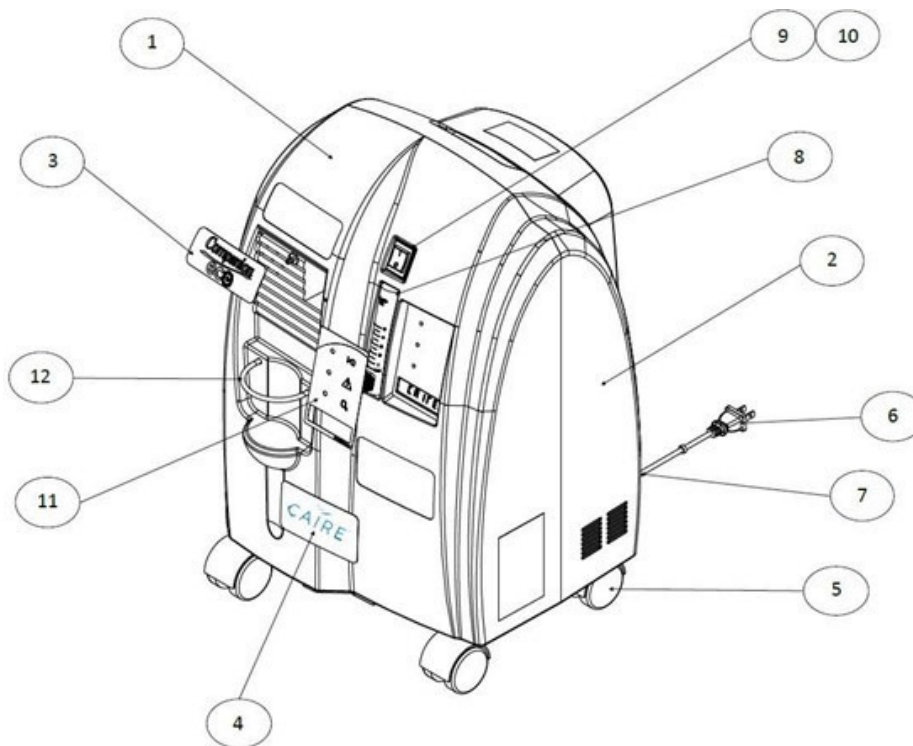

**Figure 1: Outer Components**

**Figure 2: Compressor and Inside Front Case**

Item Number Companion 5	Item Number Companion 5 MDD	Description	Qty	Seq No
CB209-1S	CB209-4S	Circuit Board 120VAC/60Hz	1	1
CB209-2S	CB209-5S	Circuit Board 240VAC/60Hz	1	1
CB209-3S	CB209-6S	Circuit Board 240VAC/50Hz	1	1
15063258	15063258	Tubing, Black, Comp Intake	1	2
15062925	15062925	Filter, Compressor Intake	1	3
15062896	15062896	Cover, Compressor Intake	1	4
15062984	15062984	Screw, Compressor Intake Cover	1	5
15063426	15063426	Product Filter (HEPA)	1	6
20815629	20815629	Braided Tubing from Compressor to Heat Exchanger	1	7
15062917S	15062917S	Compressor Assembly (120V) Includes Capacitor	1	8
15067697S	15067697S	Compressor Assembly (230V) Includes Capacitor	1	8
FN041-1	FN041-1	Cooling Fan 115V	1	9
FN001-2	FN001-3	Cooling Fan 230V	1	9
20914768	20914768	Heat Exchanger	1	10
21496797*	21496797*	Oetiker Hose Clamp	2	11
CL036-1*	CL036-1*	Ratcheting Hose Clamp	1	11
15062888	15062888	Shelf Fan	1	12
15062853	15062853	Screw, Fan Shelf	4	13
15062984	15062984	Screw, Circuit Board	4	14
21127245	21127245	Bottom Gross Particle Filter	1	15
20815627	20815627	Tubing, Braided, from Heat Exchanger to Sieve Beds	1	16
20809627	20809627	Wire Harness, Cooling Fan (115V-230V)	1	17
SW002-3	SW002-3	Temperature Switch	1	18
NA	21354096	Adapter, Alarm Buzzer	1	19
NA	15062984	Screw, Adapter Buzzer	2	20
NA	AL022-1	Alarm Buzzer	1	21
NA	SC177-1	Screw, Alarm Buzzer	2	22

\*Please refer to Service Bulletin PN 21360547 for Hose Clamp change.

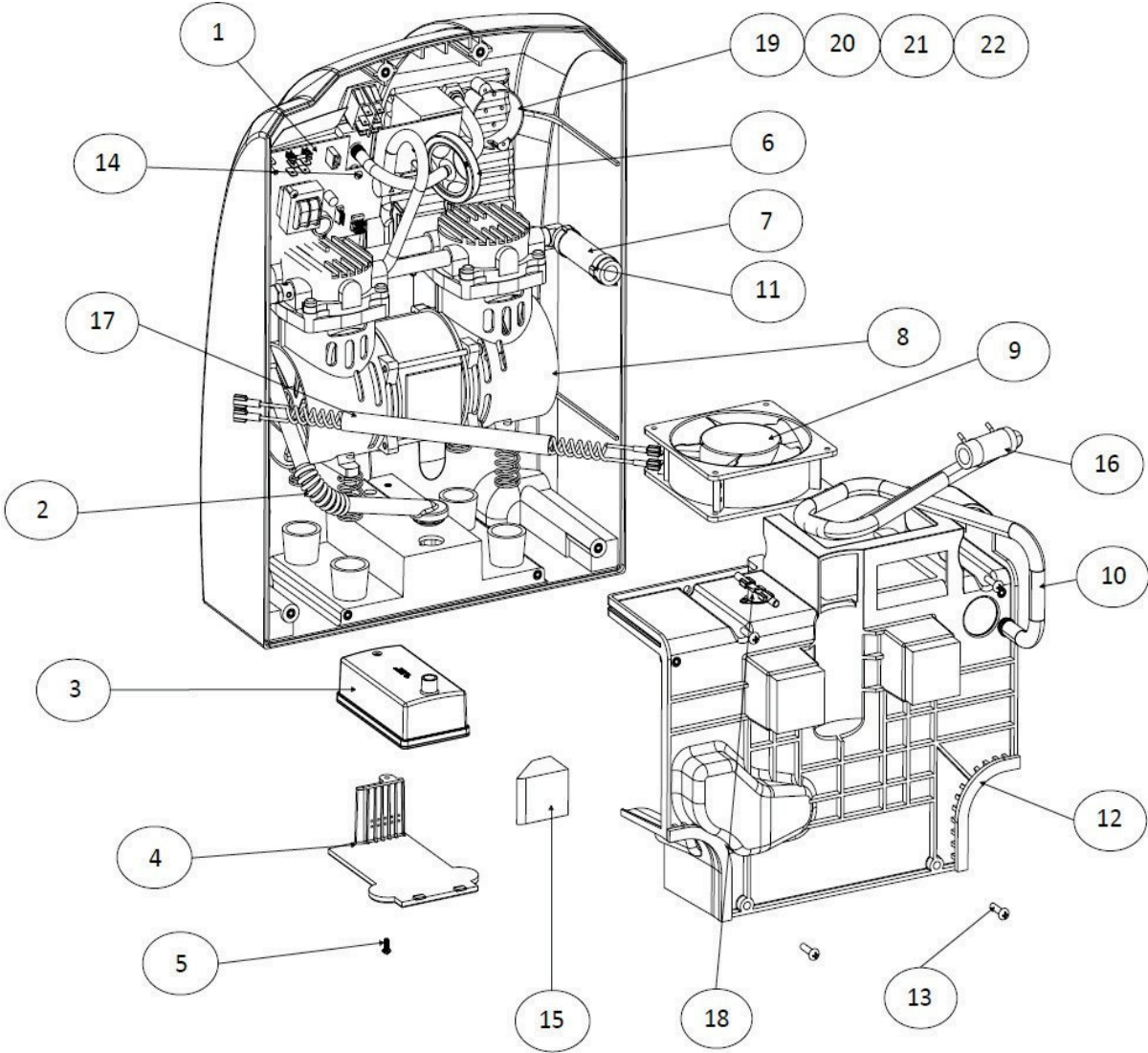



Figure 2: Compressor and Inside Front Case

# COMPANION 5™

**Figure 3: Sieve Bed Assembly and Back Case**

Item Number Companion 5	Item Number Companion 5 MDD	Description	Qty	Seq No
15062853	15062853	Screw, Case	6	1
21127244*	NA	Handle Gross Particle Filter	1	2
21127239*	NA	Filter Gross Particle Screen	1	3
15063531	15063531	Regulator	1	4
20913392S	20913392S	Valve Assembly	1	5
15063565	15063565	Muffler, Exhaust	1	6
BE561-1S	BE561-1S	Sieve Bed Assembly	1	7
15089714	15089714	Tubing, Silicone, Check Valve to O2 port	1	8
15063653	15063653	Check Valve Diaphragm	1	9
15089722	15089722	Tubing, Silicone, Sieve Beds to Check Valve	1	10
15063338	15063338	Grommet, Rubber, Top Mount	1	11
20902395	20902395	Grommet, Rubber, Bottom Mount	2	12

\*Please refer to Service Bulletin PN 21494442 for Companion 5 Top Intake Filter Removal.

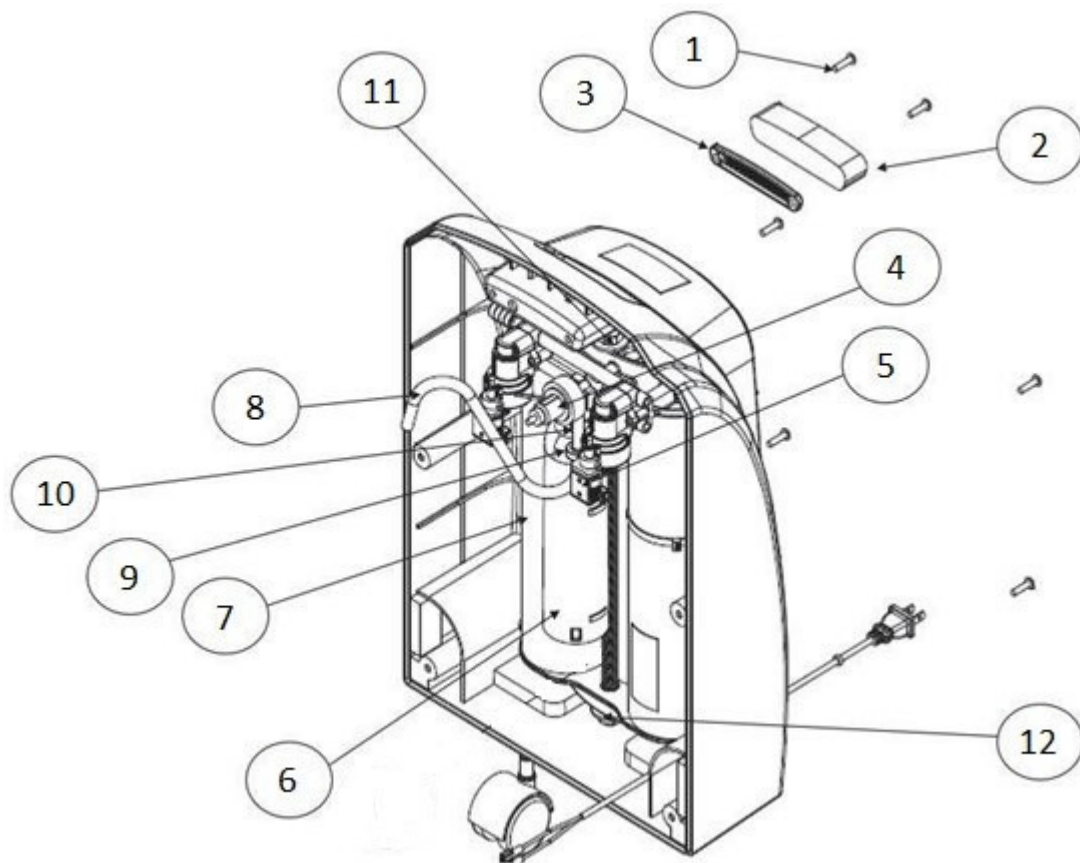

**Figure 3: Sieve Bed Assembly and Back Case**

Copyright © 2024 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.