



SERVICE BULLETIN

PN: 21220076

RELEASE DATE: October 31, 2017

REVISION DATE(S): March 12, 2018 (Rev B), October 18, 2018 (Rev C), March 05, 2019 (Rev D)

MODEL: NewLife Elite, Intensity 10 and VisionAire 5

ISSUE: IEC 80601-2-69 Oxygen Concentrator Particular Standard Compliance

NOTES: To ensure NewLife Elite, Intensity 10 and VisionAire oxygen concentrators are compliant with IEC 60601-1 3.1, select models will have new alarm conditions, new alarm lights, and labels. As part of the product update, both VisionAire and NewLife units will have identical alarm conditions going forward on newly manufactured units. These devices will now be IEC 60529 IP21 rather than IPX1. This change will be accompanied by revised alarm label, new circuit board part number, and an updated alarm code troubleshooting tables. Only the model numbers listed in the table below will continue to bear the CE mark on newly produced units and these will be the only models to be compliant to the latest 60601 standard.

See bulletin 21288973 for details on discontinued NewLife models and their replacement model numbers.

Previously manufactured units built without these new alarms and labels are not required to be updated to the new standard.

When servicing 3-LED model VisionAire units, if a main circuit board needs replacing, be sure to replace it with CAIRE item CB199. The previous design CB078 boards are not forwards compatible with 3-LED VisionAire units. Likewise CB199 is not backwards compatible with previous design VisionAire 5 units (built with 1 or no LED lights). See

bulletin 21190303 for details on VisionAire main board replacements for models built with 1 or no alarm LED lights.

When servicing 3-LED NewLife units, if a main board needs replacing, be sure to replace it with CB200 service parts. The previous design circuit boards (CB154 and CB160) are not forward compatible with 3 LED NewLife models. See bulletin 21329212 for information on retrofitting newer style main circuit boards (CB200) onto NewLife models originally built with 1 or no alarm LEDs. Specific additional parts and procedures are required to accomplish such a retrofit.

For devices within manufacturer's warranty, be sure to contact CAIRE for authorized repair service.

Model	Voltage	Plug Type	Beginning S/N
NewLife Intensity 10			
AS099-110	120V, 60Hz	IEC Type A, Domestic	CBB0118340001
AS099-103	230V, 50Hz	IEC Type F, mainland Europe	CBB0118080334
AS099-210	230V, 50Hz	IEC Type G, UK, Saudi Arabia,	N/A
NewLife Elite			
AS005-4	120V, 60Hz	IEC Type A, Domestic	CBB0118420228
AS005-244	230V, 50Hz	IEC Type I, Argentina	CBB0118430021
AS005-270	230V, 50Hz	IEC Type G, UK, Saudi Arabia,	CBB0118480080
AS005-550	230V, 50Hz	IEC Type F, mainland Europe	N/A
AS005-555	230V, 50Hz	IEC Type F, mainland Europe	CBB0118440001
VisionAire			
AS098-5	230V, 50Hz	IEC Type F, mainland Europe	GPB0118050001
AS098-7	230V, 50Hz	IEC Type G, UK, Saudi Arabia,	GPB0119030001
AS098-15	230V, 50Hz	IEC Type F, mainland Europe	N/A

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 (0) 118 936 7060

www.cairemedical.com

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 (0) 118 936 7060

www.cairemedical.com

Copyright © 2019 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.