

---

## PROVIDER HANDOUT

---

STEP 1  
CREATE  
CODES

- Log in to the CAIREview Portal at **portal.caireview.com**
- Go to Monitors tab
- Select New Monitor
- Complete the New Monitor form to generate a Patient code
- Repeat for each Concentrator being deployed.

STEP 2  
DOWNLOAD  
APP

- Go to the Apple App store or Google Play store on the smart device to be linked with CAIREview app
- Search for the CAIREview app
- Download and install the CAIREview app on patient's smart device

STEP 3  
LINK  
CODE

- Open the app and power on the concentrator
- Use the Patient code generated from Step 1 to log into the CAIREview app
- Enable Save, press Next, hand phone to patient and have him/her accept Terms & Conditions
- Link device by clicking on Cloud icon
- Choose device and click the Link button
- Select the Send Data tile and click Concentrator to enable syncing

STEP 4  
VIEW  
DATA

- Log into the CAIREview Portal at **portal.caireview.com**
- Go to Monitors tab
- Locate the Patient code in the Monitor List
- Click the Graph button on the right to view data

CAIREview™ Support | 1-800-482-2473

[techservice.usa@chartindustries.com](mailto:techservice.usa@chartindustries.com)  
8:30 AM - 5:00 PM EST Monday - Friday

[customerservice.usa@chartindustries.com](mailto:customerservice.usa@chartindustries.com)  
7:00 AM - 7:00 PM EST Monday - Thursday  
7:00 AM - 5:00 PM EST Fridays