

# December 2016 Med Tips

## Accessories and Parts

### New Shoulder Strap

COMING IN JANUARY\* all CAIRE Stroller, Sprint, Spirit Portables Stroller will be equipped with new shoulder strap. The new shoulder strap will use the same part number as 13234703. The reason for this change is to improve strap strength and durability to make it more robust.

The new shoulder strap uses robust plastic swivel hooks to attach to the case in place of the previous snap buttons. This eliminates any fabric from coming in contact with the metal clasp, improving the strength of connection points to the LOX portable. The shoulder pad has also been redesigned to a higher strength with the same polyester material and the connection of the pad to the strap is now larger and has improved stitching. This change will help improve the strength of the strap and help to reduce the opportunity to tear at the pad with repeated use over time.

\*More information to come in next month's Med Tips\*



FIGURE #1: Current pad connection vs. the new pad connection



FIGURE #2: Old shoulder strap and New shoulder strap (right)

## Helpful Hints

### Auto-Stop Feature

Auto-stop feature reduces power when the car is idling, causing an Eclipse to revert to battery power. Customers should test the unit in their vehicle before relying on the vehicle's outlet power. In most cars that have an auto-stop feature, the auto-stop feature can be disabled.

### E3 DC Power Operation

The DC power supply does not produce sufficient power to achieve all the features that are available when running on AC power or the power cartridge. When running on DC power, the Eclipse 3 **WILL NOT**:

- Charge the power cartridge (battery) inside of the Eclipse 3
- Operate above 2.0 LPM continuous flow
- Operate above 96 mL pulse dose



## Product Information

A new CAIRE® Accessories Catalog is now available. The catalog features a new format with updated part numbers and images, and is organized by product for ease in finding the accessories you need. [Download the Full CAIRE Accessories Catalog Here](#)

### AirSep® FreeStyle® 3 Accessories with user-replaceable battery pack



#### FreeStyle Battery Pack

Item Number BT033-1



#### Carrying Bag

(Includes Shoulder Strap)

Item Number MI419-1



#### Backpack Harness

Item Number MI284-1



#### Carry-All Accessory Bag

Item Number MI320-1

Replacement  
Strap  
MI017-1



#### Intake Filter

Item Number FI144-1



#### AC/DC Power Supply

(Includes AC and DC Cords)

Item Number

PW032-1 (US)

PW032-2 (EU)

PW032-3 (AU)

PW032-4 (UK)



#### Desktop Charger

(Battery Not Included)

Item Number

BT032-1 (US)

BT032-2 (EU)

BT032-3 (AU)

BT032-4 (UK)



#### AC Power Cord

Item Number CD023-2 (US)  
CD017-2 (EU)  
CD025-1 (AU)  
CD017-4 (UK)



#### AC Power Supply Brick Only

Item Number PW031-1



#### DC Power Cord

Item Number CD034-1



  
**CAIRE®**  
A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)

  
CHARI

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0080 Rev B

## AirSep® FreeStyle® 5 Accessories with user-replaceable battery pack



### FreeStyle Battery Pack

Item Number BT033-1



### Carrying Bag

(Includes Shoulder Strap)

Item Number

MI420-1



### Backpack Harness

Item Number MI284-1



### Carry-All Accessory Bag

Item Number

MI372-1

Replacemnt


Strap

MI017-1



### Intake Filter

Item Number FI194-1

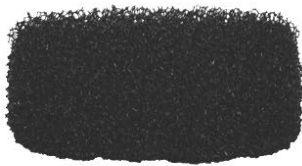

### AC Power Supply

(w/Cord Included)

Item Number

PW035-1 (US)

PW035-2 (EU)

PW035-3 (AU)

PW035-4 (UK)



### Desktop Charger

(Battery Not Included)

Item Number

BT032-1 (US)

BT032-2 (EU)

BT032-3 (AU)

BT032-4 (UK)



### AC Power Cord

Item Number CD023-2 (US)

CD017-2 (EU)

CD025-1 (AU)

CD017-4 (UK)



### AC Power Supply Brick Only

Item Number PW033-1



### DC Power Supply

Item Number PW034-1



  
**CAIRE**®  
 A Chart Industries Company

www.CAIREMedical.com



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
 USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
 PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0081 Rev D



## AirSep® FreeStyle® 3 Accessories

**Carrying Bag**  
(shoulder strap included)  
Item Number MI406-1



**Carry-All Accessory Bag**  
Item Number MI320-1



Replacement Strap MI017-1

### AC/DC Power Supply

(Includes AC and DC Cords)

Item Number PW022-1 (US)  
PW022-2 (EU)  
PW022-3 (AU)  
PW022-4 (UK)  
PW025-1 (Brick Only)



### AC Power Cord

Item Number CD023-2 (US)  
CD017-2 (EU)  
CD025-1 (AU)  
CD017-4 (UK)



### DC Power Cord

Item Number CD034-1



### Backpack Harness

Item Number MI284-1



### External Power Cartridge Kit

(Includes Battery & Charger)

Item Number BT026-1 (US)  
BT026-2 (EU)  
BT026-3 (AU)  
BT026-4 (UK)



### Power Cartridge Only

Item Number BT024-1

### Power Cartridge Desktop Charger Only

(Power Supply with Cord Included)

Item Number PW029-1 (US)  
PW029-2 (EU)  
PW029-3 (AU)  
PW029-4 (UK)



### Internal Battery

Item Number BT018-1S



### AirBelt Kit (Includes AirBelt & Charger)

Item Number BT017-1 (US)  
BT017-2 (EU)  
BT017-3 (AU)  
BT017-4 (UK)



### AirBelt Battery Only

Item Number BT016-1S



### AirBelt Charger Only

Item Number PW008-1S (US)  
PW008-2S (EU)  
PW008-3S (AU)  
PW008-4S (UK)



**CAIRE**<sup>®</sup>

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONG0102 Rev B





## AirSep® FreeStyle® 5 Accessories

**Carrying Bag**  
(shoulder strap included)  
Item Number MI407-1



**Carry-All Accessory Bag**  
Item Number MI372-1



Replacement Strap MI017-1

### AC Power Supply

Item Number PW021-1 (US)  
PW021-2 (EU)  
PW021-3 (AU)  
PW021-4 (UK)  
PW020-1 (Brick Only)



### AC Power Cord

Item Number CD023-2 (US)  
CD017-2 (EU)  
CD025-1 (AU)  
CD017-4 (UK)



### DC Power Supply

Item Number PW027-1



### Backpack Harness

Item Number MI284-1



### External Power Cartridge Kit

(Includes Battery & Charger)

Item Number BT026-1 (US)  
BT026-2 (EU)  
BT026-3 (AU)  
BT026-4 (UK)



### Power Cartridge Only

Item Number BT024-1

**Power Cartridge  
Desktop Charger Only**  
(Power Supply with Cord Included)

Item Number PW029-1 (US)  
PW029-2 (EU)  
PW029-3 (AU)  
PW029-4 (UK)



### Internal Battery

Item Number BT021-1S



### AirBelt Kit (Includes AirBelt & Charger)

Item Number BT017-1 (US)  
BT017-2 (EU)  
BT017-3 (AU)  
BT017-4 (UK)



### AirBelt Battery Only

Item Number BT016-1S



### AirBelt Charger Only

Item Number PW008-1S (US)  
PW008-2S (EU)  
PW008-3S (AU)  
PW008-4S (UK)



  
**CAIRE**<sup>®</sup>

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)

  
CHART<sup>®</sup>

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-COHC0103 Rev B



## SeQual® Eclipse 5® Accessories

\*Accessories compatible with all generations of SeQual® Eclipse™ unless otherwise noted below.

### Power Cartridge (Battery)

Item Number 7082-SEQ



### Cart

Item Number

5991-SEQ



### AC Power Supply

(Cord Included)

Item Number 5941-SEQ (US)

20553100 (EU)

20553098 (UK)



\*compatible  
with Eclipse  
2, 3, and 5

### DC Power Supply

Item Number 5942-SEQ



\*compatible with Eclipse 2, 3, and 5

### Accessory Bag

Item Number

7104-SEQ



### Desktop Charger (Cord Included)

Item Number 7112-SEQ (US)

20554079 (EU)

20554077 (UK)



### Travel Case

Item Number 5010-SEQ



### Protective Cover

Item Number

5052-SEQ



### Wheelchair Pack

Item Number

5220-SEQ



### Travel Accessory Kit

(Includes Travel Case, Power Cartridge,  
Desktop Charger w/US Cord)

Item Number

5093-SEQ



### Humidifier Adaptor Kit

(Bottle Not Included)

Item Number 7116-SEQ



  
**CAIRE®**

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-GONC0104 Rev A

## SeQual® eQuinox™ Accessories



### 12-Cell Battery

Item Number 20952900



### 24-Cell Battery

Item Number

20952906



### Cart

Item Number 4880-SEQ



### DC Power Supply

Item Number 4830-SEQ



### Accessory Bag

Item Number

4920-SEQ



### Desktop Charger (Cord Included)

Item Number 4980-SEQ (US)  
4980-C1-SEQ (EU)  
4980-8-SEQ (UK)



### AC Power Supply (Cord Included)

Item Number 4836-US-SEQ  
4836-C1-SEQ (EU)  
4836-8-SEQ (UK)



### AC Power Supply Brick Only

Item Number 4836-SEQ



### AC Power Cord

Item Number 4997-SEQ (US)  
4998-SEQ (EU)  
4999-SEQ (UK)



### Travel Power Adaptor

Item Number 5591-SEQ



### Humidifier Adaptor Kit

(Bottle Not Included)

Item Number 7116-SEQ



  
**CAIRE®**

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)

  
**CHART®**

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CDNC0105 Rev A





## AirSep® Focus™ Accessories

### Carrying Bag Kit

Item Number MI332-1

**Includes:** Bag w/ 2 Battery Pockets  
 Carrying Sleeve w/ Belt Clip  
 Battery Case w/ Belt Clip  
 Shoulder Strap



### AirSep® Focus™ Battery

Item Number BT023-1



### Battery to Power Supply Charging Cord

Item Number CD032-2



### Battery to AirSep® Focus™ Operation Cord (with I/O switch)

Item Number CD032-1



### Universal AC/DC Power Supply (Includes AC and DC Power Supply)

Item Number PW023-1 (US)  
 PW023-2 (EU)  
 PW023-3 (AU)  
 PW023-4 (UK)  
 PW019-1 (Brick Only)



### Carry-All Accessory Bag

Item Number MI320-1



Replacement Strap MI017-1

### AC Power Cord

Item Number CD023-2 (US)  
 CD017-2 (EU)  
 CD025-1 (AU)  
 CD017-4 (UK)



### DC Power Cable

Item Number CD034-1



### Carrying Sleeve with Belt Clip

Item Number MI345-1



### Battery Case with Belt Clip

Item Number MI333-1



  
**CAIRE®**

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)

  
**CHART®**

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
 USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
 PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0097 Rev B

# Concentrator Service and Maintenance Software and Service Equipment

## Maintenance Software

### AirSep® FreeStyle® and Focus™ Hour Meter Software

**Compatible With:**  
AirSep® Focus™, FreeStyle® 3, FreeStyle® 5


Item Number KI406-1



### EDAT Diagnostic Software

**Compatible With:**  
SeQual® Eclipse®, SeQual® eQuinox™

Item Number 5535-SEQ

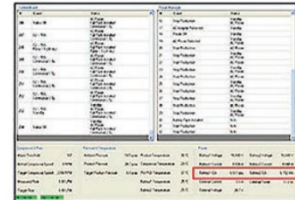

## Service Equipment

### AirSep® Test Pressure Gauge

Item Number KI036-1



Combine with tubing/ adaptors to the right to make complete test kit.

### Operating Pressure Tubing Adaptors

VisionAire KI257-2  
FreeStyle 3/5 KI257-1  
Intensity/Elite KI257-1



Combine with test pressure gauge to make complete test kit.

### Intensity Outlet Pressure Tubing Adaptors

Intensity 8 TU131-1  
Intensity 10 TU131-2



Combine with test pressure gauge to make complete test kit.

### CAIRE® Companion 5™ Tool Kit

(Includes #2 Phillips Screwdriver, Magnetizer, Slip Joint Pliers)

Item Number 20852059



**CAIRE®**

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONG0106 Rev A

# Portable and Stationary Concentrator Filters

## CAIRE® Companion 5™

Intake Filter

Item Number 15062925



## AirSep® VisionAire™

Handle Filter (optional)

Item Number FI202-1



## AirSep® NewLife® Family

Gross Particle Filter

Item Number FI002-1



Felt Filter

Item Numbers

Elite 110V MI161-1  
Elite 220V MI161-2  
Intensity 8 MI161-3  
Intensity 10 MI161-2



## SeQual® Eclipse®

Preventative Maintenance Kit Item Number 5022-SEQ

Includes:

2 Air Intake Filters (7028-SEQ)  
Compressor Intake Filter (8069-SEQ)  
HEPA Filter, External (6986-SEQ)\*  
HEPA Filter, Internal (4770-SEQ)\*  
9V Battery (8098-SEQ)  
O-Rings for Internal HEPA Filter  
(9251-SEQ & 4382-SEQ)  
10 Zip Ties

\*Contact technical service if you are unsure of which HEPA filter you need.



## SeQual® eQuinox®

Air Inlet Filter

Item Number 4845-SEQ



Oxygen Outlet Filter

Item Number 4770-SEQ



Compressor Inlet Filter

Item Number 9759-SEQ



## AirSep® FreeStyle™ 3

Intake Filter

Item Number FI144-1



\*Compatible with Internal and user-replaceable battery versions

## AirSep® FreeStyle™ 5

Intake Filter

Item Number FI194-1



\*Compatible with Internal and user-replaceable battery versions

  
**CAIRE®**

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0101 Rev B

## Oxygen Delivery Accessories



**CAIRE® Comanion 5™  
Humidifier Adaptor**  
(9 in/23 cm)

**Item Number** 20843882



**AireSep® VisionAire™  
Humidifier Adaptor**  
(21 in/53 cm)

**Item Number**  
TU255-1



**Nasal Cannula  
Adult**

7 ft (2.1 m) CU002-1  
25 ft (7.6 m) CU002-4  
50 ft (15.2 m) CU002-6

**Pediatric**

7 ft (2.1 m) CU002-7

**Fingertip Pulse  
Oximeter**

**Item Number** 7568-SEQ



**Humidifier Bottle**

**Item Number** HU003-1



**Oxygen Outlet Adaptor**

**Item Numbers**

**Black** F0025-1

**Green** B-776945-00



**Oxygen Tubing**

7 ft (2.1 m) CU004-2  
25 ft (7.6 m) CU004-3  
50 ft (15.2 m) CU004-1

## Dual Lumen Cannulas (For Use With HELIOS H300 & H850 Liquid Oxygen Units Only)

**Sense & Delivery  
in Separate Nostrils**

**Item Numbers**

4 ft (1.2 m) B-701931-00  
7 ft (2.1 m) B-701930-00



**Sense & Delivery  
in Both Nostrils  
(Divided Tube)**

**Item Numbers**

5 ft (1.5 m) 6-778058-00  
7 ft (2.1 m) 6-778057-00



**Sense & Delivery  
in Both Nostrils  
(Two Tubes)**

**Item Numbers**

5 ft (1.5 m) 10035468  
7 ft (2.1 m) 10035467



**Sense & Delivery  
in Both Nostrils  
(Single Tube)**

**Item Numbers**

4 ft (1.2 m) 20574985  
5 ft (1.5 m) 20574977  
7 ft (2.1 m) 20574978



  
**CAIRE®**  
A Chart Industries Company  
[www.CAIREMedical.com](http://www.CAIREMedical.com)

  
**CHART®**

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR  
USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS,  
PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-COINC0096 Rev A





## CAIRE® Liquid Oxygen Service/Testing Accessories

### Solder 60/40 Lead Free

Item Number 13095635



### SF Lip Seal Tool

Item Number 97405590



### Soldering Flux

Item Number AAA0645



### Krytox Thread Lubricant (2oz)

Item Number 20605136



### Christo-Lube Fluoro-Lubricant (2oz)

Item Number CA200071



### Teflon Tape

Item Number CA406310



### Liberator 15LPM Upgrade Kit

Item Number 13440224



### Condensate Bottle Kits

Item Numbers

Liberator 13660403

Companion B-701362-SV

HELIOS B-701546-00



### Liberator G4 Meter Upgrade Kit

Item Numbers

G2 to G4 13199653

G3 to G4 13219221



### Stroller/Sprint G3 Meter Upgrade Kit

Item Numbers

G2 (Tan) to G4

13199653

G3 to G4

13199661

\*Order with Dome Cover #12932526 (Blue)



### Low Loss/EMS G4 Meter Upgrade Kit

Item Number 13579584



**CAIRE**<sup>®</sup>

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)

CHART

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.


© 2016 CAIRE Inc. All Rights Reserved ML-C0NC0099 Rev B

# CAIRE® Liquid Oxygen Transport Accessories



## Rolling Cart

Item Number 20765565

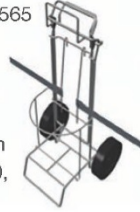

Note: Compatible with Stroller, Hi Flow, H850, C1000/C1000T

## HELiOS™ H850 Backpack

Item Number 069209



## HELiOS™ H300 Hip Pack

Item Number B-701654-00

Belt Extension (48-81 in / 121-206 cm) 14951835



## Companion® C1000/T Backpack

Item Number 10018138



## Companion® C500/C550 Hip Pack

Item Number B-778625-00



## Stroller®, Sprint™, Spirit™ 600/1200, Hi Flow Backpack

Item Number 13214825



## Spirit™ 300 Belt Pack



Item Number 11843361

Waist Extension 11843396

Side Pouch (small) 11843370

Side Pouch (large) 11843388

## RollerBase (CE Marked) w/2 Brakes



Item Numbers

Companion® (14 in/35.6 cm) 10855678

HELiOS™ (15 in/38.1 cm) 14880387

Lib 20-45 (14 in/35.6 cm) 10855678

Lib 60 (16 in/40.6 cm) 11363069

## RollerBase (No CE Mark) w/o Brakes



Item Numbers

Companion® (14 in/35.6 cm) 10585866

HELiOS™ (15 in/38.1 cm) 14880803

Lib 20-45 (14 in/35.6 cm) 10585866

Lib 60 (16 in/40.6 cm) 11194074

## Replacement Casters (Pack of 5)

Item Numbers

2 Brakes, 3 No Brakes 14880379

5 No Brakes 14880361



**CAIRE**<sup>®</sup>

A Chart Industries Company

[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0098 Rev A

## CAIRE® Liquid Oxygen System Transfilling Accessories



### Liquid Oxygen Transfer Line

#### Item Numbers

- 4 ft (1.2 m) 9713139
- 6 ft (1.8 m) 9713119
- 8 ft (2.4 m) 10546550
- 10 ft (3.0 m) 10565161
- 12 ft (3.7 m) 10562411



### CAIRE® Top Fill Transfer Line Adaptor

Item Number 10678157



### CAIRE® Side Fill Transfer Line Adaptor

Item Number 97212023



### TF/SF Dual Fill Transfer Line Adaptor

Item Number 10897958



\*Alternate QDVs Available Upon Request

### Large Reservoir Vent Wrench

Item Number 97202005



Liberator (All), Companion/HELIOS (Manufactured After July 2012)

### Small Reservoir Vent Wrench

Item Number B-775182-00



Companion/HELIOS (All), Liberator (Manufactured After July 2012)

### Pressure Testing Assembly

Item Number B-701732-00



DISS Adaptor attaches to FCV outlet of Reservoir

### SuperFlex Hose (6ft, 1.8m) w/ No Swivel Nuts

Item Number 97406555



### Dewar Cap

Item Number 3910486



  
**CAIRE®**  
 A Chart Industries Company  
[www.CAIREMedical.com](http://www.CAIREMedical.com)



PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.

© 2016 CAIRE Inc. All Rights Reserved ML-CONC0100 Rev A

PLEASE CONSULT THE APPLICABLE PRODUCT INSTRUCTIONS FOR USE FOR PRODUCT INDICATIONS, CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, AND DETAILED SAFETY INFORMATION.