

Hi, just a reminder that you're receiving this email because you have expressed an interest in CAIRE Inc. Don't forget to add julie.liberti@chartindustries.com to your address book so we'll be sure to land in your inbox.

You may [unsubscribe](#) if you no longer wish to receive our emails.



CAIRE®

A Chart Industries Company

AirSep®  SeQual®

www.CAIREmedical.com

Med Tips

In This Issue

[Helpful Hints / FAQs](#)
[Service Schools & Tradeshow](#)
[Contact Us](#)

PDF Version

To view Med Tips in PDF,
[click here.](#)

Join Our List

[Join Our Mailing List!](#)

May 2014

PRODUCT INFORMATION

HEPA Filter Part Number Change for Eclipse Units

Please note that the new style HEPA Filter part number for the Eclipse units is now **4770-SEQ** (previous PN: 9765-SEQ). The design and functionality has not changed and this filter is also compatible with the eQuinox. This filter is also included in the Eclipse PM Kit, PN: 5022-SEQ.



Eclipse 3 and Eclipse 5 Provider Mode Menu Options

Providers can access the Provider Mode by pressing the non-smoking button on the control panel to display information about the Eclipse. To cycle through the menu, simply press the non-smoking button repeatedly.



The below table can be used to determine what function you will arrive at by the number of repeated presses.

<u>Number of Repeated Presses</u>	<u>Menu Option</u>	<u>Description</u>
1	ALRM=###	Alarm Code
2	PS=#	Pulse Mode Sensitivity Adjustment
3	HRS = ##	Hours of Operation
4	9V = V	9-Volt Battery Status

5	BOL =	Rise Time Setting for Bolus Delivery
6	CB#####	Control Board PN (first 4 numbers) and Rev. (last 2 numbers and assume decimal in between the two numbers)
7	PM#####	Power Manager PN (first 4 numbers) and Rev. (last 2 numbers and assume decimal in between the two numbers)
8	-	Returns to the Main Menu

AirSep LifeStyle - Product Discontinued

As of January 2014, CAIRE has discontinued production of new AirSep LifeStyle portable oxygen concentrators. The AirSep FreeStyle 5 has all of the same flow rate capabilities as the LifeStyle with pulse settings 1-5. It can be used as a direct replacement for customers that require the function of the LifeStyle. Please contact us for more information about the FreeStyle 5.




HELPFUL HINTS / FAQs

In-Line Restrictor for NewLife Intensity and Intensity 10 O.M. Units

Q: My NewLife Intensity 10 is reading good purity, however, my Low O2 Indicator Light is on. What is the issue?

A: One thing to check for would be to ensure that flow is being delivered to the O2 Board. If there is no flow being delivered, check for crimped tubing in the lines. If there is

flow being delivered to the inlet side of the restrictor and nothing is coming out of the outlet side, then the restrictor is clogged and can be replaced using PN: **OF060-1**.

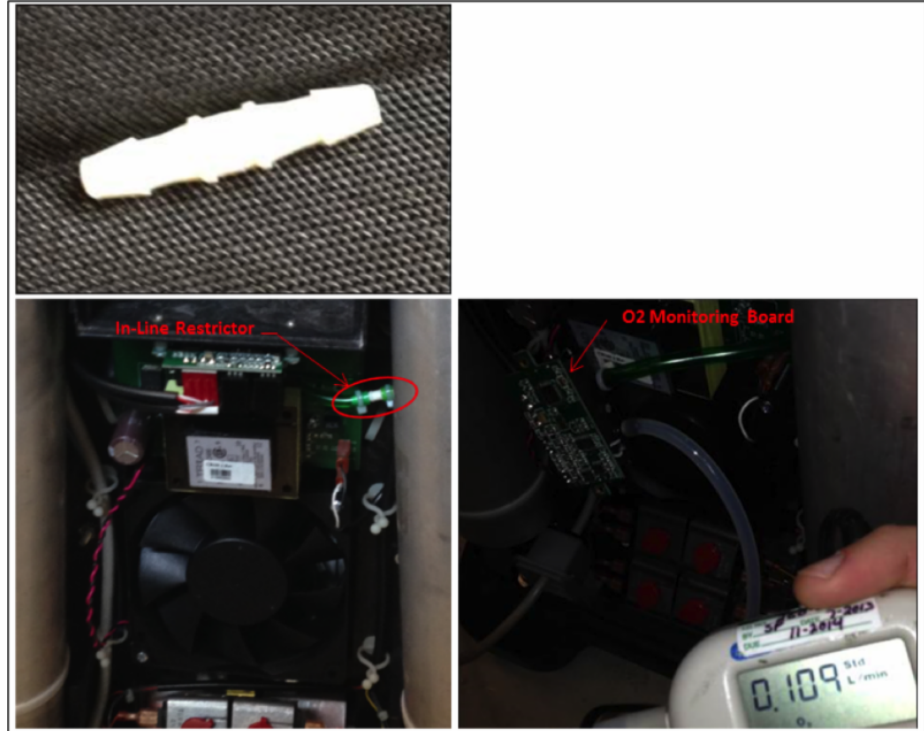


CPC Connector Kit for HELiOS Portables

Q: I have a newer HELiOS portable and noticed the Liquid Leak Detector Test, Primary and Secondary Relief Valve Test, and Pressure Hold Test require that you connect the HELiOS Oxygen Supply Line into the quick connect of the HELiOS portable. I see a plug where the CPC Connector usually is. How do I perform the tests?

A: Newer HELiOS Portables are now manufactured without the CPC Connector (See January 2014 Med Tips for further details). In order to perform these tests, you will need the CPC Quick Connector Kit which can be ordered using PN: **20748595**.



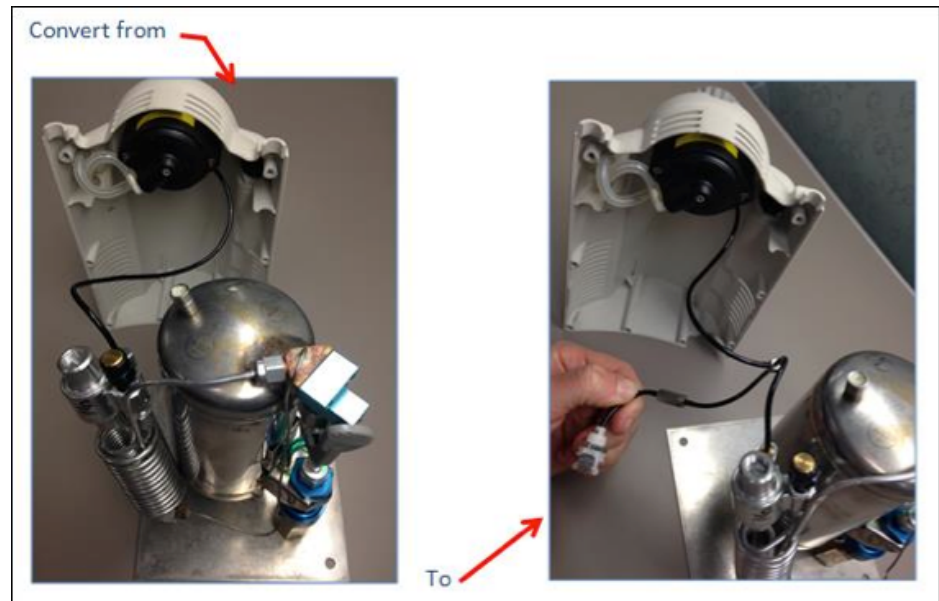

Adding the CPC Connector Kit to a Newer-Style Unit

ACCESSORIES

AirSep External Power Cartridge and Charger

CAIRE now offers a new external power cartridge compatible with the AirSep Focus™, FreeStyle™, and FreeStyle 5™. Benefits with this product are listed below:

- * NEW LIGHTWEIGHT alternative to the AirSep AirBelt
- * External charger can charge 2 batteries simultaneously
- * Built-in belt clip for easy wearability
- * Fully charges battery in 5 hours or less
- * Easy-access battery gauge displays remaining charge



ORDERING INFORMATION

BT024-1 AirSep External Power Cartridge

BT026-1 External Power Cartridge & Charger Combination (US)

BT026-2 External Power Cartridge & Charger Combination
(EUROPEAN)

BT026-3 External Power Cartridge & Charger Combination
(AUSTRALIAN)

BT026-4 External Power Cartridge & Charger Combination (UK)

PW029-1 Charger and Power Supply For Charger (US)*

PW029-2 Charger and Power Supply For Charger (EUROPEAN)*

PW029-3 Charger and Power Supply For Charger (AUSTRALIAN)*

PW029-4 Charger and Power Supply For Charger (UK)*

*Battery not included.

SERVICE SCHOOLS & TRADESHOWS

Mark Your Calendars for our Upcoming 2014 Service Schools!

USA

2014 Service School dates have been released!

For additional details and registration information about these trainings, please visit the ["Service Schools"](#) page on our website.

Concentrator Training: June 9, 10, 11 in Ball Ground, GA

LOX Training: June 12 & 13 in Ball Ground, GA

**Ball Ground trainings include a tour of our CAIRE and SeQual manufacturing facility.*

Concentrator Training: August 18, 19, 20 in Buffalo, NY

LOX Training: August 21 & 22 in Buffalo, NY

**Buffalo trainings include a tour of our AirSep manufacturing facility.*

Concentrator Training: November 10, 11, 12 in Ball Ground, GA

LOX Training: November 13 & 14 in Ball Ground, GA

**Ball Ground trainings include a tour of our CAIRE and SeQual manufacturing facility.*

Europe

2014 Service School dates have been released!

For additional details and registration information about these trainings, please visit the ["Events & Service Schools"](#) tab of our website.

English LOX Training: April 15 & 16 in Wuppertal, Germany

English Concentrator Training: May 20, 21, 22 in Wuppertal Germany

French LOX Training: June 2 & 3 in France

French Concentrator Training: June 4, 5, 6 in France

Italian Concentrator Training: June 10, 11, 12 in Padova, Italy

Spanish LOX Training: June 16 & 17 in Barcelona, Spain

**Spanish Concentrator Training: June 18, 19, 20 in
Barcelona, Spain**

**German LOX Training: September 9 & 10 in Wuppertal,
Germany**

**German Concentrator Training: October 14, 15, 16 in
Wuppertal, Germany**

English LOX Training: October 28 & 29 in Wokingham, UK

Italian LOX Training: October 29 & 30 in Padova, Italy

**English Concentrator Training: November 18, 19, 20 in
Wokingham, UK**

Tradeshows

We will be attending the following trade shows this month.
We would love for you to stop by our booth!

CO-SRC Annual Meeting

April 30-May 2

Vail, CO

CAIRE

California Society for Pulmonary Rehabilitation-CSPR

May 14 - 16

San Diego, CA

CAIRE

FOCUS on Respiratory Care Spring Show

May 15 - 17

Orlando, FL

CAIRE

Canadian Society of Respiratory Therapy with OPIQ

May 22 - 24

Montreal, QB

CAIRE

CONTACT US

For ordering information, contact Customer Service:

United Kingdom +44 (0) 1189 367060

France +33 (0) 561 429 411

Germany +49 (0) 202 739 55420

Italy +39 049 879 9601

customerservice.europe@chartindustries.com

Australia/New Zealand Asia/Pac Rim +61 2 9749 4333 customerservice.australia@chartindustries.com 770.721.7759 csasia@chartindustries.com
The Americas US Toll-Free 770.721.7759 800.482.2473 customerservice.usa@chartindustries.com

For technical information, contact Technical Service:

United Kingdom +44 (0) 1189 367060
France +33 (0) 561 429 411
Germany +49 (0) 202 739 55420
Italy +39 049 879 9160
techservice.europe@chartindustries.com
Asia/Pac Rim +61 2 9749 4333
techservice.asia@chartindustries.com
The Americas 770.721.7759
techservice.usa@chartindustries.com
US Toll-Free 800.482.2473
techservice.usa@chartindustries.com

[Forward email](#)



Try it FREE today.

This email was sent to snystrom@sequal.com by julie.liberti@chartindustries.com | [Update Profile/Email Address](#) | Instant removal with [SafeUnsubscribe™](#) | [Privacy Policy](#).

Chart Industries | 2200 Airport Industrial Dr. | Ste. 500 | Ball Ground | GA | 30107