

May 2020 Med Tips

PRODUCT UPDATES

Companion 5™ Oxygen Concentrator Standard Compliance

A new version of the Companion 5™ concentrator is being released that is compliant to IEC 60601-1 edition 3.1 and IEC 60601-1-2 4th edition. Please refer to the operating and service instructions for concentrators compliant to the latest IEC 60601 edition (21420526 English and 21420526-C4 multi language) and Technical Service Manual (14940840) that are specific for this concentrator.

The new changes from the previous model Companion 5™ concentrators are:

- IP21 Water Ingress Protection: Including a silicone cover over the power switch
- All alarm LEDs will now be yellow
- O2 warm-up code: The display shows “O2 UP” until 85% outlet purity is reached
- New alarm sound: The audible alarm is now an intermittent beep. Alarm buzzer is separate from the board and is installed in the front case.

These updates will be available for the item numbers below:

Item Number	Description	Input Power	CE Mark?
21471908	Companion 5 Oxygen Concentrator	230 VAC, 50 Hz	Yes
21471909	Companion 5 Oxygen Concentrator with Bluetooth capability	230 VAC, 50 Hz	Yes

For more details about the Companion 5™, please refer to the Service Bulletin 21494864.

Concentrators with Air Outlet Option

The Air Outlet option on a Visionaire or New Life Elite enables a patient to use a nebulizing device. The air outlet provides a 6 lpm airflow at a pressure of 12 psig. The Air Outlet does not deliver oxygen-enriched air.

The New Life Intensity does not required an Air Outlet to use the nebulizer, as the outlet pressure is higher than the Visionaire or the New Life Elite.

Below is the list of the Visionaire and the New Life Elite available with the Air Outlet option:

Item Number	Description
AS098-708	AIRSEP, 5L, 230V, 50HZ, AIR, VISIONAIRE
AS098-711	AIRSEP,5L,230V,50HZ,AIR,W/O2 MONITOR,VISIONAIRE
AS005-270	AIR SEP,5L,230V/50HZ,AIR,O2 MON, W/ECO,OXYSIL,MULT-KT,UK CD
AS005-264	AIR SEP,5L,120/60,AIR,O2 MON, OXY SIL
AS005-265	AIR SEP,5L,120/60,AIR,O2 MON, OXY SIL,WITH BRAZILIAN CORD
AS005-244	AIR SEP,5L,230V/50HZ,O2 MON, OXY SIL,MULT-KT,ARGENTINA CORD
AS005-263	AIR SEP,5L,230/60,AIR,O2 MON, UK CORD,ARABIC

Please refer to Service Bulletin PN 21288973 for New Life models available and to Service Bulletin PN 21353847 for Visionaire models available.

Oxygen Delivery Accessories

In addition to the standard cannulas, CAIRE offers oxygen masks for oxygen delivery. Both Pediatric and Adult masks are available to order. They include a 7" tubing length. Please refer to the pictures below:



CU006-1: MASK, CONCENTRATION,
OVER THE EAR STYLE W/7' TUBING



CU007-1: MASK, PEDIATRIC,
CONCENTRATION, W/7' TUBING

All the other Oxygen delivery accessories are accessible in the CAIRE Knowledge center web site. Please use the below link for the Oxygen Delivery and Cannulas Accessory Sheet:

http://files.caireinc.com/ML-CONC0096_B_Cannulas_Accessories_Page.pdf

Liberator Drain Tube Diameters Explained

The Liberator reservoirs have been manufactured with a larger drain tube inside of the condensation pan since 11/24/2014. The change was made to reduce the possibility of the tube becoming blocked while the unit collects condensation.



Updated Style Drain Tube (Left, PN 20818093) and Previous Style drain tube (Right, PN 14224662)

Please refer to Service Bulletin PN 20820794 for more details.

Helpful Hints & FAQs

CAIRE Knowledge Center Web Site

CAIRE is proud to offer free access to a Knowledge Center for our customers. The CAIRE Knowledge Center contains a wide variety of documentation on our products and accessories. For ease, it is divided into different sections. Not only can our customers access user/technical manuals, parts lists and operating instructions which are available to be downloaded in pdf form, but it also contains information on design changes, part number modifications and other important information for our customers. This information is detailed in our Service Bulletin or Med Tips communications.

You can access our CAIRE Knowledge Center by using the following link:

<https://www.caireinc.com/providers/business-support-programs/knowledge-center/>

If you have any questions, please contact our Technical Service team at Techservice.global@caireinc.com



For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

Customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

Customerservice.europe@caireinc.com

© Copyright 2020 CAIRE Inc. All Rights Reserved. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

Please consult the applicable product instructions for use for product indications, contraindications, warnings, precautions, and detailed safety information



SERVICE BULLETIN

PN: 21494864

RELEASE DATE: April 24, 2020

MODEL: Companion 5™

ISSUE: Companion 5™ Oxygen Concentrator Standard Compliance

A new version of the Companion 5 concentrator is being released that is compliant to IEC 60601-1 edition 3.1 and IEC 60601-1-2 4th edition.

Please reference the operating and service instructions for concentrators compliant to the latest IEC 60601 edition (21420526 English and 21420526-C4 multi language) and Technical Service Manual (14940840) that are specific for this concentrator.

Note: For operating and service instructions prior to this latest design IEC 60601 version please refer to the existing User Manual (15062781 and 15062781-C4) and Technical Manual (14940837).

The new changes from the previous model Companion 5 concentrators are:

- IP21 Water Ingress Protection: Including a silicone cover over the power switch
- All alarm LEDs will now be yellow
- O2 warm-up code: The display shows “O2 UP” until 85% outlet purity is reached
- New alarm sound. The audible alarm is now an intermittent beep. Alarm buzzer is separate from the board and is installed in the front case. These updates will be available for the item numbers below:

Item Number	Description	Input Power	CE Mark?
21471908	Companion 5 Oxygen Concentrator	230 VAC, 50 Hz	Yes
21471909	Companion 5 Oxygen Concentrator with Bluetooth capability	230 VAC, 50 Hz	Yes

These changes do not apply to 120 V models of the Companion 5, which will continue to be manufactured with the previous design (items 15067005, 21190674, or 21225813). Item 21139079 will also continue to be manufactured at the previous design.

For customers in Europe who previously ordered item 15067021, going forward 21471908 or 21471909 are the available substitute items. Item 15067021 will still be available as necessary depending on product registration requirements for countries outside the EU.



Figure 1. Updated Companion 5 Exterior

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

Techservice.global@caireinc.com

Europe: +44 (0) 118 936 7060

Techservice.global@caireinc.com

For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

Customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

Customerservice.europe@caireinc.com

Copyright © 2020 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.



SERVICE BULLETIN

PN: 21288973

DATE: 10/01/2018

REVISED: 12/17/2018

MODEL: AirSep® NewLife® Intensity 10L and Elite 5L Oxygen Concentrators

ISSUE: Discontinuation of **select** models of AirSep® NewLife® Intensity 10L and AirSep® NewLife® Elite 5L. **The most common NewLife® models will still be available going forward.**

NOTES: Manufacturing of select models of AirSep® NewLife® Intensity 10L and AirSep® New Life® Elite 5L concentrators documented in the tables below will cease globally on December 31, 2018 to simplify the CAIRE Inc. product offerings and provide a more streamlined order and delivery process for our customers. CAIRE will continue to provide service and parts through 2021 to support warranty obligations.

The last day CAIRE Inc. will take orders for the models below is November 1, 2018. All orders are final and will be shipped per order instructions.

Note: Models previously manufactured without the O2 monitor feature will instead have the O2 monitor installed on any newly manufactured units going forward.

Table 1: AirSep® New Life® Elite 5L Concentrators

NewLife Elite SKU's - Stocking				
Model Type	Item	Description	Change	New Part Number
120V/60HZ Model	AS005-4	Standard, O2 Monitor, English Man, US Cord	No change	AS005-4
230V/50HZ Model	AS005-270	Standard, O2 Monitor, Air Outlet, Oxy Silent, Multi Lang. Man, UK Cord	No change	AS005-270

230V/50HZ Model	AS005-555	Standard, O2 Monitor, Oxy Silent, Multi Lang. Man, EUR Cord.	No change	AS005-555
120V/60HZ Model	AS005-264†	Standard, O2 Monitor, Oxy Silent, Air Outlet, Mask Kit, English Man, US Cord	No change	AS005-264†
120V/60HZ Model	AS005-265†	Standard, O2 Monitor, Oxy Silent, Air Outlet, Port. Man, BRA Cord.	No change	AS005-265†
230V/50HZ Model	AS005-244†	Standard, O2 Monitor, Air Outlet, Oxy Silent, Multi Lang. Man, ARG Cord	No change	AS005-244†
230V/50HZ Model	AS005-550	Dual Flow, O2 Monitor, English Man, EUR Cord	No change	AS005-550
230V/60HZ Model	AS005-241†	Standard, O2 Monitor, Multi Lang. Man, UK Cord, Arabic Labels	No change	AS005-241†
230V/60HZ Model	AS005-263†	Standard, O2 Monitor, Air Outlet, Multi Lang. Man, UK Cord, Arabic Labels	No change	AS005-263†

AirSep® New Life® Elite 5L SKU's Obsolete				
Model Type	Item	Description	Change	New Part Number
120V/60HZ Model	AS005-87	Standard, Non O2 Monitor, Air Outlet, English Man, US Cord	Replaced by	AS005-264
120V/60HZ Model	AS005-84	Standard, O2 Monitor, English Man, US Cord	Replaced by	AS005-4
120V/60HZ Model	AS005-1U	Re- MFG, Standard, Non O2 Monitor, English Man, US Cord	Replaced by	AS005-4
120V/60HZ Model	AS005-4U	Re- MFG, Standard, O2 Monitor, English Man, US Cord	Replaced by	AS005-4
120V/60HZ Models	AS005-243	Standard, O2 Monitor, Oxy-Silent, English Manual, US Cord	Replaced by	AS005-264
120V/60HZ Models	AS005-1	Standard, Non O2 Monitor, Oxy-Silent, English Manual, US Cord	Replaced by	AS005-4
230V/50HZ Model	AS005-508	Standard, Non O2 Monitor, Air Outlet, Multi Lang. Man, EUR Cord	Replaced by	AS005-270
230V/50HZ Model	AS005-225	Standard, O2 Monitor, Multi Lang. Man, EUR Cord	Replaced by	AS005-555

230V/50HZ Model	AS005-255	Standard, O2 Monitor, Multi Lang. Man, UK Cord	Replaced by	AS005-270
230V/50HZ Model	AS005-300	Standard, O2 Monitor, Chinese Manual, China Cord	Replaced by	AS005-555*
230V/50HZ Model	AS005-227	Standard, O2 Monitor, Mutli Lang. Man, AUS Cord	Replaced by	AS005-555*
230V/50HZ Model	AS005-230	Standard, O2 Monitor, Multi Lang. Man, UK Cord	Replaced by	AS005-270
230V/50HZ Model	AS005-511	Standard, O2 Monitor, Air Outlet, Multi Lang. Man, EUR Cord	Replaced by	AS005-270*
230V/50HZ Models	AS005-502	Standard, Non O2 Monitor, Multi Lang. Manual, EUR Cord	Replaced by	AS005-555
230V/50HZ Models	AS005-266	Standard, Non O2 Monitor, Multi Lang. Manual, UK Cord	Replaced by	AS005-270
230V/60HZ Model	AS005-3	Standard, Non O2 Monitor, English Manual, EUR Cord	Replaced by	No replacement
230V/60HZ Model	AS005-261	Standard, Non O2 Monitor, Air Outlet, Port Manual, EUR Cord	Replaced by	No replacement
230V/60HZ Model	AS005-6	Standard, O2 Monitor, English Man, EUR Cord	Replaced by	No replacement

Table 2: AirSep® NewLife® Intensity 10L

AirSep® NewLife® Intensity 10L SKU's Stocking				
Model Type	Item	Description	Change	New Part Number
120V/60HZ Model	AS099-101	Standard, O2 Monitor, English Man, US Cord (State-of-the-Art Design)	No change	AS099-101
120V/60HZ Model	AS099-4	Dual Flow, O2 Monitor, English Man, US Cord	No change	AS099-4
230V/50HZ Model	AS099-10	Dual Flow, O2 Monitor, Multi Lang Man, EUR Cord	No change	AS099-10
230V/50HZ Model	AS099-103	Standard, O2 Monitor, Multi Lang Man, Euro Cord (State-of-the-Art Design)	No change	AS099-103
230V/50HZ Model	AS099-210	Standard, O2 Monitor, Multi Lang Man, UK Cord (State-of-the-Art Design)	No change	AS099-210

230V/50HZ Model	AS099-211	Standard, O2 Monitor, Multi Lang Man, AUS Cord	No change	AS099-211
230V/50HZ Model	AS099-213	Standard, O2 Monitor, Humidifier Bottle Kit, Mult Lang Man, EUR Cord (State-of-the-Art Design)	No change	AS099-213
230V/50HZ Model	AS099-201	Dual Flow, O2 Monitor, Multi Lang Man, UK Cord	No change	AS099-201
230V/50HZ Model	AS099-200	Dual Flow, O2 Monitor, Multi Lang Man, AUS Cord	No change	AS099-200

AirSep® NewLife® Intensity 10L SKU's obsolete				
Model Type	Item	Description	Change	New Part Number
120V/60HZ Model	AS099-100	Standard, Non O2 Monitor, English Manual, US Cord	Replaced by	AS099-101
120V/60HZ Model	AS099-205	Standard, Non O2 Monitor, English Manual, US Cord, Mask Kit	Replaced by	AS099-101**
120V/60HZ Model	AS099-206	Standard, O2 Monitor, English Manual, Brazilian/EUR Cord, Brazilian Labels	Replaced by	AS099-213
120V/60HZ Model	AS099-1	Dual Flow, Non O2 Monitor, English Manual, US Cord	Replaced by	AS099-4
120V/60HZ Model	AS099-100U	Re-MFG Standard, Non O2 Monitor, English Manual, US Cord	Replaced by	AS099-101
120V/60HZ Model	AS099-101U	Re-MFG Standard, O2 Monitor, English Manual, US Cord	Replaced by	AS099-101
120V/60HZ Model	AS099-1U	Re-MFG Dual Flow, Non O2 Monitor, English Manual, US Cord	Replaced by	AS099-4
120V/60HZ Model	AS099-4U	Re-MFG Dual Flow, O2 Monitor, English Manual, US Cord	Replaced by	AS099-4
230V/50HZ Model	AS099-102	Standard, Non O2 Monitor, Multi Lang Manual, EUR Cord	Replaced by	AS099-103*
230V/50HZ Model	AS099-207	Standard, Non O2 Monitor, Multi Lang Manual, UK Cord	Replaced by	AS099-210
230V/50HZ Model	AS099-208	Standard, Non O2 Monitor, Multi Lang Manual, AUS Cord	Replaced by	AS099-211

230V/50HZ Model	AS099-212	Standard, O2 Monitor, Multi Lang Manual, ARG Cord	Replaced by	AS099-211***
230V/50HZ Model	AS099-209	Standard, Non O2 Monitor, Multi Lang Manual, EUR Cord, Humidifier Kit	Replaced by	AS099-213
230V/50HZ Model	AS099-20	Standard, O2 Monitor, Chinese Manual, Chinese Cord	Replaced by	AS099-103*
230V/50HZ Model	AS099-203	Standard, O2 Monitor, Multi Lang Manual, EUR Cord	Replaced by	AS099-10
230V/50HZ Model	AS099-202	Standard, O2 Monitor, Multi Lang Manual, EUR Cord, Humidifier Kit	Replaced by	AS099-213
230V/50HZ Model	AS099-204	Standard, O2 Monitor, Humidifier Bottle Kit, Multi Lang Man, EUR Cord	Replaced by	AS099-213
230V/50HZ Model	AS099-7	Dual Flow, Non O2 Monitor, Multi Lang Manual, EUR Cord	Replaced by	AS099-10
230V/50HZ Model	AS099-215	Dual Flow, Non O2 Monitor, Multi Lang Manual, UK Cord	Replaced by	AS099-201
230V/50HZ Model	AS099-214	Dual Flow, Non O2 Monitor, Multi Lang Manual, AUS Cord	Replaced by	AS099-200
230V/50HZ Model	AS099-104	Dual Flow, O2 Monitor, Chinese Cord, Chinese Manual	Replaced by	AS099-10*
230V/50HZ Model	AS099-103U	Re-MFG Standard, O2 Monitor, Multi Lang Manual, EUR Cord	Replaced by	AS099-103
230V/50HZ Model	AS099-10U	Re-MFG Dual Flow, O2 Monitor, Multi Lang Man, EUR Cord	Replaced by	AS099-10

†Models previously manufactured without the O2 monitor option will be manufactured with the O2 monitor option going forward.

*Must purchase item 5591-SEQ as an accessory to allow equivalent plug type as the obsolete model.

**Purchase item MS013-1 as an accessory to receive mask and tubing.

***Replacement model equipped with a 3-prong (grounded) plug in place of 2-prong plug.

Note: Models replacing the obsolete Chinese model units include a multi-lingual manual but do not contain Chinese as one of the language options. Contact Customer Service or your local CAIRE sales rep to inquire about user manual language options.

CAIRE will continue to manufacture and stock a variety of AirSep® NewLife® Intensity 10L and AirSep® New Life® Elite 5L models and a CAIRE representative can help you select the right concentrator for your business needs.

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 1344 403100

www.cairemedical.com

For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 1344 403100

www.cairemedical.com

Copyright © 2018 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.



SERVICE BULLETIN

PN: 21353847

DATE: December 13, 2018

REVISION DATE(S): March 20, 2020 (Rev B)

MODEL: VisionAire® Oxygen Concentrators

ISSUE: Cease commercial activity of select models of VisionAire® Oxygen Concentrators. The most common VisionAire® models will still be available going forward.

NOTES: Manufacturing of select models of VisionAire® concentrators documented in the tables below ceased globally on December 31, 2018 to simplify the CAIRE Inc. product offerings and provide a more streamlined order and delivery process for our customers. CAIRE will continue to provide service and parts through 2021 to support warranty obligations.

Table 1: VisionAire Concentrators – No Changes

VisionAire SKU's - Stocking			
Model Type	Item Number	Description	Change
230V/50HZ	AS098-708	AIRSEP,5L,230V,50HZ,AIR, VISIONAIRE	NO CHANGE
230V/50HZ	AS098-805	AIRSEP,5L,230V,50HZ,W/O2 MONITOR,VISIONAIRE,CHINA	NO CHANGE
230V/50HZ	AS098-806*	AIRSEP,5L,230V,50HZ,W/O2, CHINA	NO CHANGE
230V/50HZ	AS098-17	AIRSEP,5L,230V,50HZ,W/O2 MON,AU CORD, VA	NO CHANGE
230V/50HZ	AS098-18	AIRSEP, 5L, 230V, 50HZ, UK CORD, VA	NO CHANGE
230V/60HZ	AS098-20	AIRSEP,5L,230V,60HZW/O2 MON,UK CORD, VA	NO CHANGE
230V/50HZ	AS098-5	AIRSEP,5L,230V,50HZ,W/O2 MONITOR,VISIONAIRE	NO CHANGE
230V/60HZ	AS098-6	AIRSEP,5L,230V,60HZ,WITH O2 MONITOR,VISIONAIRE	NO CHANGE
230V/50HZ	AS098-7	AIRSEP,5L,230V,50HZ,W/O2 MON, UK CORD,VISIONAIRE	NO CHANGE
230V/50HZ	AS098-711	AIRSEP,5L,230V,50HZ,AIR,W/O2 MONITOR,VISIONAIRE	NO CHANGE

230V/50HZ	AS098-15	AIRSEP,5L,230V,50HZ, WITH O2 MONITOR,SWISS CORD,VISIONAIRE	NO CHANGE
230V/60HZ	AS098-27	AIRSEP, 5L, 230V, 60HZ, WITH O2 MON, KOREA CORD, VA	NO CHANGE

Table 2: VisionAire Concentrators – Obsolete No Replacements

VisionAire SKU's – Obsolete No Replacement			
Model Type	Item	Description	Change
230V/60HZ	AS096-5	AIRSEP,3L,230V,60HZ, O2 MONITOR, VISIONAIRE 3	OBS NO RB
115V/60HZ	AS096-6	AIRSEP,3L,115V,60HZ, VISIONAIRE 3	OBS NO RB
115V/60HZ	AS096-7	AIRSEP,3L,115V,60HZ,O2 MONITOR, VISIONAIRE 3	OBS NO RB
230V/50HZ	AS096-1	AIRSEP,3L,230V,50HZ, VISIONAIRE 3	OBS NO RB
230V/50HZ	AS096-3	AIRSEP,3L,230V,50HZ,O2 MONITOR, UK VISIONAIRE 3	OBS NO RB
115V/60HZ	AS096-8	AIRSEP,2L,115V,60HZ,O2 MONITOR, VISIONAIRE 2	OBS NO RB
230V/50HZ	AS096-2	AIRSEP,3L,230V,50HZ,O2 MONITOR, VISIONAIRE 3	OBS NO RB
230V/50HZ	AS096-2AP	AIRSEP,3L,230V,50HZ,A/P,W/O2, VISIONAIRE 3	OBS NO RB
230V/50HZ	AS096-3AP	AIRSEP,3L,230V,50HZ,AP,W/O2, VISIONAIRE 3	OBS NO RB
230V/50HZ	AS098-26	AIRSEP,5L,230V,50HZ,W/O2	OBS NO RB

Table 3: VisionAire Concentrators – Obsolete, Replace with Companion 5 with BT PN 21225813 or Companion 5 PN 15067005

VisionAire SKU's – Obsolete No Replacement				
Model Type	Item	Description	Change	New Part Number
115V/60HZ	AS098-101	AIRSEP,5L,115V,60HZ,VISIONAIRE	OBS RB	21225813 or 15067005
115V/60HZ	AS098-101U	AIRSEP,VISIONAIRE V,RE-MFG	OBS RB	21225813 or 15067005
115V/60HZ	AS098-104	AIRSEP,5L,115V,60HZ,WITH O2	OBS RB	21225813 or 15067005
115V/60HZ RE-MFG	AS098-104U	AIRSEP,VISIONAIRE V, RE-MFG	OBS RB	21225813 or 15067005

115V/60HZ	AS098-707*	AIRSEP,5L,115V,60HZ,AIR	OBS RB	21225813 or 15067005
115V/60HZ	AS098-710	AIRSEP,5L,115V,60HZ,AIR,W/O2	OBS RB	21225813 or 15067005
115V/60HZ	AS098-4	AIRSEP,5L,115V,60HZ,WITH O2	OBS RB	21225813 or 15067005
115V/60HZ	AS098-1	AIRSEP,5L,115V,60HZ,VISIONAIRE	OBS RB	21225813 or 15067005
115V/60HZ	AS098-101C	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-101S	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-104C	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-104S	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-1C	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-4C	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-4M	CARE PROGRAM VISIONAIRE 5 W/OM	OBS RB	21225813 or 15067005
115V/60HZ	AS098-4S	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-1S	AIRSEP VISIONAIRE,115V,60HZ	OBS RB	21225813 or 15067005
115V/60HZ	AS098-604	AIRSEP,5L,115V,60HZ,O2 MON	OBS RB	21225813 or 15067005
115V/60HZ	AS098-107	AIRSEP,5L,115V,60HZ,AIR	OBS RB	21225813 or 15067005
115V/60HZ	AS098-104CORE	AIRSEP,5L,115V,60HZ,WITH O2	OBS RB	21225813 or 15067005

115V/60HZ	AS098-101CORE	AIRSEP,5L,115V,60HZ,VISIONAIRE	OBS RB	21225813 or 15067005
115V/60HZ	AS098-110	AIRSEP,5L,115V,60HZ,AIR,W/O2	OBS RB	21225813 or 15067005
115V/60HZ	AS098-604	AIRSEP,5L,115V,60HZ,O2 MON	OBS RB	21225813 or 15067005
115V/60HZ	AS098-713*	AIRSEP, 5L, 115V, 60HZ, AIR	OBS RB	21225813 or 15067005
115V/60HZ	AS098-21	AIRSEP, 5L, 115V, 60HZ, W/BESMED STARTER KIT, VA	OBS RB	21225813 or 15067005
115V/60HZ	AS098-719	AIRSEP, 5L, 115V, 60HZ, AIR, W/BESMED STARTER KIT, VA	OBS RB	21225813 or 15067005

Table 4: VisionAire Concentrators – Obsolete, Replace with VisionAire

VisionAire SKU's – Obsolete No Replacement				
Model Type	Item	Description	Change	New Part Number
230V/50HZ	AS098-16	AIRSEP, 5L, 230V, 50HZ, AU CORD	OBS RB	AS098-17
230V/50HZ	AS098-2	AIRSEP, 5L,230V,50HZ,VISIONAIRE	OBS RB	AS098-5
230V/60HZ	AS098-712‡	AIRSEP,5L,230V,60HZ,AIR,W/O2	OBS RB	AS098-6
230V/50HZ	AS098-716*	AIRSEP, 5L, 230V, 50HZ, AIR, W/O2, UK CORD	OBS RB	AS098-711
230V/60HZ	AS098-3	AIRSEP,5L,230V,60HZ,VISIONAIRE	OBS RB	AS098-6
230V/50HZ	AS098-715*	AIRSEP, 5L, 230V, 50HZ, AIR, UK CORD	OBS RB	AS098-708
230V/60HZ	AS098-19	AIRSEP, 5L, 230V, 60HZ, UK CORD	OBS RB	AS098-20
230V/60HZ	AS098-200	AIRSEP, 5L, 230V, 60HZ, UK CORD, ARABIC	OBS RB	AS098-20
230V/50HZ	AS098-22**	AIRSEP, 5L, 230V, 50HZ	OBS RB	AS098-5
230V/50HZ	AS098-23**	AIRSEP, 5L, 230V, 50HZ, W/O2	OBS RB	AS098-5
230V/50HZ	AS098-24**	AIRSEP, 5L, 230V, 50HZ, W/O2	OBS RB	AS098-5
230V/50HZ	AS098-25†	AIRSEP, 5L, 230V, 50HZ, W/O2	OBS RB	AS098-5
230V/60HZ	AS098-709	AIRSEP,5L,230V,60HZ,AIR	OBS RB	AS098-6
230V/60HZ	AS098-714*	AIRSEP, 5L, 230V, 60HZ, AIR, BRAZIL CORD	OBS RB	AS098-6
230V/60HZ	AS098-717‡	AIRSEP, 5L, 230V, 60HZ, AIR, UK CORD	OBS RB	AS098-805
230V/50HZ	AS098-718‡‡	AIRSEP, 5L, 230V, 50HZ, AIR, NEBULIZER	OBS RB	AS098-711

230V/50HZ	AS098-802	AIRSEP,5L,230V,50HZ	OBS RB	AS098-805
115V/60HZ	AS098-1M	CARE PROGRAM VISIONAIRE 5	OBS RB	AS098-5
230V/60HZ	AS098-200	AIRSEP, 5L, 230V, 60HZ, UK CORD ARABIC	OBS RB	AS098-20
230V/50HZ	AS098-5AP	AIRSEP,5L,230V,50HZ,A/P,W/O2	OBS RB	AS098-5
230V/50HZ	AS098-5C	AIRSEP VISIONAIRE,230V,50HZ	OBS RB	AS098-5
230V/50HZ	AS098-5M	AIRSEP VISIONAIRE,230V,50HZ	OBS RB	AS098-5
230V/50HZ	AS098-5S	AIRSEP VISIONAIRE,230V,50HZ	OBS RB	AS098-5

***Must purchase item 5591-SEQ as an accessory to allow equivalent plug type.**

****Purchase item KI506-1 as an accessory to receive cannula and bottle. †Purchase starter kit separately S0316-1.**

‡Will no longer have air outlet.

‡‡Will need to purchase Nebulizer PN R0132.

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

techservice.global@caireinc.com

Europe: +44 (0) 118 936 7060

techservice.global@caireinc.com

For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

customerservice.europe@caireinc.com

Copyright © 2020 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.



SERVICE BULLETIN

PN: 20820794

RELEASE DATE: October 28, 2014

REVISION DATE(S): March 20, 2019 (Rev B)

MODELS: All Liberator Reservoirs

ISSUE: All Liberator reservoirs will be equipped with a larger diameter drain tube.

NOTES: Effective 11/24/2014, the Liberator reservoirs will be manufactured with a larger drain tube inside of the condensation pan.

The purpose of the larger diameter drain tube will be to reduce the possibility of the tube clogging while the unit collects condensation.

The Drain tube acts as a hose for water to travel from the condensation pan to the drain bottle. The drain bottle allows the patient to empty the water that has been collected from the unit.

This change was made to insure the optimal amount of condensation can be collected at a fast rate of speed when the unit is running at a higher setting. This will eliminate the down time between draining the condensation from the unit to the drain bottle while increasing the amount of condensation that has been collected.

Figure 1 shows two different drain tubes. The one on the left is the new style drain tube. The tube on the right is the previous style drain tube.

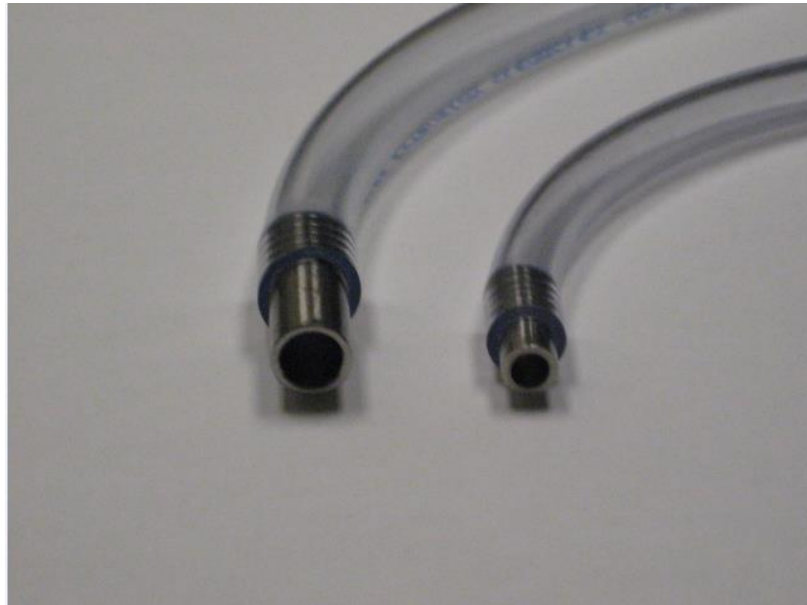


Figure 1. *New Style Drain Tube (Left, PN 20818093) and Previous Style drain tube (Right, PN 14224662)*

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 (0) 118 936 7060

www.cairemedical.com

For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

www.cairemedical.com

Europe: +44 (0) 118 936 7060

www.cairemedical.com

Copyright © 2019 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.