


PRODUCT UPDATES

Eclipse 3 case color change

CAIRE® announces the color change for the SeQual® Eclipse 3 cases to match the SeQual® Eclipse® 5 case color. Please note when ordering Eclipse® 3 spare case parts or returning an Eclipse® 3 to a CAIRE® authorized service center that the case panel colors may not match if case panels require replacing.

Once any existing stock of the older color case material is depleted, the newer case color will be the only color available for spare parts or service

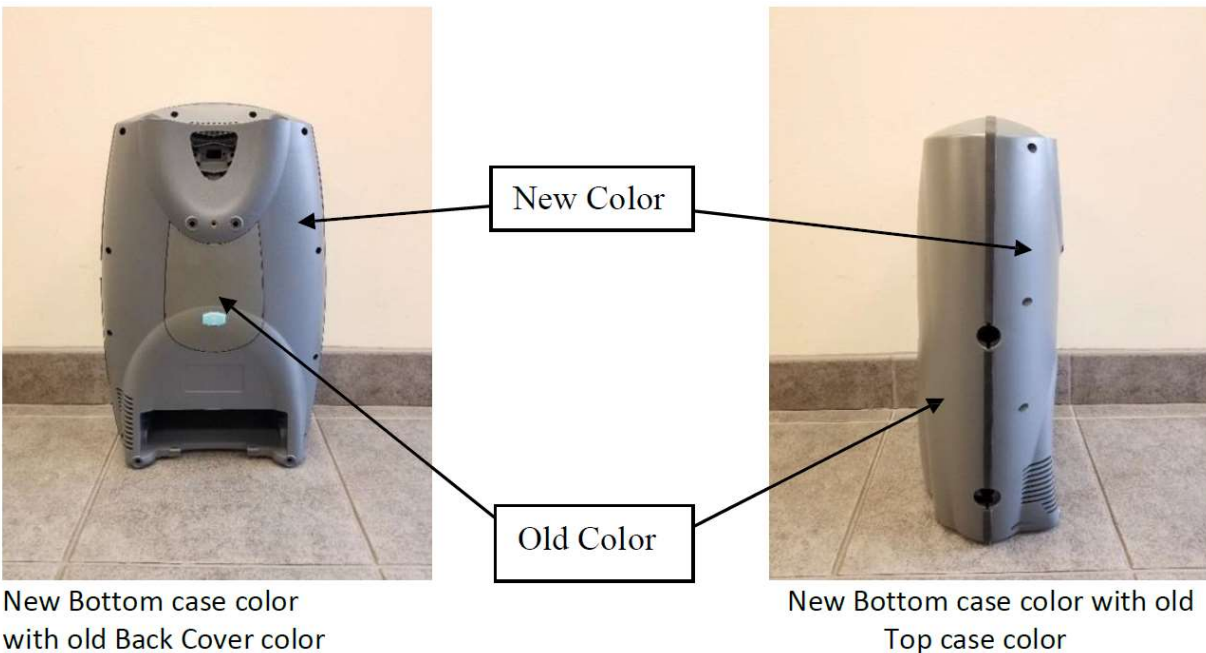

CAIRE® POC battery lithium content

The table below details the battery lithium content of the CAIRE® Portable Oxygen Concentrator.

A Material Safety Data Sheet is available for the batteries upon request to TechnicalServiceGlobal@caireinc.com, this document is required by the Air Lines when transporting a Portable Oxygen Concentrator on board.

Product	Battery Part Number	Battery Output Voltage	Battery Capacity	Weight of Lithium contained in Battery
SAROS™	9723-SEQ	14.8 VDC	88 W-hours	7.20 grams
Eclipse®	7082-SEQ	14.8 VDC	2 x 97.5 W-hours	2 x 7.92 grams (15.84 grams Total)
eQuinox™ 12 cell	4823-SEQ	14.8 VDC	89 W-hours	7.38 grams
eQuinox™ 24 cell	4972-SEQ	14.8 VDC	2 x 95 W-hours	2 x 7.92 grams (15.84 grams Total)
FreeStyle® Comfort 8 cell	BT034-1S	14.4 VDC	96.5 W-hours	7.68 grams
FreeStyle® Comfort 16 cell	BT035-1S	14.4 VDC	2 x 96.5 W-hours	2 x 7.68 grams (15.36 grams Total)
FreeStyle® 3 with Internal Battery	BT018-1S	14.8 VDC	59.2 W-hours	4.8 grams
FreeStyle® 5 with Internal Battery	BT021-1S	14.8 VDC	76.96 W-hours	5.68 grams
FreeStyle® 3/5 with External Battery	BT033-1	14.4 VDC	96.5 W-hours	7.92 grams
Focus™	BT023-1	14.8 VDC	38.48 W-hours	3.14 grams

Helpful Hints & FAQs

NewLife® Concentrator circuit board replacement

The old circuit board style is no longer available from CAIRE, when replacing a circuit board on a NEWLIFE Concentrator an upgrade to the current 60601 standard is required.

New retrofit KIT PN:

Kit Part Number	Main Board Included in Kit	Model
KI608-4	CB200-4	NewLife 5L (230 V, 50 Hz)
KI608-3	CB200-3	NewLife 5L (120 V, 60 Hz)
KI608-2	CB200-2	NewLife 10L (230 V, 50 Hz)
KI608-1	CB200-1	NewLife 10L (120 V, 60 Hz)

Please refer to Service Bulletin PN: 21329212 for more details.

New CAIRE Website

- a) **What is the new Website address of CAIRE?**
- b) The new Website can be accessed at the following address: <https://www.caireinc.com/> and the Respiratory Healthcare Knowledge Center can be accessed at:
<https://www.caireinc.com/providers/business-support-programs/knowledge-center/>

**CONTACT:**

For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

www.caireinc.com/providers

Europe: +44 (0) 118 936 7060

www.caireinc.com/providers

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

www.caireinc.com/providers

Europe: +44 (0) 118 936 7060

www.caireinc.com/providers

© Copyright 2019 CAIRE Inc. All Rights Reserved. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

Please consult the applicable product instructions for use for product indications, contraindications, warnings, precautions, and detailed safety information.