

Miscellaneous Parts

Description	L20	L30	L37	L45	L60	#
POI Multi-languages	MN234-C4	MN234-C4	MN234-C4	MN234-C4	MN234-C4	1
Top Label – All Languages	13717377	13717377	13717377	13717377	13717377	1
* Symbolic Label	Reference Service Bulletin 21174579					1
Setup Instructions - English	10668160	10668160	10668160	10668160	10668160	1
Setup Instructions - Italian	10668160	10668160	10668160	10668160	10668160	1
Setup Instructions - Norwegian	10734657	10734657	10734657	10734657	10734657	1
Setup Instructions - Dutch	10668160	10668160	10668160	10668160	10668160	1
Setup Instructions - Swedish	10736978	10736978	10736978	10736978	10736978	1
Setup Instructions - Spanish	10743529	10743529	10743529	10743529	10743529	1
Setup Instructions - Japanese	10757365	10757365	10757365	10757365	10757365	1
Setup Instructions - Danish	10757373	10757373	10757373	10757373	10757373	1
Setup Instructions - German	10668160	10668160	10668160	10668160	10668160	1
Setup Instructions - French	10668160	10668160	10668160	10668160	10668160	1
Setup Instructions - Portuguese	11032171	11032171	11032171	11032171	11032171	1
Setup Instructions - Finnish	11676170	11676170	11676170	11676170	11676170	1
Setup Instructions - Greek	11771599	11771599	11771599	11771599	11771599	1
Setup Instructions - Hungarian	11837295	11837295	11837295	11837295	11837295	1
Setup Instructions - Hebrew	13226181	13226181	13226181	13226181	13226181	1
Setup Instructions - Turkish	13957400	13957400	13957400	13957400	13957400	1

* Bottom labels for all languages have been replaced by the symbolic label.

G4 LIBERATOR® RESERVOIR




Configurable List


Key	Description	Lib 20	Lib 30	Lib 37	Lib 45	Lib 60	#
	Shroud - G4 Blue	13726943	13726943	13726943	13726943	13726943	1
	Shroud - G4 Light Gray	14026877	14026877	14026877	14026877	14026877	1
	Shroud - G4 Dark Gray	13693707	13693707	13693707	13693707	13693707	1
1	Dewar	11033975	11034054	11033916	11034118	11389023	1
2	DF Manifold	20583547	20583578	20583577	20583576	20583576	1
3	Bracket Assembly (Design after 4.1.2020)	13307335S*	13307335S*	13307335S*	13307335S*	13307335S*	1
3	Bracket Assembly (Design before 4.1.2020)	21495392S*	21495392S*	21495392S*	21495392S*	21495392S*	1
4	G4 Harness Assembly	13787869	13787869	13787869	13787869	13787869	1
7	Economizer Assembly (Design after 4.1.2020)	13277851S*	13277851S*	13277851S*	13277851S*	13277851S*	1
7	Economizer Assembly (Design before 4.1.2020)	21495393S*	21495393S*	21495393S*	21495393S*	21495393S*	
8	Vent Valve	20580870	20580870	20580870	20580870	20580870	1
16	0-15 LPM Vaporizer Coil	11018994S	11018994S	11018994S	11018994S	11018994S	1
17	0-15 LPM Breathing Coil	11940524S	11940524S	11940524S	11940524S	11940524S	1
28	Drain Tube Assembly	10657807	10657807	10657807	10657807	10657807	1
36	O-Ring - Viton - Manf to Dewar	CA405086	CA405086	CA405086	CA405086	CA405086	1
38	Split-Lock Washer - Manifold	CA200038	CA200038	CA200038	CA200038	CA200038	5
39	FCV Extension G4	13168347	13168347	13168347	13168347	13168347	1
52	Manifold Bolts - SS 1/4-20*5/8"	2910501	2910501	2910501	2910501	2910501	5
62	Cable Tie	10735983	10735983	10735983	10735983	10735983	4
72	Top Fill PB QDV	15101621	15101621	15101621	15101621	15101621	1
	Top Fill LifeOx QDV	CA003778T	CA003778T	CA003778T	CA003778T	CA003778T	1
	Pop-Off Svce Assy (Includes 73-77)	10576513-SV	10576513-SV	10576513-SV	10576513-SV	10576513-SV	1
73	Pop-Off Assy Screw - SS #8-32*3/8	2913821	2913821	2913821	2913821	2913821	2
74	Split-Lock Washer - Pop-Off Assy	CA405145	CA405145	CA405145	CA405145	CA405145	2
75	Pop-Off Assembly (see page 6)	11908049	11908049	11908049	11908049	11908049	1
	Pop-Off Spring	10931351	10931351	10931351	10931351	10931351	1
77	Pop-Off Sleeve	10576441	10576441	10576441	10576441	10576441	1
	Packaging Assembly	13500768	20659293-SV	20659293-SV	13500784	13500813	1
	Packaging Assy Tall QDV (Penox)	13938647	13939527	13939527	13940000	13940018	1
79	Dust Cover - Black, TF QDV	15075347	15075347	15075347	15075347	15075347	1
84	Side Fill CAIRE QDV	10577858T	10577858T	10577858T	10577858T	10577858T	1
85	Caplug - Red, SF QDV	CA403468	CA403468	CA403468	CA403468	CA403468	1
	SF QDV Lip Seal	CA110104	CA110104	CA110104	CA110104	CA110104	1
	SF QDV O-Ring Hex Coupler	CA404838	CA404838	CA404838	CA404838	CA404838	1
86	Humidifier Adapter Elbow	CA405451	CA405451	CA405451	CA405451	CA405451	1
87	Condensate Bottle Bracket	10584513	10584513	10584513	10584513	11892892	1
88	Condensate Bottle	CA404899	CA404899	CA404899	CA404899	CA404899	1
90	Shroud Washer SS #10	2910991	2910991	2910991	2910991	2910991	1
91	Shroud Screw SS #10-24*1/2"	2910391	2910391	2910391	2910391	2910391	1

*Reference Service Bulletin 21495031 if replacing the Flow Control Valve, Secondary Relief Valve, or Economizer Valve

G4 LIBERATOR® RESERVOIR

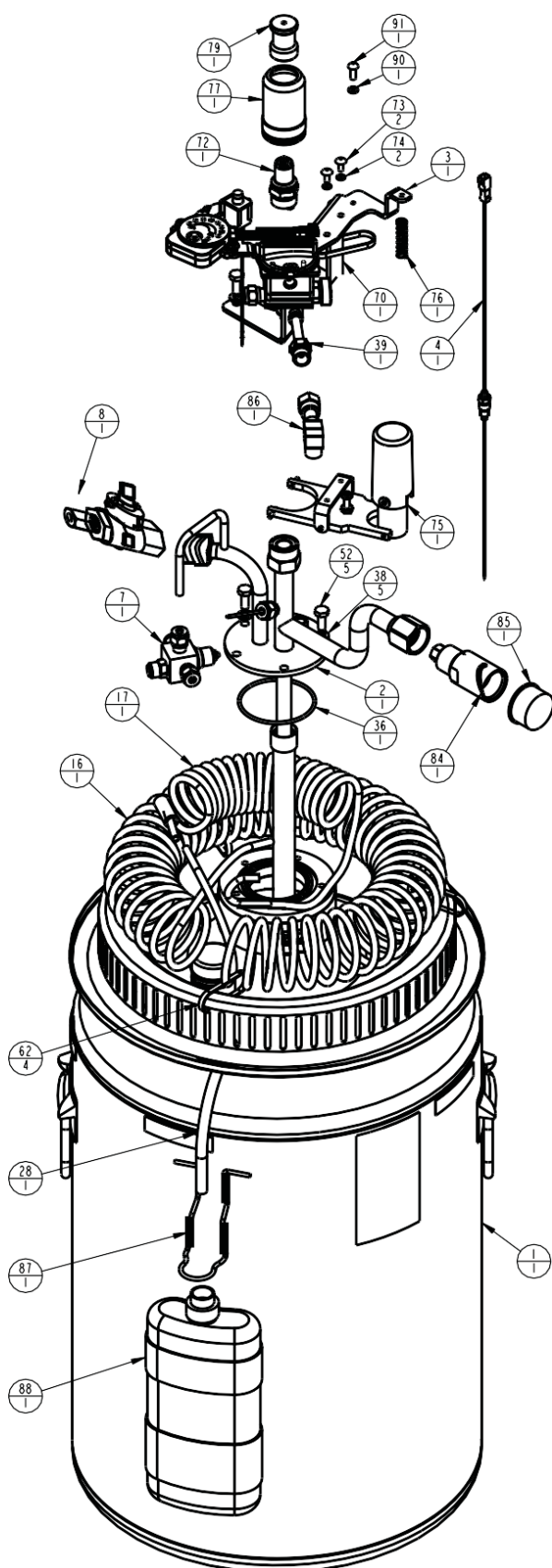

Design before 4.1.2020

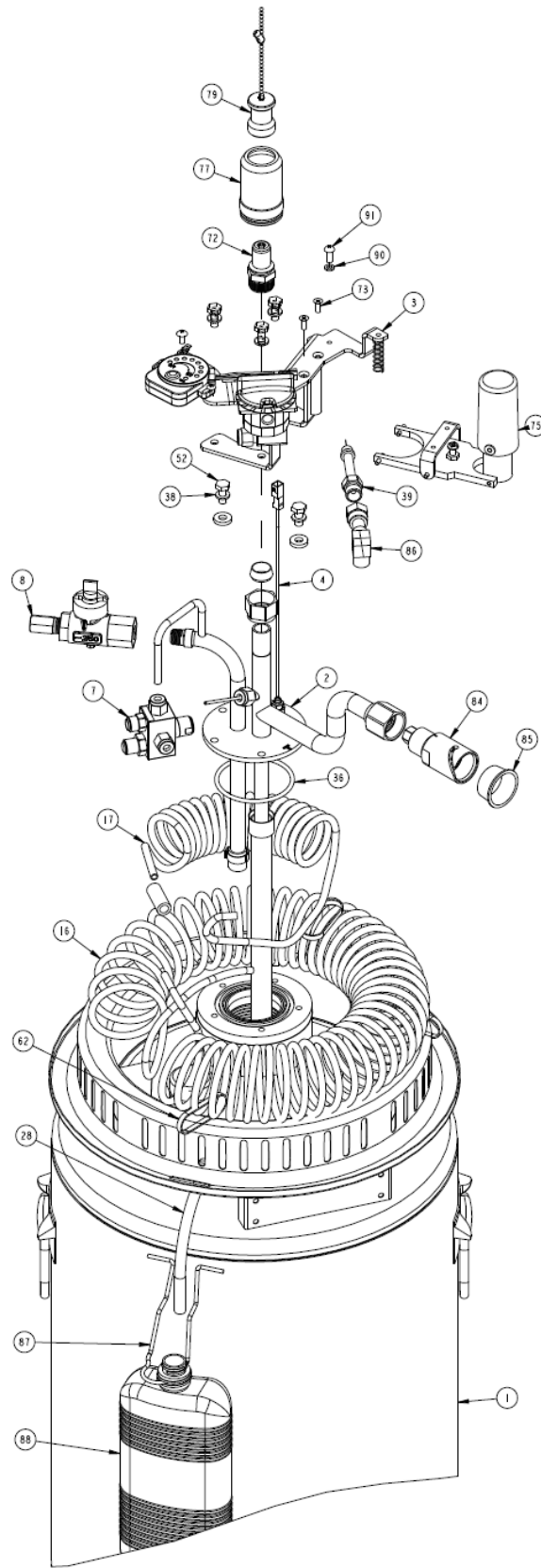

Design after 4.1.2020


G4 LIBERATOR® RESERVOIR



Bracket Assembly


Key	Description	Lib 20	Lib 30	Lib 37	Lib 45	Lib 60	#
1	Bracket G4	13708104	13708104	13708104	13708104	13708104	1
2	0-15LPM 16 pos. Flow Control Valve (Design after 4.1.2020)	21449808S*	21449808S*	21449808S*	21449808S*	21449808S*	1
2	0-15LPM 16 pos. Flow Control Valve (Design before 4.1.2020)	13110145*	13110145*	13110145*	13110145*	13110145*	
3	30 PSI Secondary Relief Valve (Design after 4.1.2020)	21215214*	21215214*	21215214*	21215214*	21215214*	1
4	0-15 LPM 16 Position Decal	13086149	13086149	13086149	13086149	13086149	1
5	16 Position Lock Plate	13095627	13095627	13095627	13095627	13095627	1
6	Flow Control Valve Screws	20724438	20724438	20724438	20724438	20724438	2
7	Flow Control Knob - Red	10567941	10567941	10567941	10567941	10567941	1
8	G4 Service Meter	13352401	13352401	13352401	13352401	13352401	1
9	Meter Screw - SS #6-32*3/8"	14263928	14263928	14263928	14263928	14263928	2
10	Ground #8 Spade Terminal	10597904	10597904	10597904	10597904	10597904	1
11	Split-Lock Washer - Ground - #8	CA405145	CA405145	CA405145	CA405145	CA405145	1
12	Screw - Ground - SS #8-32*3/8"	CA405164	CA405164	CA405164	CA405164	CA405164	1
13	Ferrule 1/4	B-775063-00	B-775063-00	B-775063-00	B-775063-00	B-775063-00	1
15	Tube Nut 1/4	B-775265-00	B-775265-00	B-775265-00	B-775265-00	B-775265-00	1

*Reference Service Bulletin 21495031 if replacing the Flow Control Valve, Secondary Relief Valve, or Economizer Valve

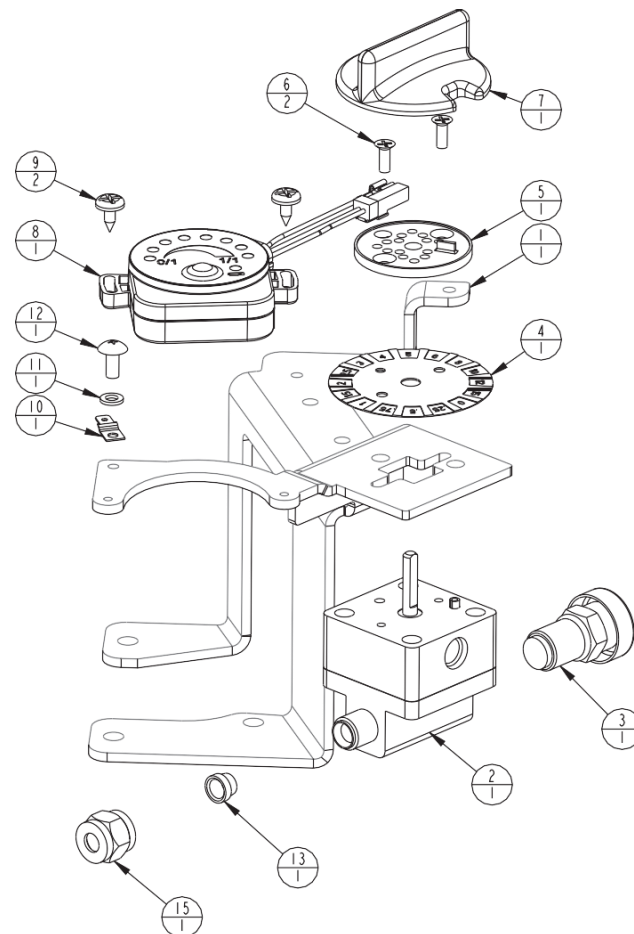

Bracket Assembly (Design before 4.1.2020)

G4 LIBERATOR® RESERVOIR

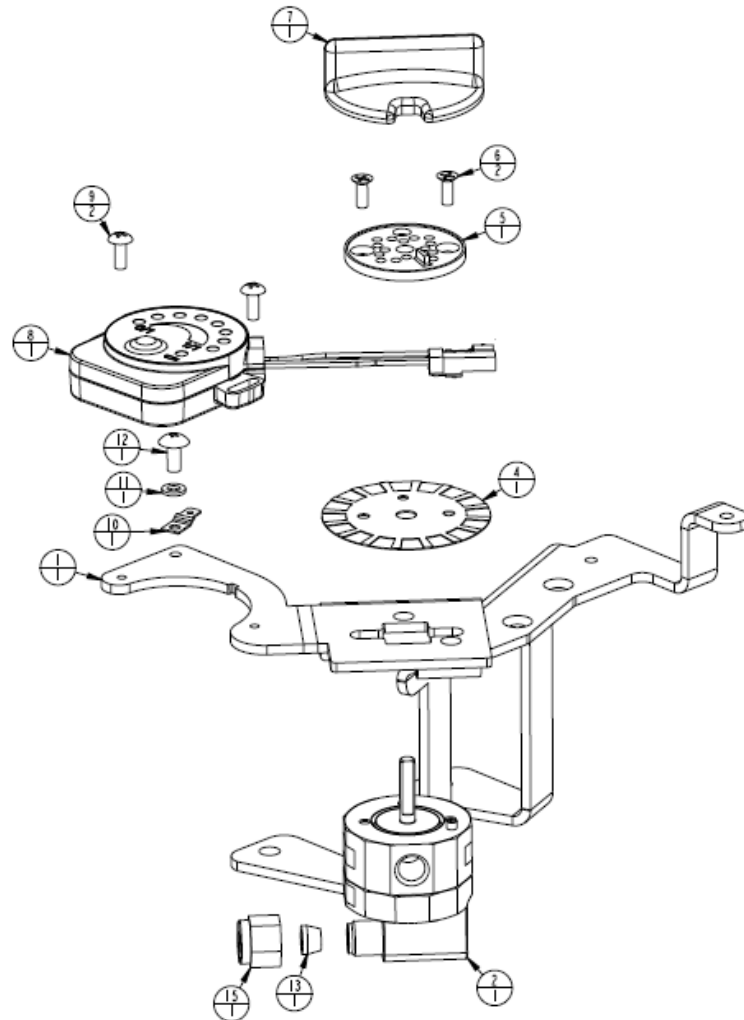

Bracket Assembly (Design after 4.1.2020)


G4 LIBERATOR® RESERVOIR




Economizer

Key	Description	Lib 20	Lib 30	Lib 37	Lib 45	Lib 60	#
1	Economizer Regulator (Design after 4.1.2020)	21341644*	21341644*	21341644*	21341644*	21341644*	1
	Economizer Regulator (Design before 4.1.2020)	10531833LAN*	10531833LAN*	10531833LAN*	10531833LAN*	10531833LAN*	
2	23 PSI PRV	10585882	10585882	10585882	10585882	10585882	1
3	Ferrule 1/4	B-775063-00	B-775063-00	B-775063-00	B-775063-00	B-775063-00	3
4	30 PSI Secondary Relief Valve (Design before 4.1.2020)	13110217*	13110217*	13110217*	13110217*	13110217*	
5	Tube Nut 1/4	B-775265-00	B-775265-00	B-775265-00	B-775265-00	B-775265-00	3

*Reference Service Bulletin 21495031 if replacing the Flow Control Valve, Secondary Relief Valve, or Economizer Valve

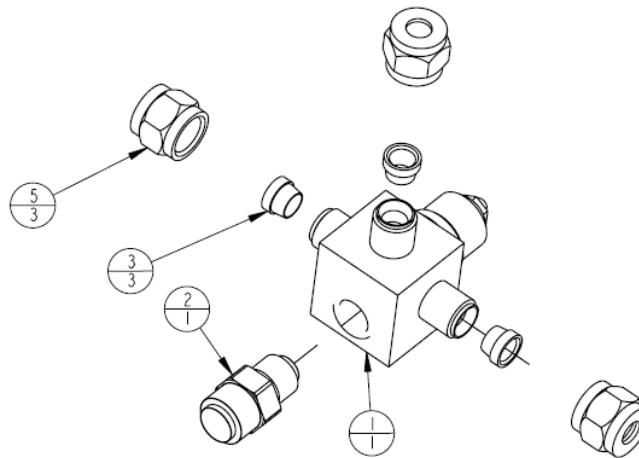

Economizer (Design before 4.1.2020)

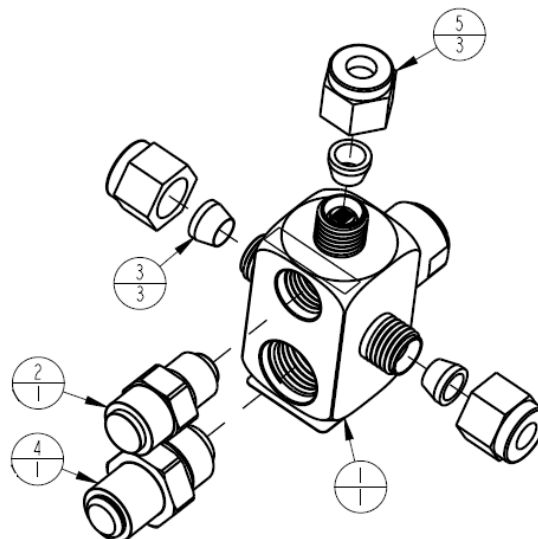


Economizer (Design after 4.1.2020)

G4 LIBERATOR® RESERVOIR



Pop-Off Assembly

Key	Description	Lib 20	Lib 30	Lib 37	Lib 45	Lib 60	#
1	Pop-Off Assembly	11908049	11908049	11908049	11908049	11908049	1
3	Pop-Off Sleeve	10576441	10576441	10576441	10576441	10576441	1
9	Pop-Off Spring	10931352	10931352	10931352	10931352	10931352	1

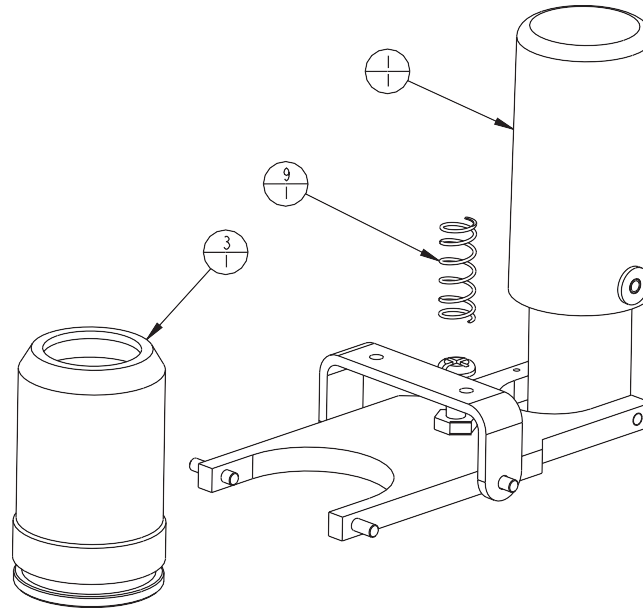

Copyright © 2020 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.