

Figure 1: Focus Housing

Item Number	Description	Quantity	Bubble Seq No
CA303-1S	HOUSING, UPPER	1	1
CA304-1S	HOUSING, LOWER	1	2
SC110-1	SCREW, HOUSING	2	3
LA417-1	LABEL, WARNING	1	4

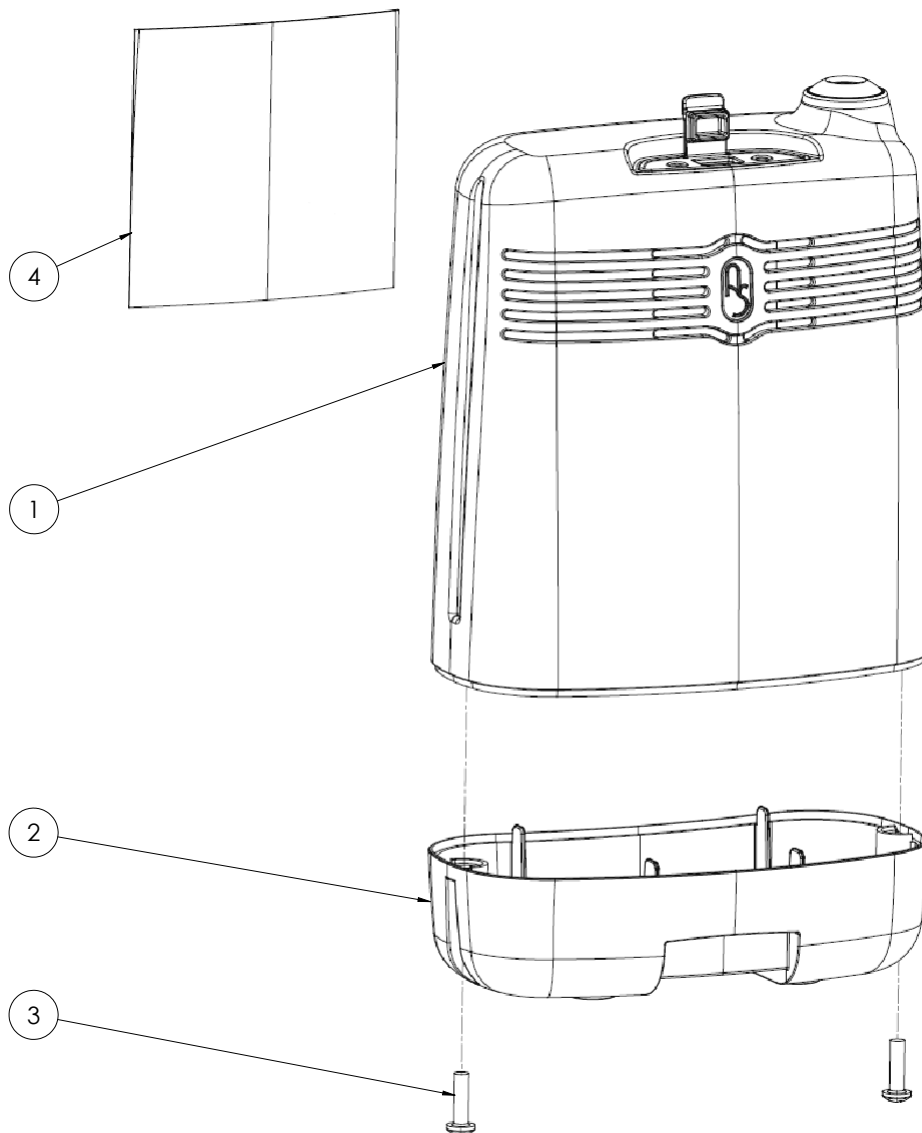


Figure 1: Focus Housing

Figure 1: Focus Circuit Board

Item Number	Description	Quantity	Bubble Seq No
CB123-1S	CIRCUIT BOARD ASSY (Includes # 1-7)	1	
CB162-1	CIRCUIT BOARD	1	1
FN051-1	FAN	1	2
CA330-1	EXHAUST DUCT	1	3
FO161-1	FOAM, EXHAUST DUCT	1	4
SC129-1	SCREW, EXHAUST DUCT	2	5
WA058-1	WASHER, EXHAUST DUCT	2	6
NU042-1	NUT, EXHAUST DUCT	2	7
SC090-3	SCREW, CIRCUIT BOARD	4	8

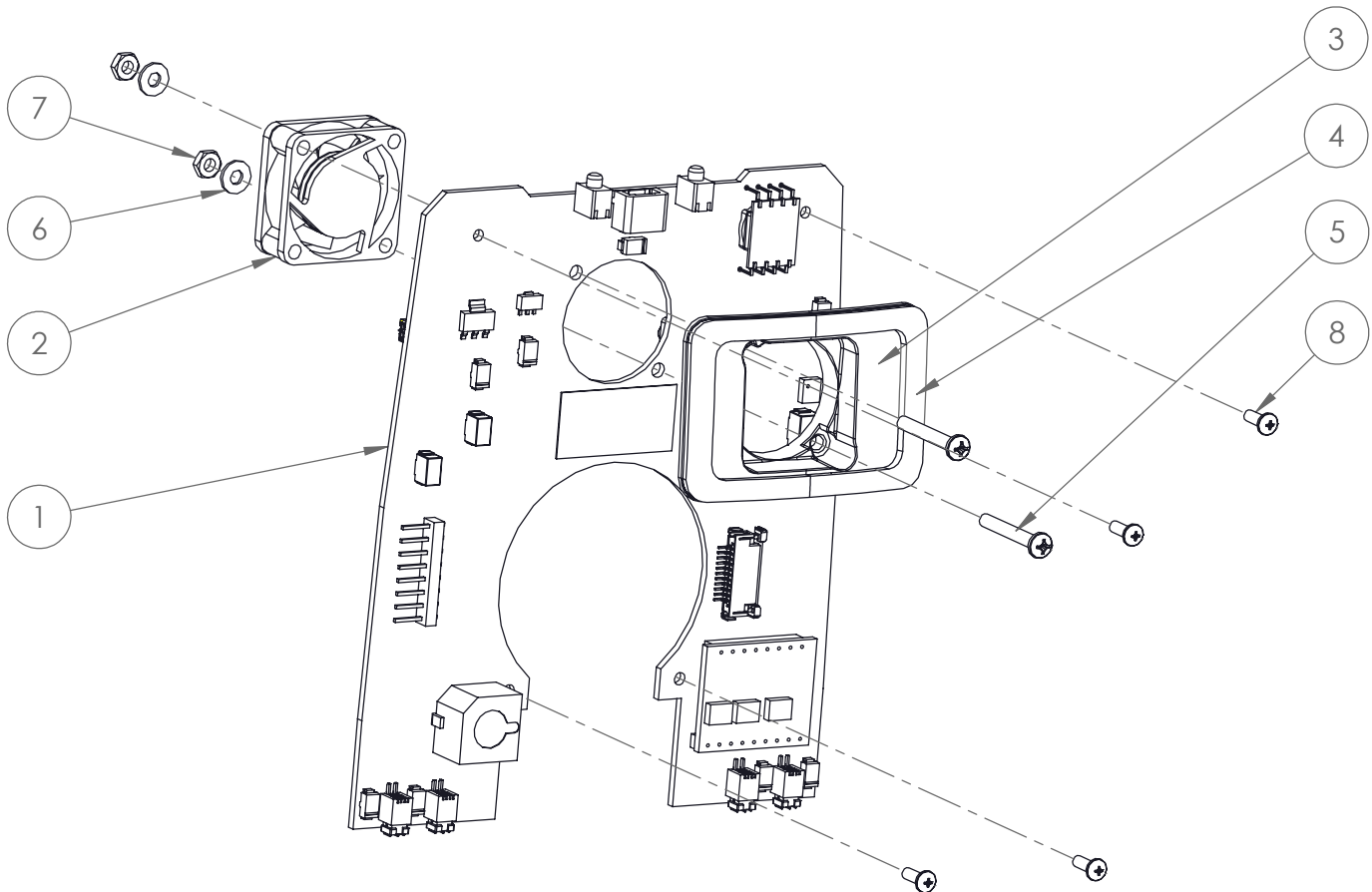


Figure 2: Focus Circuit Board

Figure 3: Focus PSA System

Item Number	Description	Quantity	Bubble Seq No
MA231-1S	MANIFOLD ASSY, PRODUCT (Includes # 2-5, 14)	1	1
VA557-2	VALVE, EQ/OCD (Includes Screws)	2	2
F0732-1S	FITTING, O2 OUTLET	1	3
GS027-1	GASKET, O2 OUTLET	1	4
TU196-1S	TUBING ASSY	1	5
S0317-1S	COMPRESSOR/FEED/WASTE MANIFOLD ASSY (Includes # 7-14)	1	6
MA213-1S	MANIFOLD ASSY, FEED/WASTE (Includes # 8)	1	7
VA557-1	VALVE, FEED/WASTE (Includes Screws)	4	8
OR025-2	O-RING, COMPRESSOR SEAL	1	9
SC103-2	SCREW, COMPRESSOR	4	10
CO383-1S	COMPRESSOR ASSY (Includes # 12)	1	11
FI171-1	FILTER, COMPRESSOR INLET	1	12
TU055-022S	TUBING, 1/4" OD, COMPRESSOR	1	13
MO024-2	MOUNT, VIBRATION	4	14
BE209-1S	SIEVE BED SET (Includes # 16, 17)	1	15
OR027-1	O-RING, 3.5MM ID, 6.5MM OD	2	16
OR027-2	O-RING, 6.0MM ID, 9.0MM OD	2	17
TA177-1S	TANK ASSY, PRODUCT	1	18
BK193-1	BRACKET, PRODUCT TANK	1	19
MI269-1	NEOPRENE TAPE, PRODUCT TANK	1	20
SC090-4	SCREW, PRODUCT TANK	1	21
SC105-3	SCREW, PSA SYSTEM	1	22
MI318-1	SPACER, PSA SYSTEM	1	23

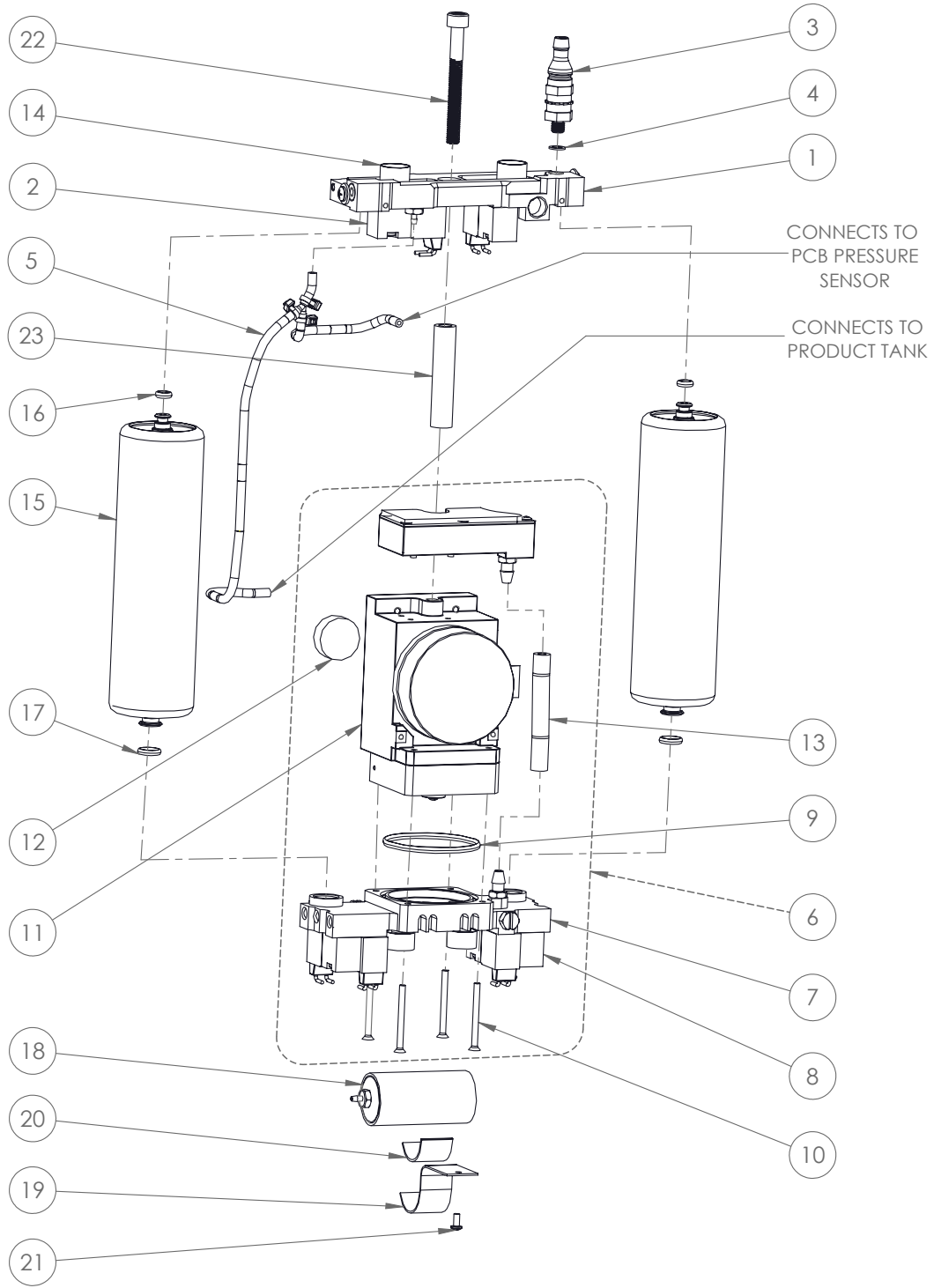


Figure 3: Focus PSA System