

QUICK START GUIDE

HOMESTYLE™

THIS QUICK START GUIDE IS NOT INTENDED TO REPLACE THE DETAILED INFORMATION AVAILABLE IN YOUR USER MANUAL. PLEASE REFER TO THE USER MANUAL FOR IMPORTANT CRITICAL INFORMATION REGARDING USE AND OPERATION OF THE PRODUCT.

1 Unpack your HOMESTYLE™ Aspen or Cortina*



User Manual



Additional Filter



User Manual



Additional Filter



2 Getting to know your HOMESTYLE™

Review all warnings, cautions and additional device information in your User Manual. Become familiar with the key features of the HOMESTYLE and the User Interface.

FRONT



BACK



1 Power Switch

ON/OFF power switch used to initiate or stop power supply to the unit. The ON position is indicated by the "I" symbol, and the OFF position is indicated by the "O" symbol.

2 Hour Meter

Digital display of the elapsed operation time of the concentrator. Cannot be reset.

3 Flow Meter

Used to adjust and display the flow rate of oxygen delivered. Flow rates range from 0.5 - 5 liters per minute (LPM) and can be adjusted by turning the knob.

4 Outlet Barb

Oxygen exits here. Either a single lumen nasal cannula or standard oxygen tubing is attached to deliver to the patient. The maximum length of oxygen tubing attached to the concentrator is 36 ft. (11 m).

5 Filter Compartment

Accessible filter compartment allows for easy access to filter for cleaning or replacement.

6 Humidifier Bottle

The stand contains an elastic band used to secure the humidifier bottle. Consult your healthcare provider to see if humidification is right for you.

7 Power Cord

Cord plugs into outlet to power device.

Buzzer: An audible alarm (or buzzer) alerts you to the operating condition of the device, either a warning or failure, and confirms a valid key press.

On steps 2 through 6, photos of the HOMESTYLE Aspen Stationary Oxygen Concentrator are shown.

3 Set Up

If using a humidifier bottle, first fill the bottle with distilled water to the manufacturer's specified level. Place bottle in designated support stand and secure with the elastic band. Connect the threaded end of the humidifier tube to the bottle, and the other to the HOMESTYLE outlet barb. Connect oxygen tubing and/or a nasal cannula to the outlet barb on the humidifier bottle.



If not using a humidifier bottle, connect the oxygen tubing and/or a nasal cannula directly to the HOMESTYLE outlet barb.




4 Insert Filter to Unit

Open the compartment on the back of the HOMESTYLE and insert your air filter. Once inserted, close the compartment.



5 Power On and Warm Up

Plug the AC power cord into a wall (AC) outlet.

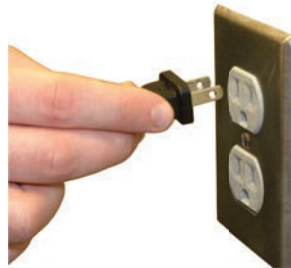

Toggle the Power Switch to the On position.



When the HOMESTYLE is powered on properly, the hour counter will remain on.

Note: After initially powering ON the device, please allow up to ten (10) minutes for the device to reach its performance specifications.

The power cord shown is for illustration only; included plugs will match the standard for your country.

6 Adjust Flow Setting to Prescribed Level

Use the Flow Control Knob to select the flow prescribed by your physician.



WARNING: IT IS VERY IMPORTANT TO SELECT ONLY THE PRESCRIBED LEVEL OF OXYGEN. DO NOT CHANGE THE FLOW SELECTION UNLESS YOU HAVE BEEN DIRECTED TO DO SO BY A LICENSED CLINICIAN.

Breathe normally through the nasal cannula.

To adjust flow rate: Turn counter-clockwise to increase flow.

Turn clockwise to decrease flow.

The middle of the ball indicates flow rate. See picture below.

*Model colors may vary.

© Copyright 2026 CAIRE Inc. All Rights Reserved. SEE PRODUCT WARRANTY STATEMENT FOR COMPLETE INFORMATION. Please consult the applicable product instructions for use for product indications, contraindications, warnings, precautions, and detailed safety information. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.