

QUICK START GUIDE

VisionAire® 5

THIS QUICK START GUIDE IS NOT INTENDED TO REPLACE THE DETAILED INFORMATION AVAILABLE IN YOUR USER MANUAL. PLEASE REFER TO THE USER MANUAL FOR IMPORTANT CRITICAL INFORMATION REGARDING USE AND OPERATION OF THE PRODUCT.

1 Unpack your VisionAire® 5



User Manual



2 Getting to know your VisionAire 5

Review all warnings, cautions and additional device information in your User Manual. Become familiar with the key features of the VisionAire 5 and the User Interface.

FRONT



BACK



1 Hour Meter	Digital display of the elapsed operation time of the concentrator. Cannot be reset.
2 Power Switch	ON/OFF power switch used to initiate or stop power supply to the unit. The ON position is indicated by the "I" symbol, and the OFF position is indicated by the "O" symbol.
3 Outlet Barb	Oxygen exits here. Either a single lumen nasal cannula or standard oxygen tubing is attached to deliver to the patient. The maximum length of oxygen tubing attached to the concentrator is 50 ft. (15.2 m).
4 Flow Meter	Used to adjust and display the flow rate of oxygen delivered. Flow rates range from 0.5 - 5 liters per minute (LPM) and can be adjusted by turning the knob.
5 Filter Compartments	Accessible filter compartments in the handles allow for easy removal and cleaning of optional filters.
6 Humidifier Bottle	The stand contains an elastic band used to secure the humidifier bottle. Consult your healthcare provider to see if humidification is right for you.
7 Power Cord	Cord plugs into outlet to power device.

Buzzer: An audible alarm (or buzzer) alerts you to the operating condition of the device, either a warning or failure, and confirms a valid key press.

3 Set Up

If using a humidifier bottle, first fill the bottle with distilled water to the manufacturer's specified level. Place bottle in designated support stand and secure with the elastic band. Connect the threaded end of the humidifier tube to the bottle, and the other to the VisionAire 5 outlet barb. Connect oxygen tubing and/or a nasal cannula to the outlet barb on the humidifier bottle.



If not using a humidifier bottle, connect the oxygen tubing and/or a nasal cannula directly to the VisionAire 5 outlet barb.



4 Insert Optional Filters to Unit

Insert optional air filters into the handles of each side of the VisionAire 5.



5 Power On and Warm Up

Plug the AC power cord into a wall (AC) outlet.



Toggle the Power Switch to the On position.



When the VisionAire 5 is powered on properly, the hour counter will remain on.

Note: After initially powering ON the device, please allow up to ten (10) minutes for the device to reach its performance specifications.

The power cord shown is for illustration only; included plugs will match the standard for your country.

6 Adjust Flow Setting to Prescribed Level

Use the Flow Control Knob to select the flow prescribed by your physician.



WARNING: IT IS VERY IMPORTANT TO SELECT ONLY THE PRESCRIBED LEVEL OF OXYGEN. DO NOT CHANGE THE FLOW SELECTION UNLESS YOU HAVE BEEN DIRECTED TO DO SO BY A LICENSED CLINICIAN.

Breathe normally through the nasal cannula.

To adjust flow rate: Turn counter-clockwise to increase flow.

Turn clockwise to decrease flow.

The middle of the ball indicates flow rate. See picture below.