
AirSep® NewLife® Concentrator Air Outlet Installation

Parts Included:

1. Plastic 90 degree air outlet valve (quantity=1)
2. Washer 0.875" OD x 0.562" ID x 0.050" thick (quantity=4)
3. Fitting assembly (from inside of concentrator attached to black tube) (quantity=1)
4. Nylon adapter (not shown) (quantity=1)
5. Tie wrap (not shown) (quantity=1)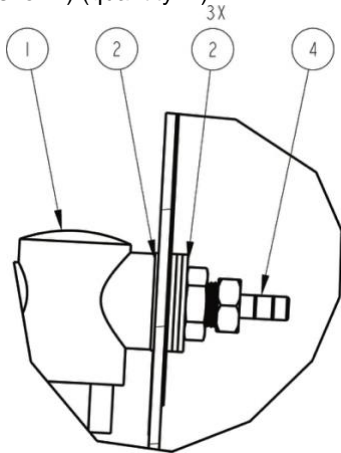

Procedure:

1. Remove fitting assembly from black tube inside concentrator
2. Per the above figure, place three washers (item 2) onto fitting assembly.
3. Install assembly through 9/16" hole located in control panel.
4. Place one washer (item 2) onto fitting on front of control panel.
5. Install 90 degree ball valve (item 1) on fitting assembly. Be careful not to cross thread.
6. Rotate ball valve until snug tight.
7. Brace ball valve while tightening fitting assembly hex with open ended wrench.
8. Install black tube onto barb of fitting assembly. Secure with tie wrap.
9. Install nylon adapter into ball valve threads.

Copyright © 2018 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

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Copyright © 2018 CAIRE Inc. CAIRE Inc. si riserva il diritto di interrompere la produzione o di modificare prezzi, materiali, apparecchiature, qualità, descrizioni, specifiche e/o processi dei propri prodotti in qualsiasi momento, senza preavviso e senza ulteriori obblighi o conseguenze. Tutti i diritti non espressamente indicati nel presente documento sono riservati, laddove applicabili.
Copyright © 2018 CAIRE Inc. CAIRE Inc. forbeholder seg retten til å stanse sine produkter, eller endre priser, materialer, utstyr, kvalitet, beskrivelser, spesifikasjoner og/eller prosesser for sine produkter når som helst, uten forvarsel og uten videre forpliktelser eller konsekvenser. Alle rettigheter som ikke er uttrykkelig nevnt her, er forbeholdt oss, som anvendelig.
Copyright © 2018 CAIRE Inc. CAIRE Inc. förbehåller sig rätten att när som helst utan förvarning upphöra med försäljning av sina produkter eller ändra produkternas priser, material, utrustning, kvalitet, beskrivningar, specifikationer och/eller prosesser utan vidare förpliktelser eller följder. Alla rättigheter som inte uttryckligen anges här är reserverade av oss, enligt tillämplighet.