



Med Tips



February, 2010

HELIOS Carrying Cases

The HELIOS 300 can be carried in a convenient belt pack. The HELIOS 850 can be carried in a convertible backpack/shoulder strap.

Part Number	Description
B-701654-00	BAG, HIP PACK, H-300
069209	HELIOS BACKPACK H850



Condensate Bottles for Reservoirs

The condensate bottle for the HELIOS reservoir is internal while the Liberator and Companion reservoirs have external bottles. In the past, the Companion reservoir used brown cylindrical containers. These are not available and have been replaced by flat drain bottle. Please use the part numbers below.



HELIOS Standard and Universal Reservoirs

Part Number	Description
B-701546-00	MOISTURE CONTAINER



Companion Reservoir

Part Number	Description
B-701320-00	BOTTLE, DRAIN, FLAT
B-701360-00	BRACKET, FLAT DRAIN BOTTLE
B-701359-00	ELBOW, FLAT DRAIN BOTTLE



Liberator Reservoir

Part Number	Description
CA404899	CONDENSATE BOTTLE
10584513	BRACKET CONDENSATE BOTTLE

Liquid Oxygen Use Times for On-Demand Portables

Caire offers three portable devices that deliver oxygen based on inspiration, or on-demand. These are listed below along with the approximate use times. Please be advised, these are estimates and will vary from patient to patient based on breathing rate

Spirit

	Off	1.0	1.5	2.0	3.0	4.0	5.0	CF
Spirit 300	16:00	14:30	10:00	7:00	4:40	3:30	3:00	2:10
Spirit 600	30:00	27:00	18:40	13:00	8:40	6:30	5:30	4:00
Spirit 1200	60:00	54:00	37:10	28:00	17:20	13:00	11:10	8:00

HELIOS 300

Flow	Use Time
Off	-
0.12	15:00
0.25	15:00
0.5	10:00
0.75	6:30
1	13:00
1.5	12:00
2	10:00
2.5	8:30
3	6:30
3.5	6:00
4	5:00

HELIOS 850

Flow	Use Time
Continuous Flow	
C1	10:30
C2	5:30
C3	3:30
C4	2:30
C5	2:00
C6	1:30
Demand Flow	
D1.5	22:30
D2	18:30
D2.5	16:00
D3	11:30
D4	9:30

Note: Above Times are in hours and minutes format (00:00)

Customer & Technical Info

For ordering information contact

Customer Service:

Europe: +44 1344 403100
Australia/New Zealand: +61 2 9749 4333
US Toll Free: 800.482.2473
Asia/ Pac Rim: 770.721.7759
The Americas: 770.721.7759

For technical information contact

Technical Service:

Europe:
 Phone: +44.1344.403100
 Email: technicalservice.europe@chart-ind.com
US Toll Free
 Phone: 800.482.2473
 Email: techservice.usa@chart-ind.com
Asia/Pac Rim
 Phone: 770.721.7759
 Email: techservice.usa@chart-ind.com

www.cairemedical.com