



Med Tips



July & August, 2010

PORTABLE PACK COMPATIBILITY

Below is a list of CAIRE portables and the carrying packs with which they are compatible.



C1000 / C1000T	<ul style="list-style-type: none"> • C1000/C1000T Backpack: 10018138 • Airlift Backpack: 13214825
C550	<ul style="list-style-type: none"> • C550 Hip Pack: B-778625-00
H300	<ul style="list-style-type: none"> • H300 Hip Pack: B-701654-00 • Burgundy Hip Pack w/ Lincare Logo: 10003480
H850	<ul style="list-style-type: none"> • H850 Backpack: 069209 • C1000/C1000T Backpack: 10018138 • Airlift Backpack: 13214825
Spirit 300	<ul style="list-style-type: none"> • Belt Pack: 11843361
Spirit 600	<ul style="list-style-type: none"> • Airlift Backpack: 13214825
Spirit 1200	<ul style="list-style-type: none"> • Airlift Backpack: 13214825
Sprint	<ul style="list-style-type: none"> • Airlift Backpack: 13214825
Stroller / Hi Flow	<ul style="list-style-type: none"> • C1000/C1000T Backpack: 10018138 • Airlift Backpack: 13214825

CLEANING CAIRE UNITS

When cleaning and disinfecting any CAIRE unit, it is highly recommended that you use only CAIRE approved products and solutions, as failure to do so may result in damage to the unit. Below is a list of approved cleaning substances.

Companion & HELiOS Units

Function	Acceptable Products
Cleaning	<ul style="list-style-type: none"> • Sporicidin Disinfectant Solution from Sporicidin International • Mild dish washing detergent / warm water solution
Disinfecting	<ul style="list-style-type: none"> • Sporicidin Disinfectant Solution • Household bleach diluted 1:10 with water, freshly made within 24 hours
Leak Detecting	<ul style="list-style-type: none"> • Snoop Liquid Leak Detector from Swagelok Co.

Liberator, Stroller, Sprint, & Spirit Units

Function	Acceptable Products
Cleaning	<ul style="list-style-type: none"> • Household glass cleaner • Mild soap solution
Disinfecting	<ul style="list-style-type: none"> • Microbac Forte from Bode Chemie Hamburg • Wex-Cide from Wexford Labs, Inc • Vesphene Ilse from Steris Corp
Leak Detecting	<ul style="list-style-type: none"> • Snoop Liquid Leak Detector from Swagelok Co.

Customer & Technical Info

For ordering information contact

Customer Service:

Europe: +44 1344 403100
Australia/New Zealand: +61 2 9749 4333
US Toll Free: 800.482.2473
Asia/Pac Rim: 770.721.7759
The Americas: 770.721.7759

For technical information contact

Technical Service:

Europe:
 Phone: +441344 403100
 Email: technicalservice.europe@chart-ind.com
US Toll Free
 Phone: 800.482.2473
 Email: techservice.usa@chart-ind.com
Asia/Pac Rim
 Phone: 770.721.7759
 Email: techservice.usa@chart-ind.com