August-September-October 2024 Med Tips

Don't miss out on our upcoming Service Schools!

CAIRE Service School Trainings and Webinars 2024

CAIRE offers In-person and Virtual Service Schools covering both LOX and Concentrator lines.

Each class is a comprehensive program that focuses on the technical and service aspects of the CAIRE family products. Class time is divided between lecture and hands-on workshops. The seminars will help the student develop a better understanding of how liquid oxygen equipment or concentrators work, how to identify the symptoms and causes of potential problems and how to use the technical information that is available in the Technical Manuals. Registration for CAIRE’s Service Schools is mandatory. Registration forms must be received two weeks prior to the start of the class in order to guarantee availability and materials.

Table 1 - Details of upcoming training.

Type	Language	Location	Training Dates	Registration by	Training Type
Conc USA	English	Ball Ground, GA USA	04-06 Nov-24	11-October	In person
LOX USA	English	Ball Ground, GA USA	07 Nov-24	11-October	In person
LOX Germany	German	Langenfeld, Germany	28 November-24	08-November	In person

*We reserve the right to cancel training seminars at short notice. Customers should ensure that they take necessary precautions to ensure costs are covered in case of cancellation.

** Training will be in the form of a Webinar.

Concentrator Service School Topics include:

- Concentrator hazards and safety precautions
- Principles of pressure, flow, and saturation
- Functions of the major components of a portable/stationary concentrator
- Theory of operation
- Set up and use of test equipment
- Troubleshooting, repair and performance verification/testing procedures

LOX Service School Topics include:

- Liquid oxygen (LOX) hazards and safety precautions
- Principles of pressure, flow and liquid oxygen saturation
- Functions of the major components of a liquid oxygen system
- HELIOS™ system theory of operation
- Reservoir and portable filling procedures
- Set up and use of test equipment

Please submit your registration forms no later than the dates in Table 1 above. If you are interested in the service schools mentioned above, and it is past the registration date, please reach out to the registration emails below. Please note that as travel restrictions, and Covid-related guidelines evolve, we intend to restore CAIRE Service School to in person.

To register for Service School, please email:

Techservice.global@caireinc.com for USA Training or jim.gibson@caireinc.com for European Training.

PRODUCT UPDATES

FreeStyle Comfort™ Software V 2.9.1 Update

CAIRE is releasing a software update to all FreeStyle Comfort models.

Version 2.9.1 firmware changes the audible signal for 'No Breath Detected' and 'Low Concentration' from activating every 30 seconds to activating every 10 minutes.

This change also discontinues the 'Breath Rate Exceeded' alarm from repeating every 60 seconds.

Additionally, this update changes the audible signal for 'Low Battery Warning' from activating at 10% battery level to 20% battery level while showing the below display.

There are no changes to the overall function or usage of the concentrator, but the new software changes the alarm priorities.

Note: Devices in the field do not require a mandatory firmware update

Previous Version	Updated Version
2.9.0 or earlier	2.9.1

Please refer to Service Bulletin 21499071 for more details on alarm priorities.

Companion™ 5 Humidifier Adapter

Due to a supplier obsolescence, the Companion 5 humidifier adapter, part number 20843882, will be changing in appearance. There will no longer be a 90 degree elbow connection to the humidifier bottle as shown in figures 1 and 2 below.

This will be a running change with no start date, and the previous version will be available until stock is depleted.

Fig 1. Previous version

Fig 2. Updated Version

Please refer to Service Bulletin 21499884.

CAIRE Liquid Oxygen Product Line Rationalization

Effective November 15, 2024, CAIRE will be implementing a rationalization of our Liquid Oxygen product line. This process involves consolidating our existing offerings into a more streamlined selection that meets all customer needs effectively.

As part of this rationalization, our base unit range will be consolidated into a single Dual Fill configuration. This unified version will integrate both Top Fill and Side Fill options, providing enhanced flexibility by including both as standard features in all our base units. For our portable units, the goal is to simplify the product range while still offering a comprehensive array of options that deliver the best clinical outcomes for ambulatory patients.

CAIRE will continue to manufacture and stock a variety of CAIRE Liquid Oxygen models. Please contact Customer Service, or Technical Service for assistance in determining which replacement model(s) will best fulfill your requirements.

CAIRE Liquid Oxygen SKUs - Discontinued			CAIRE Liquid Oxygen SKUs - Replaced By	
Product Line	Item No.	Description	Item No.	Description
Liberators	14800332	FNL LIB 20T DF CP M 15LPM 20	13256195	FNL LIB 20T DF PB M 15LPM 20
Liberators	13256435	FNL LIB 20T SF CA F 15LPM 20	13256195	FNL LIB 20T DF PB M 15LPM 20
Liberators	13256654	FNL LIB 20T TF PB M 15LPM 20	13256195	FNL LIB 20T DF PB M 15LPM 20
Liberators	13258449	FNL LIB 30T SF CA F 15LPM 20	14454960	FNL LIB 30T DF CP M 15LPM 20
Liberators	13258537	FNL LIB 30T SF CA F 15LPM 20	14454960	FNL LIB 30T DF CP M 15LPM 20
Liberators	13258641	FNL LIB 30T TF PB M 15LPM 20	13256929	FNL LIB 30T DF PB M 15LPM 20
Liberators	13259919	FNL LIB 37T TF PB M 15LPM 20	13259599	FNL LIB 37T DF PB M 15LPM 20
Liberators	13262077	FNL LIB 45T SF CA F 15LPM 20	14454986	FNL LIB 45T DF CP M 15LPM 20
Liberators	13262085	FNL LIB 45T SF CA F 15LPM 20	14454986	FNL LIB 45T DF CP M 15LPM 20
Liberators	13262253	FNL LIB 45T TF PB M 15LPM 20	13261728	FNL LIB 45T DF PB M 15LPM 20
Liberators	20651170	FNL LIB 45T TF PB M 2LPM 20PSI	13261728	FNL LIB 45T DF PB M 15LPM 20
Liberators	13263723	FNL LIB 60T DF PB M 15LPM 20	13263758	FNL LIB 60T DF PB M 15LPM 20
Liberators	13263811	FNL LIB 60T TF PB M 15LPM 20	13263758	FNL LIB 60T DF PB M 15LPM 20
Sprint/Strollers	20663674	FNL HI FLOW T TF PB F 15LPM G4	14768299	FNL HI FLOW T TF PB F 15LPM G4

Additional Models and information about country registration is available in the Service Bulletin 21499554.

SAROS 4000™ Air Worthiness Rated (AWR)

CAIRE is advising customers of our new compliance with RTCA-DO-160G Section 8 Categories S and U for SAROS 4000 concentrator.

SAROS 4000 units manufactured after 08/26/2024 or starting with serial number CBB082432XXXX are airworthy rated for both fixed wing and rotary wing applications.

For fixed wing aircraft, a battery mounted to the Saros can be charged and discharged.

For rotary wing aircraft, battery usage is application-dependent and not recommended by CAIRE. The changes do not impact existing regulatory clearances (such as 510K or CE requirements).

See table below for Saros 4000 service part changes.

Saros 4000	Saros 4000 AWR	DESCRIPTION
SP9710V2-SEQ	SP9710V2-SEQ	Circuit Board Assembly
9767-1-SEQ	9767-1-SEQ	Compressor Fan
SP9240-SEQ	SP9240V2-SEQ	Compressor & Product Tank Assembly
SP9238-SEQ	SP9238V2-SEQ	Compressor Assembly
SP9729-SEQ	SP9729-SEQ	ATF Module Assembly
1405-SEQ	AD001-4	Loctite Threadlocker

Please refer to Service Bulletin 21499364.

Helpful Hints & FAQs

NewLife Intensity 10 - Pressure Outlet Test Procedure

The product regulator is factory-set at 20 PSIG (138 kPa) for the Intensity 10 and should not require adjustment. To check for proper adjustment of the product regulator, take the following steps:

1. Install outlet pressure test kit (TU131-2) to the outlet of the NewLife Intensity 10.
2. Set the unit's ON/OFF power switch to the ON position.
3. Allow the unit to run for five minutes.
4. Turn the flowmeter adjustment knob counterclockwise until it stops (wide open).
5. The outlet pressure should measure 20 PSIG for the Intensity 10. If the outlet pressure is not at 20 PSIG the product regulator needs to be adjusted.

Accessories Catalog

Our Accessories catalog can be downloaded from the website for our Concentrators and LOX products, including filters, service tools, transfilling and transport accessories.

<https://www.caireinc.com/providers/business-support-programs/knowledge-center/>

Document Update

We advise our valued customers to monitor and check that they are using the most recent revision of any documents for the model of equipment they have. Documents released/updated during August to October 2024.

- SureFlow Cutsheet ML-CONC0001 D
- FreeStyle Comfort Technical Manual MN233 Rev J
- FreeStyle Comfort 4 button Technical Manual MN242 Rev D

Med Tips Distribution!

The CAIRE team is asking all current subscribers to CAIRE's Med Tips Newsletter to register in order to continue receiving this monthly communication.

Please click this link - <http://go.pardot.com/l/710913/2022-01-17/2pgcj9> - and provide us with your current information. Thank you and we appreciate your support.

CONTACT:

For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

techservice.global@caireinc.com

Europe: +44 (0) 118 936 7060

techservice.global@caireinc.com

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

customerservice.europe@caireinc.com

© Copyright 2024 CAIRE Inc. All Rights Reserved. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

Please consult the applicable product instructions for use for product indications, contraindications, warnings, precautions, and detailed safety information.

PN 21499071

RELEASE DATE: May 1, 2024

MODEL: FreeStyle Comfort™

ISSUE: FreeStyle Comfort™ Software V 2.9.1 Update

NOTES: CAIRE is releasing a software update to all FreeStyle Comfort models. Version 2.9.1 firmware changes the audible signal for 'No Breath Detected' and 'Low Concentration' from activating every 30 seconds to activating every 10 minutes.

This change also discontinues the 'Breath Rate Exceeded' alarm from repeating every 60 seconds.

Additionally, this update changes the audible signal for 'Low Battery Warning' from activating at 10% battery level to 20% battery level while showing the below display.

There are no changes to the overall function or usage of the concentrator, but the new software changes the alarm priorities.

Please refer below for the new Alarm Conditions Table listing the changes to alarms priority.

Priority #	Alarm Description	Audible Signal
1	System Malfunction Battery High Temperature	1 beep every 20 sec
2	System Malfunction High Temperature	1 beep every 20 sec
3	System Malfunction High Pressure (High System Pressure or Compressor Fault)	1 beep every 20 sec
4	System Malfunction Flow Rate (OCD Valve Failure, Obstruction of Gas Pathway)	1 beep every 30 sec
5	Low Battery Shut Down	1 beep every 30 sec
6	No Breath Detected	1 beep every 10 min
7	Warm Up	none
8	Low Concentration	1 beep every 10 min, for up to 30 min
9	Low Battery Warning	1 beep, no repeat
10	Breathing Rate Exceeded	1 beep, no repeat
11	System Malfunction Fan Failure	none
12	System Malfunction Low Pressure	none
13	System Malfunction Low Temp	none

Note: Devices in the field do not require a mandatory firmware update.

Previous Version	Updated Version
2.9.0 or earlier	2.9.1

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

Europe: +44 (0) 118 936 7060

techservice.global@caireinc.com

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

customerservice.europe@caireinc.com

Copyright © 2024 CAIRE Inc.

CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

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RELEASE DATE: August 6, 2024

MODEL: Companion 5™

ISSUE: Companion 5 Humidifier Adapter

NOTES: Due to a supplier obsolescence, the Companion 5 humidifier adapter, part number 20843882, will be changing in appearance. There will no longer be a 90 degree elbow connection to the humidifier bottle as shown in figures 1 and 2 below.

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Europe: +44 (0) 118 936 7060

techservice.global@caireinc.com

For ordering information or general inquiries, contact Customer Service:

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REVISION DATE: October 1, 2024 (Rev B)

MODEL: CAIRE Liquid Oxygen Models: Liberator / Sprint / Stroller

ISSUE: CAIRE Liquid Oxygen Product Line Rationalization

NOTES: Effective November 15, 2024, CAIRE will be implementing a rationalization of our Liquid Oxygen product line. This process involves consolidating our existing offerings into a more streamlined selection that meets all customer needs effectively.

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These changes are designed to streamline the ordering and delivery process for our customers, while enhancing our overall product offering.

Please refer to the table below for details on the models being consolidated and the recommended replacements or alternatives. CAIRE's Regulatory department has proactively reviewed country registrations, but should you identify additional registration needs, please contact the Regulatory Requests mailbox:

caire.regulatoryrequests@caireinc.com

CAIRE will continue to provide service and parts for the consolidated models in accordance with our warranty obligations, up to 5 years from the date of sale, or as long as parts remain available. We will also continue to produce and stock the alternative models listed, ensuring the continuity of our product offerings and solutions.

CAIRE Liquid Oxygen SKUs - Discontinued		
Product Line	Item No.	Description
Liberators	14800332	FNL LIB 20T DF CP M 15LPM 20
Liberators	13256435	FNL LIB 20T SF CA F 15LPM 20
Liberators	13256654	FNL LIB 20T TF PB M 15LPM 20
Liberators	13258449	FNL LIB 30T SF CA F 15LPM 20
Liberators	13258537	FNL LIB 30T SF CA F 15LPM 20
Liberators	13258641	FNL LIB 30T TF PB M 15LPM 20
Liberators	13337403	FNL LIB 30T TF PB M 2LPM 20
Liberators	13259919	FNL LIB 37T TF PB M 15LPM 20
Liberators	13262077	FNL LIB 45T SF CA F 15LPM 20
Liberators	13262085	FNL LIB 45T SF CA F 15LPM 20
Liberators	13262253	FNL LIB 45T TF PB M 15LPM 20
Liberators	20651170	FNL LIB 45T TF PB M 2LPM 20PSI
Liberators	13263723	FNL LIB 60T DF PB M 15LPM 20
Liberators	13263811	FNL LIB 60T TF PB M 15LPM 20
HF Strollers	14089341	FNL HI FLOW T SF CA M 15LPM G4, METER GRY
HF Strollers	20663674	FNL HI FLOW T TF PB F 15LPM G4, SCALE BLK
HF Strollers	14081315	FNL HI FLOW T TF PB F 15LPM G4, METER GRY
Sprint	15103150	FNL SPRINT T SF CA M 6LPM
Sprint	11928429	FNL SPRINT T TF PB F 2LPM, METER
Sprint	14109269	FNL SPRINT T TF PB F 6LPM, SCALE
Sprint	20600823	FNL SPRINT T TF PB F 6LPM, SCALE DVLBS
Sprint	20662495	FNL SPRINT T TF PB F 6LPM, SCALE
Sprint	10935731	FNL SPRINT T TF PEN F 6LPM, METER
Strollers	11933113	FNL STROLLER T SF CA M 2LPM, METER
Strollers	10734067	FNL STROLLER T SF CA M 6LPM, METER
Strollers	14909784	FNL STROLLER T SF CA M 6LPM, SCALE
Strollers	14306748	FNL STROLLER T TF PB F 2LPM, METER GRY
Strollers	14306756	FNL STROLLER T TF PB F 2LPM, SCALE GRY
Strollers	11908161	FNL STROLLER T TF PB F 6LPM, SCALE
Strollers	14765549	FNL STROLLER T TF PB F 6LPM, SCALE BLK
Strollers	10935723	FNL STROLLER T TF PEN F 6LPM, METER
Strollers	11899808BZ	FNL STROLLER T TF TAE F 6LPM, METER

CAIRE Liquid Oxygen SKUs - Replaced By	
Item No.	Description
13256195	FNL LIB 20T DF PB M 15LPM 20
13256195	FNL LIB 20T DF PB M 15LPM 20
13256195	FNL LIB 20T DF PB M 15LPM 20
13256929	FNL LIB 30T DF PB M 15LPM 20
13256929	FNL LIB 30T DF PB M 15LPM 20
13256929	FNL LIB 30T DF PB M 15LPM 20
20606186	FNL LIB 30T DF PB M 2LPM
13259599	FNL LIB 37T DF PB M 15LPM 20
13261699	FNL LIB 45T DF PB M 15LPM 20
13261699	FNL LIB 45T DF PB M 15LPM 20
13261699	FNL LIB 45T DF PB M 15LPM 20
20606186	FNL LIB 30T DF PB M 2LPM
13263758	FNL LIB 60T DF PB M 15LPM 20
13263758	FNL LIB 60T DF PB M 15LPM 20
14778770	FNL HI FLOW T SF CA M 15LPM G4, METER BLK
14768299	FNL HI FLOW T TF PB F 15LPM G4, METER BLK
14768299	FNL HI FLOW T TF PB F 15LPM G4, METER BLK
10845218	FNL SPRINT T SF CA M 6LPM, METER
13259433	FNL STROLLER T TF PB F 2LPM, METER
10956671	FNL SPRINT T TF PB F 6LPM, METER
10956671	FNL SPRINT T TF PB F 6LPM, METER
10956671	FNL SPRINT T TF PB F 6LPM, METER
10956671	FNL SPRINT T TF PB F 6LPM, METER
13259433	FNL STROLLER T TF PB F 2LPM, METER
10845200	FNL STROLLER T SF CA M 6LPM, METER
10845200	FNL STROLLER T SF CA M 6LPM, METER
13259433	FNL STROLLER T TF PB F 2LPM, METER
13259433	FNL STROLLER T TF PB F 2LPM, METER
10956680	FNL STROLLER T TF PB F 6LPM, METER
10956680	FNL STROLLER T TF PB F 6LPM, METER
10956680	FNL STROLLER T TF PB F 6LPM, METER
10956680	FNL STROLLER T TF PB F 6LPM, METER
11730957BZ	FNL STROLLER T TF PB F 6LPM, METER

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Additionally, a 'Build to Order' (BTO) option will be available for more customized versions, as detailed below.

CAIRE Liquid Oxygen SKUs - Build to Order (BTO)			
Product Line	Item No.	Description	Lead Times (Days)
Helios	B-702035-00	FNL H300C TF PB F 4LPM 20, JAPANESE	Quoted at Time of Order
Helios	B-702034-00	FNL H36C TF PB M 10LPM 20, JAPANESE	Quoted at Time of Order
Helios	B-702033-00	FNL H46C TF PB M 10LPM 20, JAPANESE	Quoted at Time of Order
Liberators	13256195	FNL LIB 20T DF PB M 15LPM 20	Quoted at Time of Order
Liberators	14454960	FNL LIB 30T DF CP M 15LPM 20	Quoted at Time of Order
Liberators	20606186	FNL LIB 30T DF PB M 2LPM	Quoted at Time of Order
Liberators	13258385	FNL LIB 30T DF PEN M 15LPM 20	Quoted at Time of Order
Liberators	13259599	FNL LIB 37T DF PB M 15LPM 20	Quoted at Time of Order
Liberators	14454986	FNL LIB 45T DF CP M 15LPM 20	Quoted at Time of Order
Liberators	13261728	FNL LIB 45T DF PB M 15LPM 20	Quoted at Time of Order
Liberators	13262376BZ	FNL LIB 45T TF PB M 15LPM 20	Quoted at Time of Order
Liberators	13263707BZ	FNL LIB 45T TF TAE M 15LPM 20	Quoted at Time of Order
Liberators	13263758	FNL LIB 60T DF PB M 15LPM 20	Quoted at Time of Order
Sprint	10845218	FNL SPRINT T SF CA M 6LPM, METER	Quoted at Time of Order
Strollers	14779060	FNL STROLLER T SF CA M 6LPM, METER BLK	Quoted at Time of Order
Strollers	13259433	FNL STROLLER T TF PB F 2LPM, METER	Quoted at Time of Order
Strollers	11730957BZ	FNL STROLLER T TF PB F 6LPM, METER	Quoted at Time of Order
Strollers	14768264	FNL STROLLER T TF PB F 6LPM, METER BLK	Quoted at Time of Order

CAIRE will continue to be a leader in the manufacturing of Liquid Oxygen Systems, offering the broadest range of products and modalities to meet all oxygen needs.

For assistance in selecting the best replacement model for your needs or for parts needed to configure units into discontinued models, please contact your local Sales Representative or our Customer and Technical Service teams.

CONTACT: For technical questions, contact Technical Service:

North America: 800-482-2473 or +1-770-721-7700

Europe: +44 (0) 118 936 7060

Japan: 050-5050-1275

techservice.global@caireinc.com

For ordering information or general inquiries, contact Customer Service:

North America: 800-482-2473 or +1-770-721-7700

customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

customerservice.europe@caireinc.com

Japan: 050-5050-1275

customerservice.japan@caireinc.com

PN 21499364 A

RELEASE DATE: August 1, 2024

REVISION DATE(S):

MODEL: SAROS 4000™

ISSUE: SAROS 4000 Air Worthiness Rated (AWR)

NOTES: CAIRE is advising customers of our new compliance with RTCA-DO-160G Section 8 Categories S and U for SAROS 4000 concentrator.

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For fixed wing aircraft, a battery mounted to the Saros can be charged and discharged. For rotary wing aircraft, battery usage is application-dependent and not recommended by CAIRE. The changes do not impact existing regulatory clearances (such as 510K or CE requirements).

Mechanical and electrical changes were made to protect the SAROS under extreme vibration conditions. There are no changes to the overall function or performance of the concentrator.

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Copyright © 2024 CAIRE Inc. نحتفظ شركة CAIRE Inc بالحق في وقف منتجاتها، أو تغيير الأسعار و/أو المواد و/أو المعدات و/أو الجودة و/أو الأوصاف و/أو المواصفات و/أو العمليات الخاصة بمنتجاتها في أي وقت دون إشعار مسبق ودون أي التزام أو عواقب أخرى. جميع الحقوق غير المنصوص عليها صراحةً هنا محفوظة لنا، حسب الاقتضاء.
Copyright © 2024 CAIRE Inc. تحتفظ شركة CAIRE Inc بالحق في وقف منتجاتها، أو تغيير الأسعار و/أو المواد و/أو المعدات و/أو الجودة و/أو الأوصاف و/أو المواصفات و/أو العمليات الخاصة بمنتجاتها في أي وقت دون إشعار مسبق ودون أي التزام أو عواقب أخرى. جميع الحقوق غير المنصوص عليها صراحة هنا محفوظة لنا، حسب الاقتضاء.