

Sprint & Stroller



Figure 1 & 2: Configurable Diagram Meter

Key	Description	Sprint SF	Sprint TF	Stroller SF	Stroller TF	#
1	Dewar	11033828	11033828	11033861	11033861	1
2	Manifold (Design before 1.27.2020) *	10665401	10665381	21495020S	21495026S	1
	Manifold (Design after 1.27.2020) *	10665401	10665381	10665410	10665399	1
3	Vaporizer Coil	11033086	11033086	11891064	11891064	1
4	PRV Tube	10656185	10656185	10656185	10656185	1
5	Vent Outlet Tube	10665348	10656142	10665356	10656134	1
6	Top Bracket	10861568	10861568	10861576	10861576	1
8	FCV (0-6LPM) (Design before 1.27.2020) *	11014521T	11014521T	11014521T	11014521T	1
	FCV (0-6LPM) (Design after 1.27.2020) *	11014539TS	11014539TS	11014539TS	11014539TS	1
	FCV (0-2LPM) (Design before 1.27.2020) *	14700235T	14700235T	14700235T	14700235T	1
	FCV (0-2LPM) (Design after 1.27.2020) *	21366017S	21366017S	21366017S	21366017S	1
10	30 PSI SRV	10491253T	10491253T	10491253T	10491253T	1
11	20 PSI PRV w/screen	10528423	10528423	10528423	10528423	1
12	Gen4 Meter	13352401	13352401	13352401	13352401	1
13	PRV Filter Assembly	10932098	10932098	10932098	10932098	1
16	Lock Plate, FCV	CA406438	CA406438	CA406438	CA406438	1
17	Knob, FCV	10542153	10542153	10542153	10542153	1
19	Flow Rate Decal, FCV (0-6LPM)	CA406436	CA406436	CA406436	CA406436	1
	Flow Rate Decal, FCV (0-2LPM)	14700411	14700411	14700411	14700411	1
21	Feed-Thru Fitting	CA404841	CA404841	CA404841	CA404841	3
22	Ground Terminal, #8 spade	10597904	10597904	10597904	10597904	1
23	O-Ring, Manifold	10579423	10579423	10579423	10579423	1
27	Screw, Manifold #6-32*7/16 SS	CA404853	CA404853	CA404853	CA404853	4
28	Screw, Bracket/Meter #6-32*3/8 SS	CA002305	CA002305	CA002305	CA002305	5
29	Screw, FCV #6-32*1/2 SS	20724438	20724438	20724438	20724438	2
31	Lock Washer, Bracket/Manifold #6 SS	CA404996	CA404996	CA404996	CA404996	7
32	Ferrule 3/16	B-775068-00	B-775068-00	B-775068-00	B-775068-00	5
33	Nut, Tube 3/16	CA405087	CA405087	CA405087	CA405087	2
35	Strap, Vaporizer Coil	CA403730	CA403730	CA403730	CA403730	1
36	Fill Tube, QDV		10656126		10656118	1
38	Ferrule 1/4		B-775063-00		B-775063-00	2
40	Nut 1/4		B-775062-00		B-775062-00	2
	TF QDV Lip Seal		B-775260-00		B-775260-00	1
	TF QDV Inlet Fitting Seal		10656071		10656071	1
45	QDV, PB Top Fill Portable		10566420T		10566420T	1
	QDV, Penox Top Fill Portable		10743668T		10743668T	1
	QDV, LifeOx Top Fill Portable		10985974T		10985974T	1
	QDV, Taema Top Fill Portable		13708041T		13708041T	1
46	Lock Washer, PB TF QDV		10712052		10712052	1
48	Shoulder Strap	13234703	13234703	13234703	13234703	1
52	Screw, Case #8-32*1/4 SS	10621986	10621986	10621986	10621986	2

*Note: Reference Service Bulletin PN 21495031 and Service Bulletin PN 21368597.

 CAIRE®



Sprint & Stroller

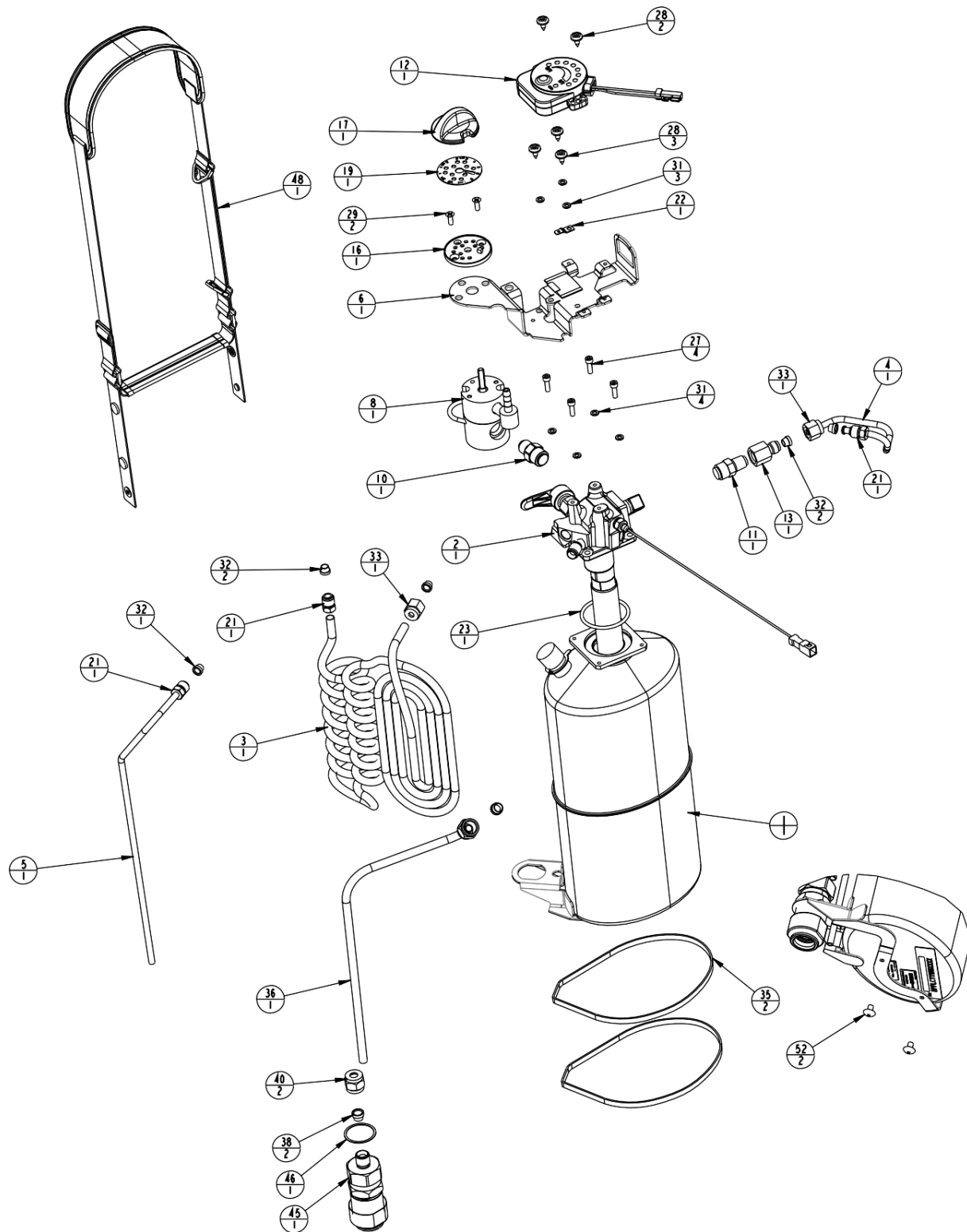


Figure 2: Configurable Diagram Top Fill with Meter

Sprint & Stroller



Figure 3 & 4: Configurable Diagram Scale

Key	Description	Sprint TF	Stroller SF	Stroller TF	#
1	Dewar	11033828	11033861	11033861	1
2	Manifold	11193151	11037554	11037546	1
3	Vaporizer Coil	11033086	11891064	11891064	1
4	PRV Tube	10656185	10656185	10656185	1
5	Vent Outlet Tube	10656142	10665356	10656134	1
6	Top Bracket	10861568	10861576	10861576	1
8	FCV (0-6LPM)	11014539TS	11014539TS	11014539TS	1
	FCV (0-2LPM)	14700235T	14700235T	14700235T	1
10	30 PSI SRV	10491288T	10491288T	10491288T	1
11	20 PSI PRV w/screen	10528423	10528423	10528423	1
13	SRV Filter Assembly	10932098	10932098	10932098	1
16	Lock Plate, FCV	CA406438	CA406438	CA406438	1
17	Knob, FCV	10542153	10542153	10542153	1
19	Flow Rate Decal, FCV (0-6LPM)	CA406436	CA406436	CA406436	1
	Flow Rate Decal, FCV (0-2LPM)	14700411	14700411	14700411	1
20	PRV Filter Assembly	CA405446	CA405446	CA405446	1
21	Feed-Thru Fitting	CA404841	CA404841	CA404841	4
23	O-Ring, Manifold	10579423	10579423	10579423	1
27	Screw, Manifold #6-32*7/16 SS	CA404853	CA404853	CA404853	4
28	Screw, Bracket/Meter #6-32*3/8 SS	CA002305	CA002305	CA002305	3
29	Screw, FCV #6-32*1/2 SS	20724438	20724438	20724438	2
31	Lock Washer, Bracket/Manifold #6 SS	CA404996	CA404996	CA404996	7
32	Ferrule, 3/16 Front	B-775068-00	B-775068-00	B-775068-00	7
33	Nut, Tube 3/16	CA405087	CA405087	CA405087	3
35	Strap, Vaporizer Coil	CA403730	CA403730	CA403730	2
36	TF QDV Fill Tube	10656126		10656118	1
38	Ferrule, 1/4 Front	B-775063-00		B-775063-00	2
40	Nut, Tube 1/4	B-775062-00		B-775062-00	2
43	SRV Coil	11037562	11037562	11037562	1
	TF QDV Lip Seal	B-775260-00		B-775260-00	1
	TF QDV Inlet Fitting Seal	10656071		10656071	1
45	QDV - TF PB	10566420T		10566420T	1
46	Lock Washer	10712052		10712052	1
48	Shoulder Strap Assembly w/ Scale	11013085	10985780	10985780	1
	Svce Shoulder Strap	13234703	13234703	13234703	1
	Svce Spring Scale Assembly	13417307	11368134	11368134	1
52	Screw, Case #8-32*1/4 SS	10621986	10621986	10621986	1

*Note: Reference Service Bulletin PN 21495031 and Service Bulletin PN 21368597.

Sprint & Stroller

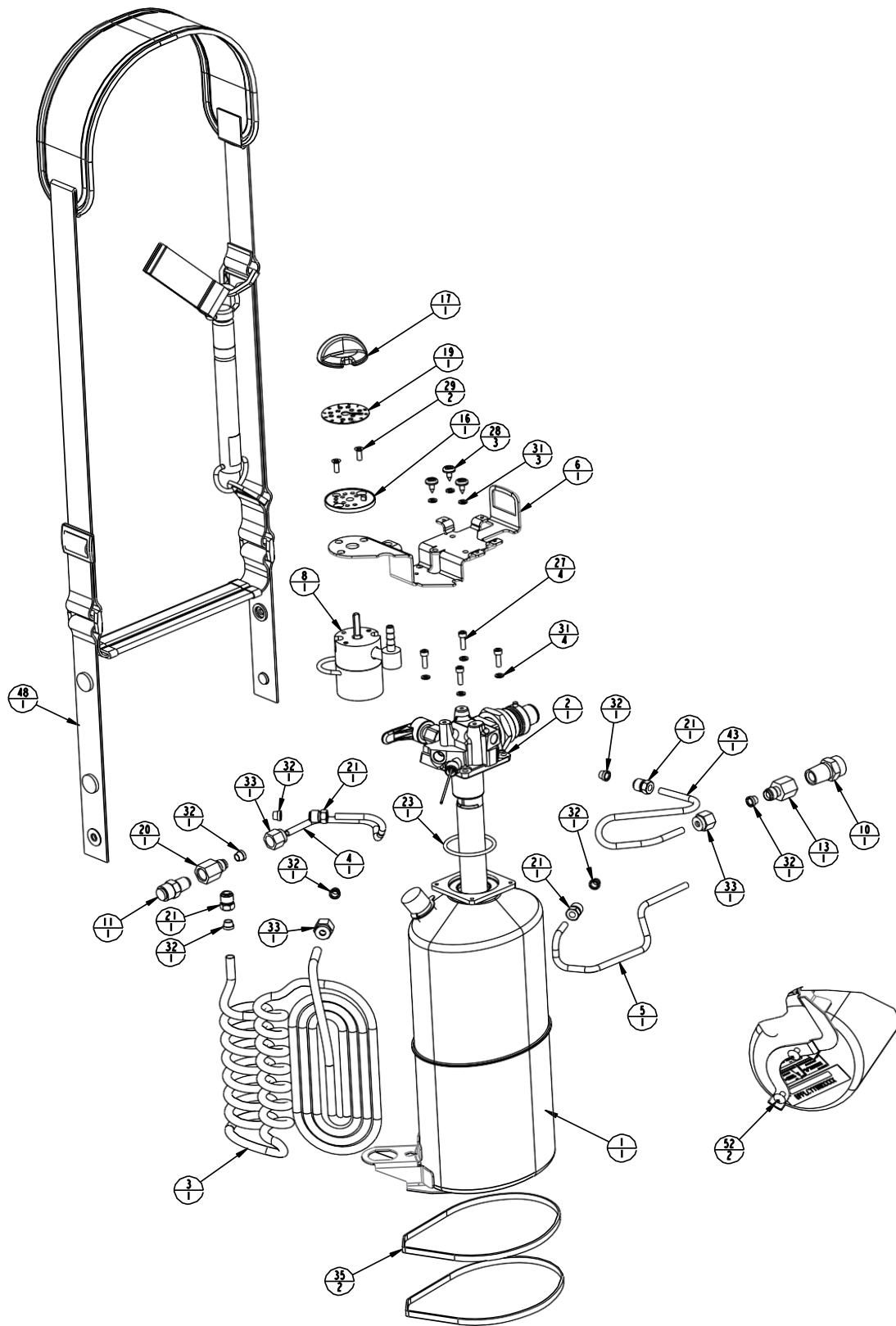


Figure 3: Configurable Diagram Side Fill with Scale

 CAIRE®



Sprint & Stroller



Figure 5-6-7: Configurable Diagram Manifold

Key	Description	Sprint SF	Sprint TF	Stroller SF	Stroller TF	#
1	Manifold Plate	11652258	11652215	11652186	11652282	1
3	Vent Valve (auto-close)	14004395	14004395	14004395	14004395	1
7	Fitting, Vaporizer Coil, SS	CA404839	CA404839	CA404839	CA404839	1
16	Retaining Ring, SF QDV	10776769		10776769		1
17	QDV, Side Fill, Caire	10577866T		10577866T		1
17	Elbow Connector, Top Fill QDV		10636395		10636395	1
18	Wire Harness (Model with G4 Meter only)	21360363	21360363	21360363	21360363	1
22	Manifold Tee (Design after 1.27.2020) *			21237266	21237266	1
	Street Tee (Design after 1.27.2020) *	21385560	21385560			1
23	Feed-Thru Fitting (Design after 1.27.2020) *			CA404841	CA404841	2
24	Ferrule 3/16 (Design after 1.27.2020) *			B-775068-00	B-775068-00	1
25	30PSI SRV (Design after 1.27.2020) *			10491253T	10491253T	1

*Note: Reference Service Bulletin PN 21495031 and Service Bulletin PN 21368597.

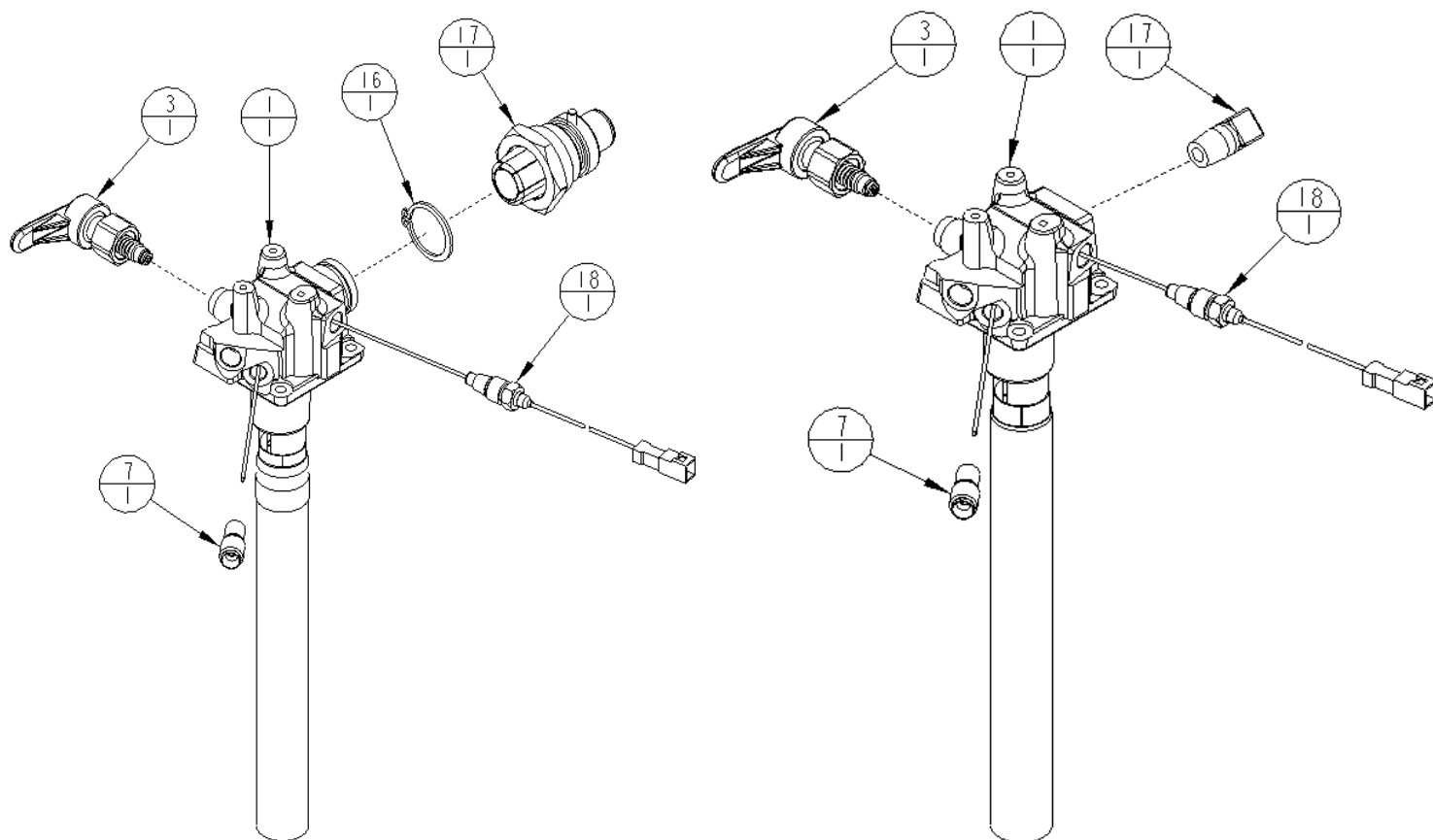


Figure 5: Side Fill (Left) and Top Fill (Right) Manifold Stroller (Design before 1.27.2020)

Sprint & Stroller

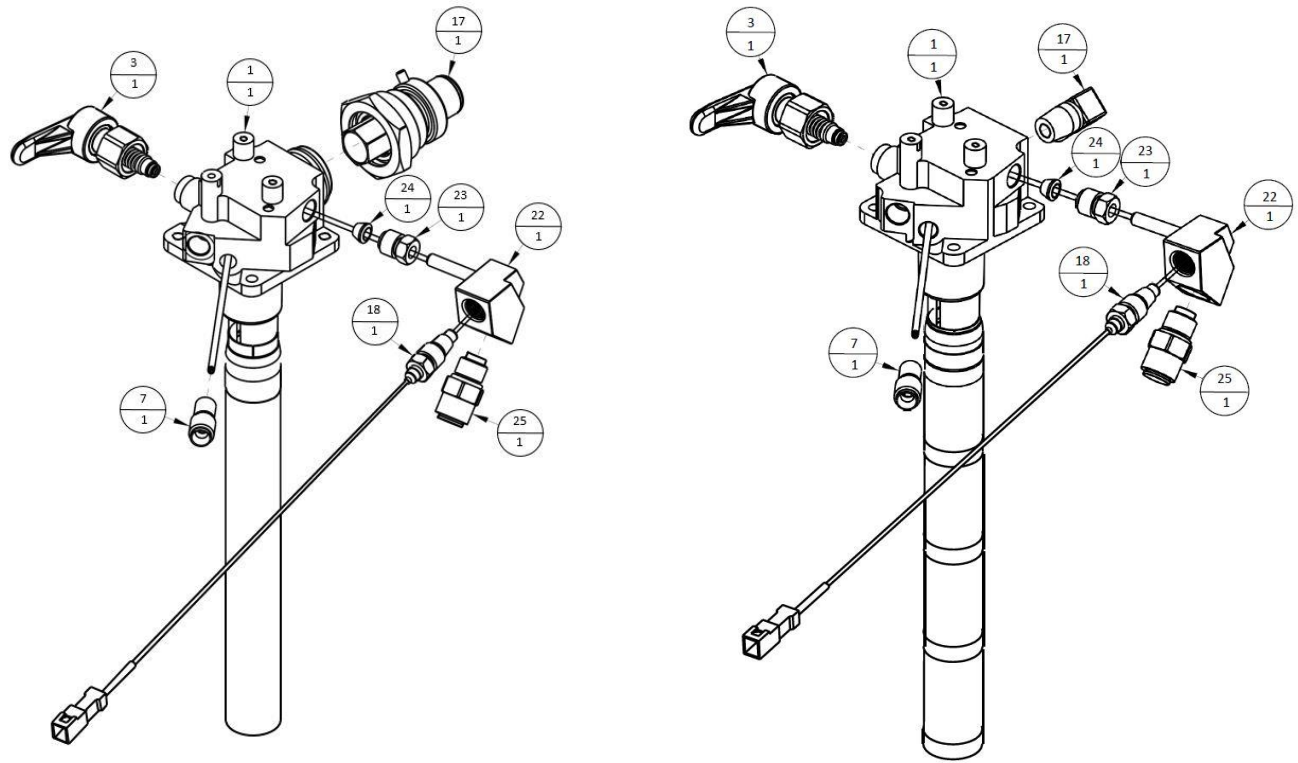


Figure 6: Side Fill (Left) and Top Fill (Right) Manifold Stroller (Design after 1.27.2020)

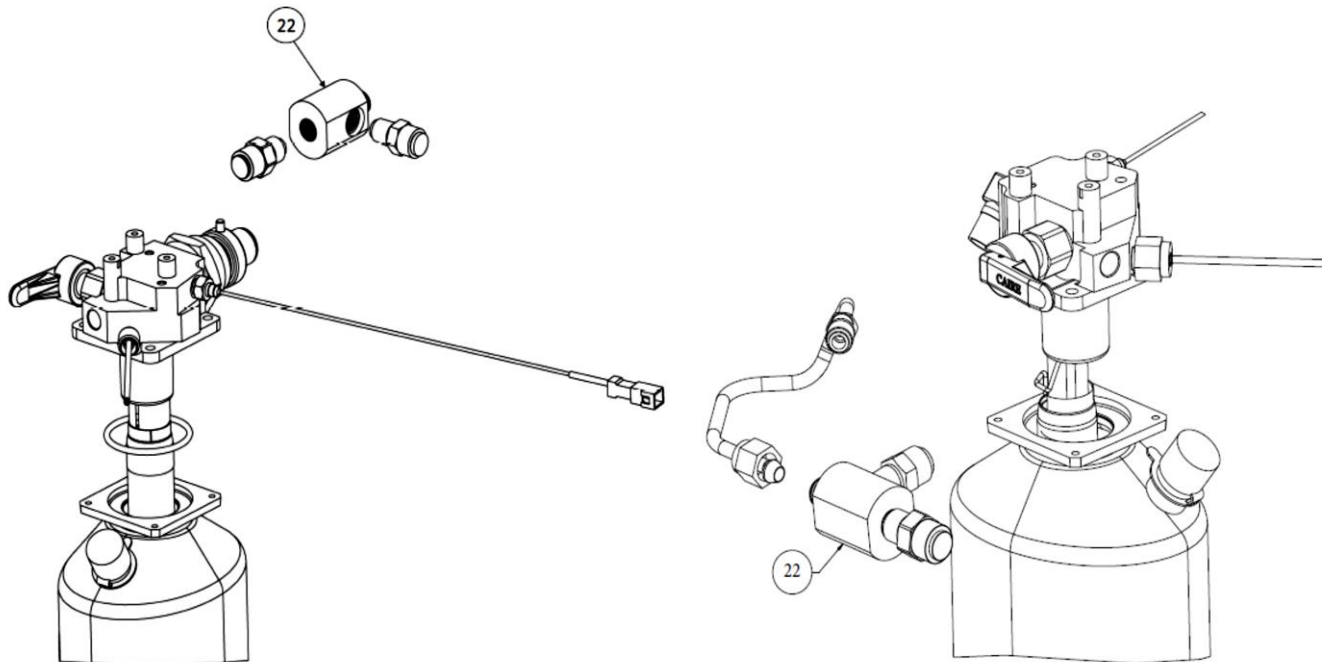


Figure 7: Side Fill (Left) and Top Fill (Right) Manifold Sprint (Design after 1.27.2020)

Sprint & Stroller



Figure 8: Configurable Diagram Case

Key	Description	Sprint SF	Sprint TF	Stroller SF	Stroller TF	#
NA	Top Label - All Languages	13712200	13712200	13734741	13734741	1
NA	Symbolic Label Kit Non CE*	21194754	21194754	21194754	21194754	1
NA	Symbolic Label Kit CE0029 2006 - May 7,2016*	21152754	21152754	21152754	21152754	1
NA	Symbolic Label Kit CE0459 May 9,2016-Present*	21150537	21150537	21150537	21150537	1
NA	Sprint-Stroller User Manual multi-languages	MN236-C4	MN236-C4	MN236-C4	MN236-C4	1
1	Condensate Pad	10542209	10542209	10542209	10542209	1
2	Condensate Cup - Blue	10542057	10542057	10542057	10542057	1
	Condensate Cup - Light Gray	10941681	10941681	10941681	10941681	1
	Condensate Cup - Dark Gray	10568249	10568249	10568249	10568249	1
3	Service Case Assy - Blue - G4	14037285	14037285	14015570	14015570	1
	Service Case Assy - Light Gray - G4	14037293	14037293	14015588	14015588	1
	Service Case Assy - Dark Gray - G4	14037277	14037277	14015561	14015561	1
	Service Case Assy - Blue - G4 - Wide			14015617	14015617	1
	Service Case Assy - Light Gray - G4 - Wide			14015596	14015596	1
	Service Case Assy - Dark Gray - G4 - Wide			14015609	14015609	1
6	Top Cover - Blue	10564328	10564328	10564310	10564310	1
	Top Cover - Blue - Wide			11668313	11668313	1
	Top Cover - Light Gray	10978281	10978281	10941701	10941701	1
	Top Cover - Light Gray - Wide			11700627	11700627	1
	Top Cover - Dark Gray	10542049	10542049	10562111	10562111	1
	Top Cover - Dark Gray - Wide			11700601	11700601	1
7	SF QDV Plug	10542137	10564361	10542137	10564361	1
8	Grommet, Condensate Cup	CA110033	CA110033	CA110033	CA110033	1
9	Plunger, Condensate Cup	CA110034	CA110034	CA110034	CA110034	1
10	TF QDV Plug	10821363	10935117	10821363	10935117	1
11	Screw, Top Cover #6-32*3/8 SS	CA002305	CA002305	CA002305	CA002305	3
12	Dome Cover - Blue - Meter	12932526	12932526	12932526	12932526	1
	Dome Cover - Light Gray - Meter	12932542	12932542	12932542	12932542	1
	Dome Cover - Dark Gray - Meter	12932534	12932534	12932534	12932534	1
	Dome Cover - Blue - Scale	11199511	11199511	11199511	11199511	1
	Dome Cover - Light Gray - Scale	11199570	11199570	11199570	11199570	1
	Dome Cover - Dark Gray - Scale	11199545	11199545	11199545	11199545	1

* Reference Service Bulletin 21174579. Bottom labels in all languages have been replaced by the symbolic label.

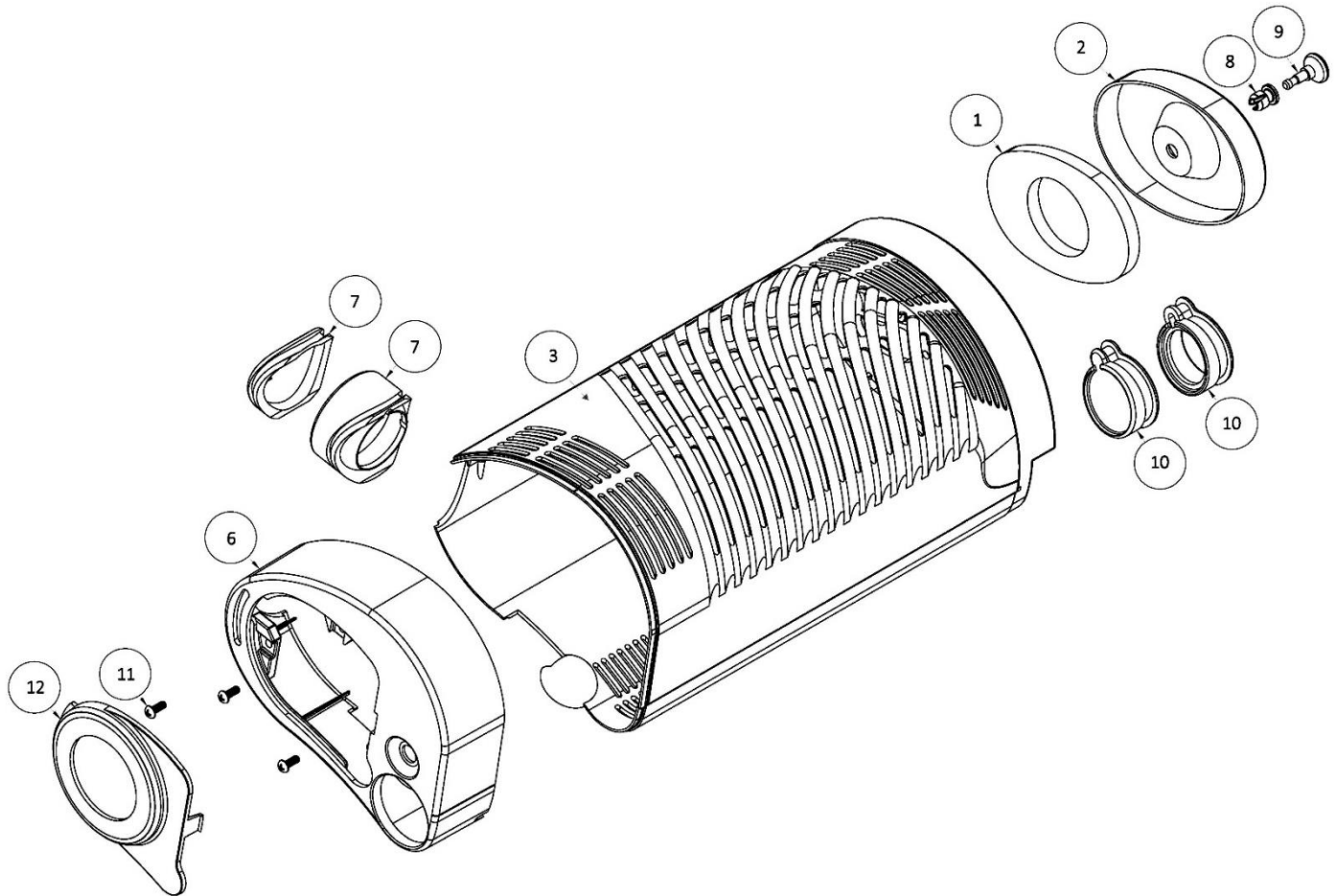


Figure 8: Configurable Diagram Case

Copyright © 2020 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.