

Miscellaneous Parts

TF Item Number	SF Item Number	Description	Quantity
14083791	14083791	Top Label - All Languages	1
*21194754	*21194754	Symbolic Label Kit Non CE	1
*21152754	*21152754	Symbolic Label Kit CE0029 2006 - May 7, 2016	1
*21150537	*21150537	Symbolic Label Kit CE0459 May 9, 2016 - Present	1
MN236-C4	MN236-C4	Hi Flow Stroller User Manual multi-languages	1

*Note: Reference Service Bulletin 21174579. Bottom labels for all languages have been replaced by the symbolic label.

Figures 1 & 2: Configurable Parts

SF Item Number	TF Item Number	Description	Quantity	Bubble Seq No
11033861	11033861	Dewar	1	1
21495020S	21495026S	Manifold (Design before 1.27.2020)	1	2
10564619S	10597787S	Manifold (Design after 1.27.2020)	1	2
12934695	12934695	Vaporizer Coil	1	3
10656185	10656185	PRV Coil	1	4
10665356	10656134	Vent Outlet Tube	1	5
10861576	10861576	Top Bracket	1	6
14034703T*	14034703T*	FCV (0 -15 LPM) (Design before 1.27.2020)	1	8
14812860TS*	14812860TS*	FCV (0 -15 LPM) (Design after 1.27.2020)	1	8
13110225LAN*	13110225LAN*	65PSI SRV (Design before 1.27.2020)	1	10
See Figures 3 & 4	See Figures 3 & 4	30PSI SRV (Design after 1.27.2020)	1	10
10528423	10528423	20PSI PRV w/screen	1	11
13352401	13352401	Gen4 Meter	1	12
CA406438	CA406438	Lock Plate, FCV	1	16
10542153	10542153	Knob, FCV	1	17
CA406437	CA406437	Flow Rate Decal, FCV (0-15 LPM)	1	19
10932098	10932098	PRV Filter Assy	1	20
CA404841	CA404841	Feed-Thru Fitting	2	21
10597904	10597904	Ground Terminal, #8 spade	1	22
10579423	10579423	O-Ring Manifold	1	23
CA404853	CA404853	Screw, Manifold #6-32*7/16 SS	4	27
CA002305	CA002305	Screw, Bracket #6-32*3/8 SS	3	28
20724438	20724438	Screw, FCV #6-32*3/8 SS	2	29
14263928	14263928	Screw, Meter #6-32*3/8 SS	2	30
CA404996	CA404996	Lock Washer, Bracket/Manifold #6 SS	7	31
B-775068-00	B-775068-00	Ferrule 3/16	6	32
CA405087	CA405087	Nut, Tube 3/16	2	33
CA403730	CA403730	Strap Vaporizer Coil	3	35
	10656118	Fill Tube, QDV	1	36

*Note: Reference Service Bulletin PN 21495031

Figures 1 & 2: Configurable Parts (continued)

SF Item Number	TF Item Number	Description	Quantity	Bubble Seq No
	B-775063-00	Ferrule 1/4	2	38
	B-775262-00	Nut 1/4	2	40
	10566420T	QDV, PB Top Fill Portable	1	45
	10743668T	QDV, Penox Top Fill Portable	1	
	10985974T	QDV, LifeOx Top Fill Portable	1	
	13708041T	QDV, Taema Top Fill Portable	1	
	B-775260-00	TF QDV Lip Seal	1	
	10712052	Lock Washer - TF QDV	1	46
13234703	13234703	Shoulder Strap	1	48
10621986	10621986	Screw, #8-32*1/4 SS	2	52
14125390	14125390	Insulator	1	53
CA406358	CA406358	Heat Shield	1	54
14122421S	21015163	Svce Case Assy - Black	1	55
14507259S	14507232S	Svce Case Assy - Gray	1	55
10542209	10542209	Condensate Pad	1	56
14400697-SV	14400697-SV	Vaporizer Coil Extension	1	57
14400689	14400689	Fitting, 3/16 Coupling	1	58

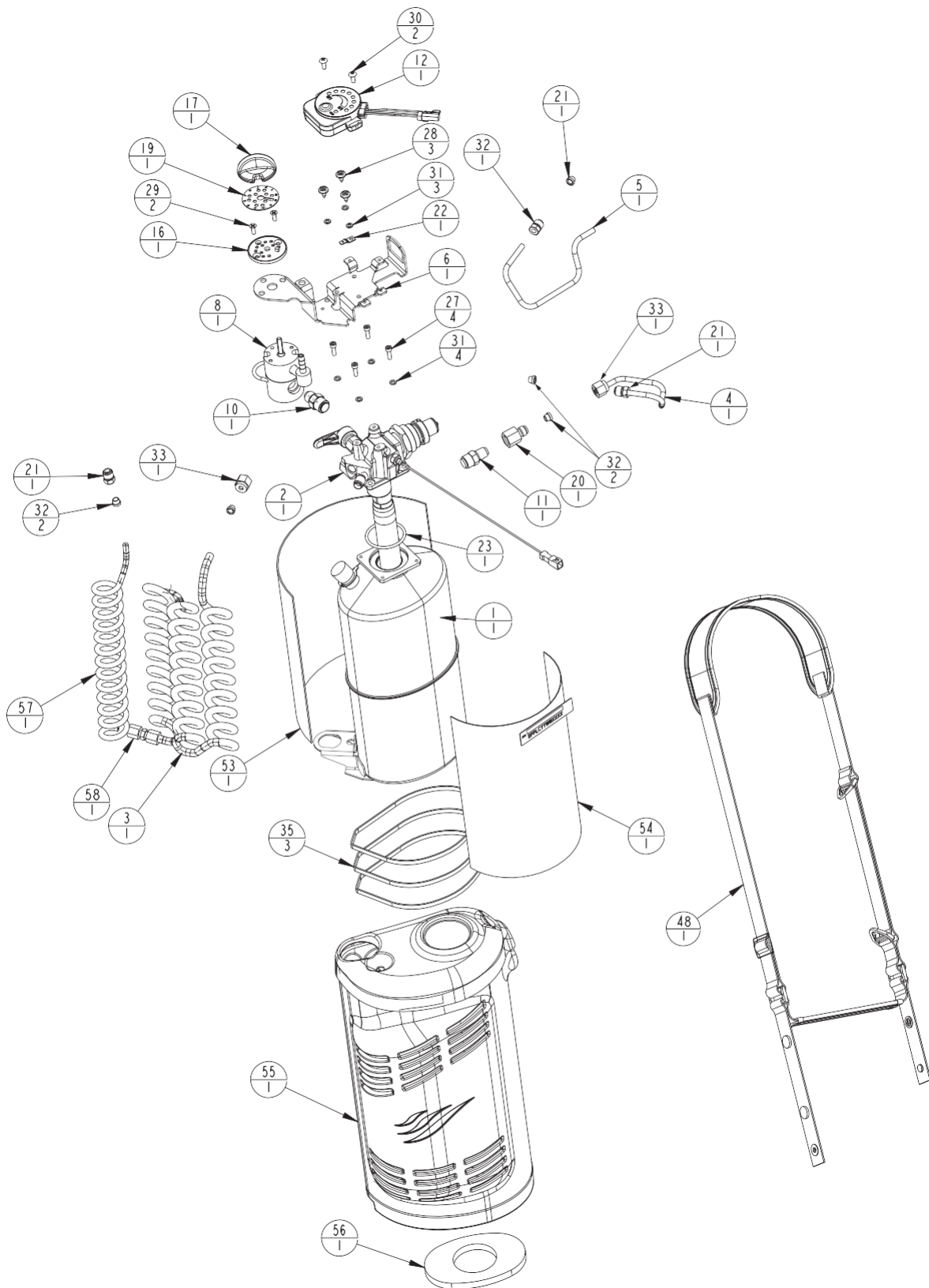


Figure 1: Side Fill Configurable Parts

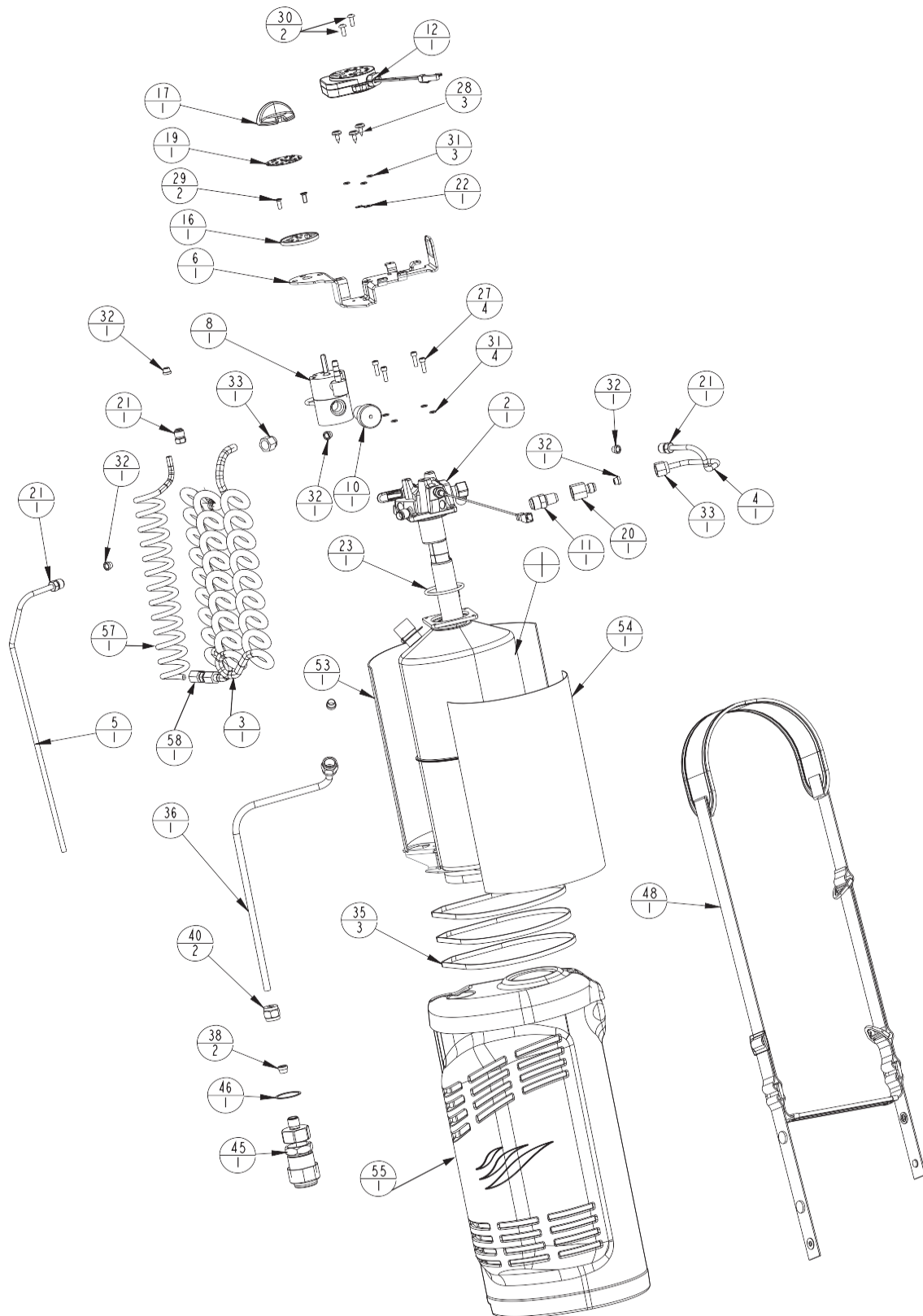


Figure 2: Top Fill Configurable Parts

Figures 3 & 4: Manifold

SF Item Number	TF Item Number	Description	Quantity	Bubble Seq No
11652186	11652282	Manifold Plate	1	1
14004395	14004395	Vent Valve (auto-close)	1	3
CA404839	CA404839	Fitting, Vaporizer Coil, SS	1	7
10776769		Retaining Ring, SF QDV	1	16
10577866T		QDV, Side Fill, Caire	1	17
	10636395	Elbow Connector, Top Fill QDV	1	17
21360363	21360363	Wire Harness, G4 Meter	1	18
21237266	21237266	Manifold Tee	1	22
CA404841	CA404841	Feed-Thru Fitting	2	23
B-775068-00	B-775068-00	Ferrule 3/16	1	24
10491253T	10491253T	30PSI SRV (Design after 1.27.2020)	1	25

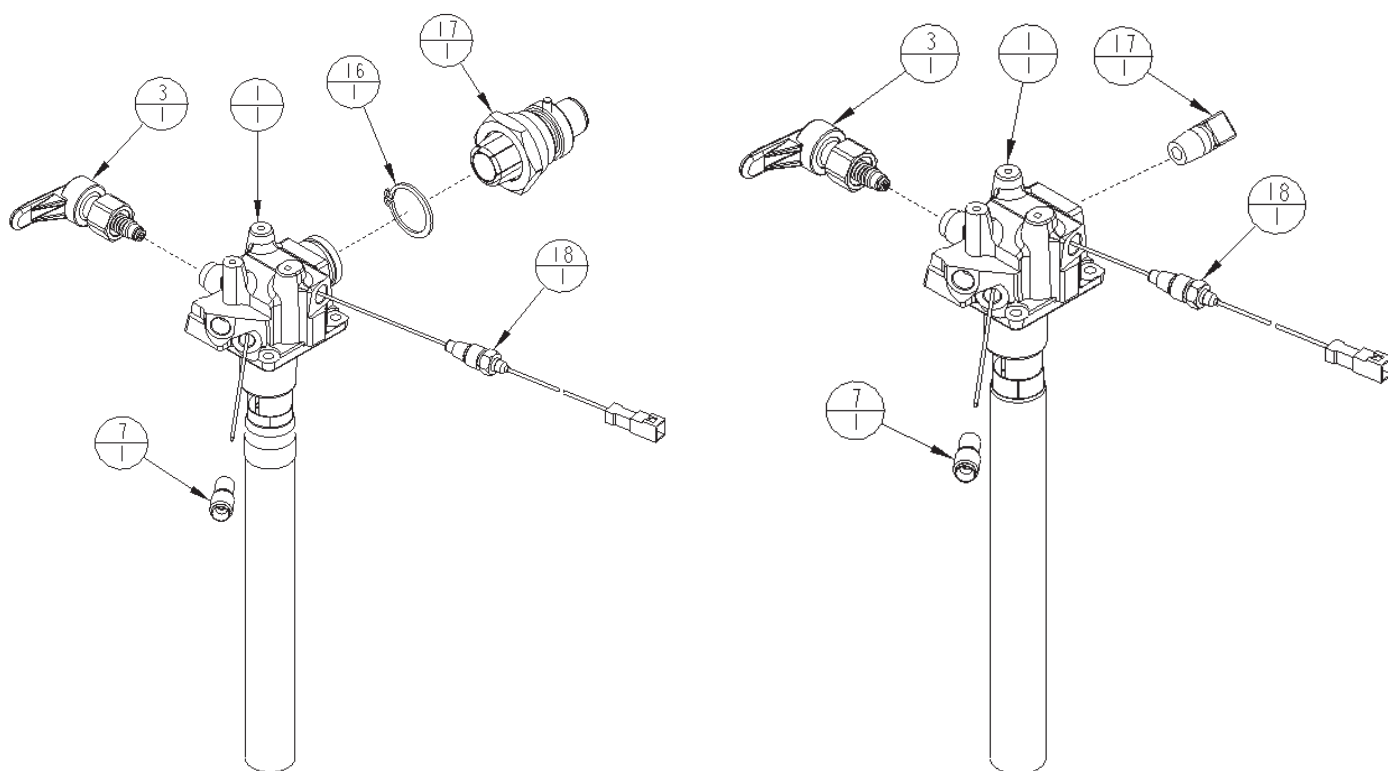


Figure 3: Side and Top Fill Manifold (Design before 1.27.2020)

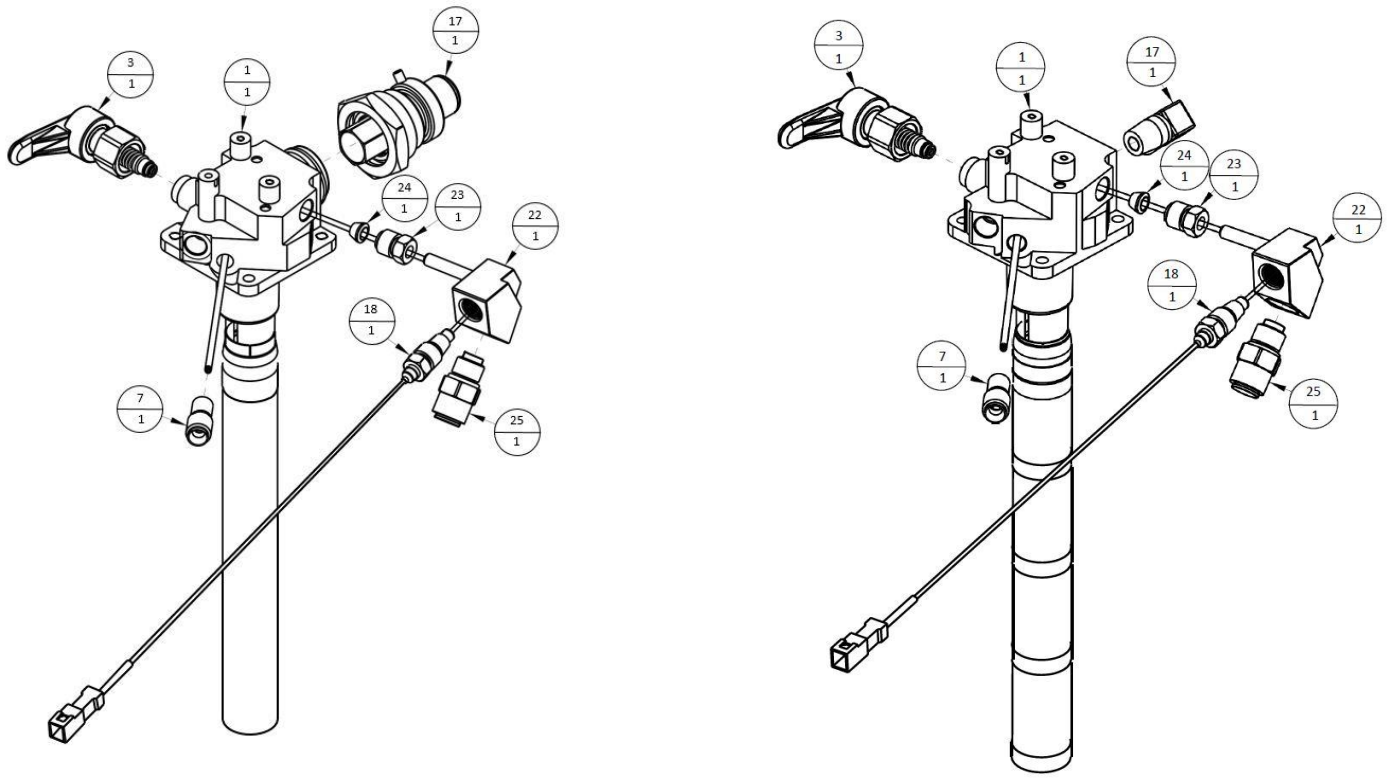


Figure 4: Side and Top Fill Manifold (Design after 1.27.2020)