



SERVICE BULLETIN

PN: 21464617

RELEASE DATE: September 3, 2019

REVISION DATE(S): September 23, 2019 (Rev B)
February 01, 2020 (Rev C)

MODEL: SeQual[®] Eclipse 5[®]

ISSUE: New Release Eclipse 5 Power Supply

NOTES: Effective September 2019 Eclipse power supply (CAIRE PN:21334759) will replace the previous version (CAIRE PN: 20852326) on all newly manufactured 2-LED Eclipse 5 devices as well as all spare power supply part kit numbers in Table 1 below.

Table 1: Spare Power Supply Kit PNs

20852326S	Kit, Power Supply US Cord
20852325S	Kit, Power Supply EU Cord
20852324S	Kit, Power Supply UK Cord

Power supply 21334759 is backwards compatible with all previously manufactured 2-LED and 3-LED Eclipse 5 units.

Note: Please reference Service Bulletin [21220048](#) when using the 3-LED power supply kit PN 5941-SEQ (US), 20553100 (EU), and 20553098 (UK)) with 2-LED units as this will require label changes on the device.

Figures 1 and 2 display the different power connections between the new part number 21334759 and the previous part number 20852326. The 21334759 new style power supplies will require different power cable – see Table 2 for details of different cord part numbers.



Figure 1: Current Eclipse 5 power supply (PN 21334759)



Figure 2: Previous Eclipse 5 power supply (PN 20852326)

Figure 3 displays the power supply connection for use on the 3-LED version Eclipse models. These part numbers are obsolete, replaced by the power supply kits listed above.



Figure 3: Eclipse 3-LED Power Supply (PN 5940-SEQ)

Table 2: – Power Cord part numbers

Power Supply PN	Region	Cord only PN	Spare Power Supply Plus Cord
5940-SEQ	USA	3588-SEQ	20852326S
5940-SEQ	UK	3590-SEQ	20852324S
5940-SEQ	EU	3589-SEQ	20852325S
5940-SEQ	AUS	20669418	21334759 + 21463776
20852326	USA	4997-SEQ	20852326S
	UK	4999-SEQ	20852324S
	EU	4998-SEQ	20852325S
	AUS	21347183	21334759 + 21463776
21334759	USA	21336129	20852326S
	UK	21475874	20852324S
	EU	21336073-C4	20852325S
	AUS	21463776	21334759 + 21463776

*Use above table in conjunction with Figure 1-3 to identify cable only part number required.

CONTACT: For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

techservice.global@caireinc.com

Europe: +44 (0) 118 936 7060

techservice.global@caireinc.com

For ordering information or general inquires, contact Customer Service:

USA: 800-482-2473

customerservice.usa@caireinc.com

Europe: +44 (0) 118 936 7060

customerservice.europe@caireinc.com

Copyright © 2020 CAIRE Inc. CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.