PN 21499071

**RELEASE DATE:** May 1, 2024

**MODEL:** FreeStyle Comfort™

**ISSUE:** FreeStyle Comfort™ Software V 2.9.1 Update

**NOTES:** CAIRE is releasing a software update to all FreeStyle Comfort models. Version 2.9.1 firmware changes the audible signal for 'No Breath Detected' and 'Low Concentration' from activating every 30 seconds to activating every 10 minutes.

This change also discontinues the 'Breath Rate Exceeded' alarm from repeating every 60 seconds.

Additionally, this update changes the audible signal for 'Low Battery Warning' from activating at 10% battery level to 20% battery level while showing the below display.

There are no changes to the overall function or usage of the concentrator, but the new software changes the alarm priorities.

Please refer below for the new Alarm Conditions Table listing the changes to alarms priority.

Priority #	Alarm Description	Audible Signal
1	System Malfunction Battery High Temperature	1 beep every 20 sec
2	System Malfunction High Temperature	1 beep every 20 sec
3	System Malfunction High Pressure (High System Pressure or Compressor Fault)	1 beep every 20 sec
4	System Malfunction Flow Rate (OCD Valve Failure, Obstruction of Gas Pathway)	1 beep every 30 sec
5	Low Battery Shut Down	1 beep every 30 sec
6	No Breath Detected	1 beep every 10 min
7	Warm Up	none
8	Low Concentration	1 beep every 10 min, for up to 30 min
9	Low Battery Warning	1 beep, no repeat
10	Breathing Rate Exceeded	1 beep, no repeat
11	System Malfunction Fan Failure	none
12	System Malfunction Low Pressure	none
13	System Malfunction Low Temp	none

Note: Devices in the field do not require a mandatory firmware update.

Previous Version	Updated Version
2.9.0 or earlier	2.9.1

**CONTACT:** For technical questions or concerns, contact Technical Service:

USA: 800-482-2473

Europe: +44 (0) 118 936 7060

[techservice.global@caireinc.com](mailto:techservice.global@caireinc.com)

For ordering information or general inquiries, contact Customer Service:

USA: 800-482-2473

[customerservice.usa@caireinc.com](mailto:customerservice.usa@caireinc.com)

Europe: +44 (0) 118 936 7060

[customerservice.europe@caireinc.com](mailto:customerservice.europe@caireinc.com)

Copyright © 2024 CAIRE Inc.

CAIRE Inc. reserves the right to discontinue its products, or change the prices, materials, equipment, quality, descriptions, specifications and/or processes to its products at any time without prior notice and with no further obligation or consequence. All rights not expressly stated herein are reserved by us, as applicable.

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Copyright © 2024 CAIRE Inc. CAIRE Inc. שומרת לעצמה את הזכות להפסיק את מכירת מוצריה, או לשנות את המחירים, החומרים, הציוד, האיכות, התיאורים, המפרטים ו/או התהליכים ביחס למוצריה בכל עת ללא הודעה מוקדמת וללא התחייבות ומבלי לשאת בהשלכות כתוצאה מהמוזכר לעיל. כל הזכויות שלא צוינו במפורש במסמך זה שמורות לנו, לפי העניין.
Copyright © 2024 CAIRE Inc. نحتفظ شركة CAIRE Inc. بالحق في وقف منتجاتها، أو تغيير الأسعار و/أو المواد و/أو المعدات و/أو الجودة و/أو الأوصاف و/أو المواصفات و/أو العمليات الخاصة بمنتجاتها في أي وقت دون إشعار مسبق ودون أي التزام أو عواقب أخرى. جميع الحقوق غير المنصوص عليها صراحةً هنا محفوظة لنا، حسب الاقتضاء.
Copyright © 2024 CAIRE Inc. تحتفظ شركة CAIRE Inc بالحق في وقف منتجاتها، أو تغيير الأسعار أو المواد أو المعدات أو الجودة أو الأوصاف أو المواصفات أو العمليات الخاصة بمنتجاتها في أي وقت دون إشعار مسبق ودون أي التزام أو عواقب أخرى. جميع الحقوق غير المنصوص عليها صراحة هنا محفوظة لنا، حسب الاقتضاء.